



nova

319

Parts Breakdown

*For models: 319B, 319BK, 319C, 319R
Serial # Starting "CHT" & "CH"*



An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your transport chair corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (**see Figure 1**).


If you need further assistance, please call our customer service team at **(800) 557-6682**.

Thank you!




Figure 1


Handle and Folding Mechanism Components

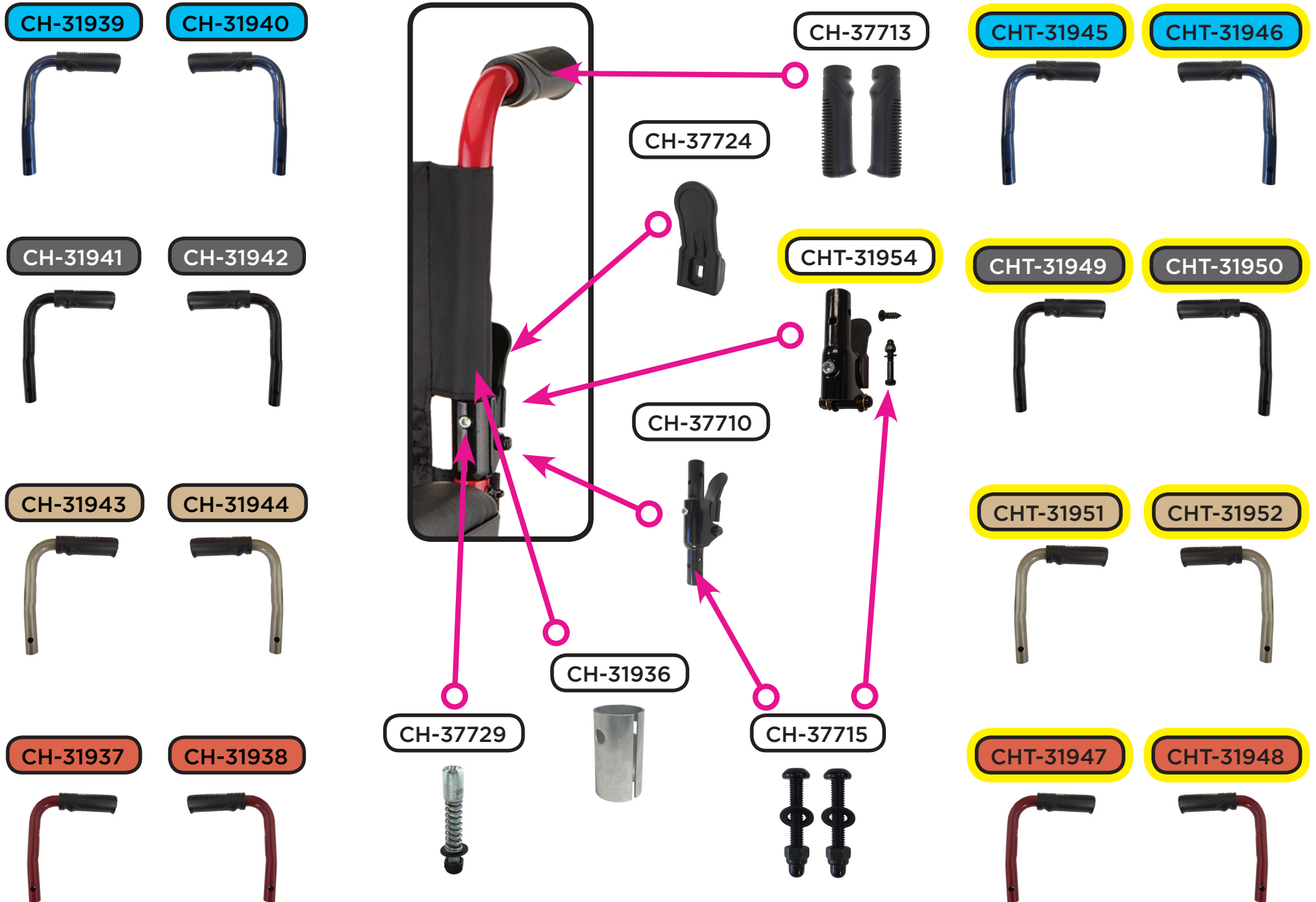


Main Body Components

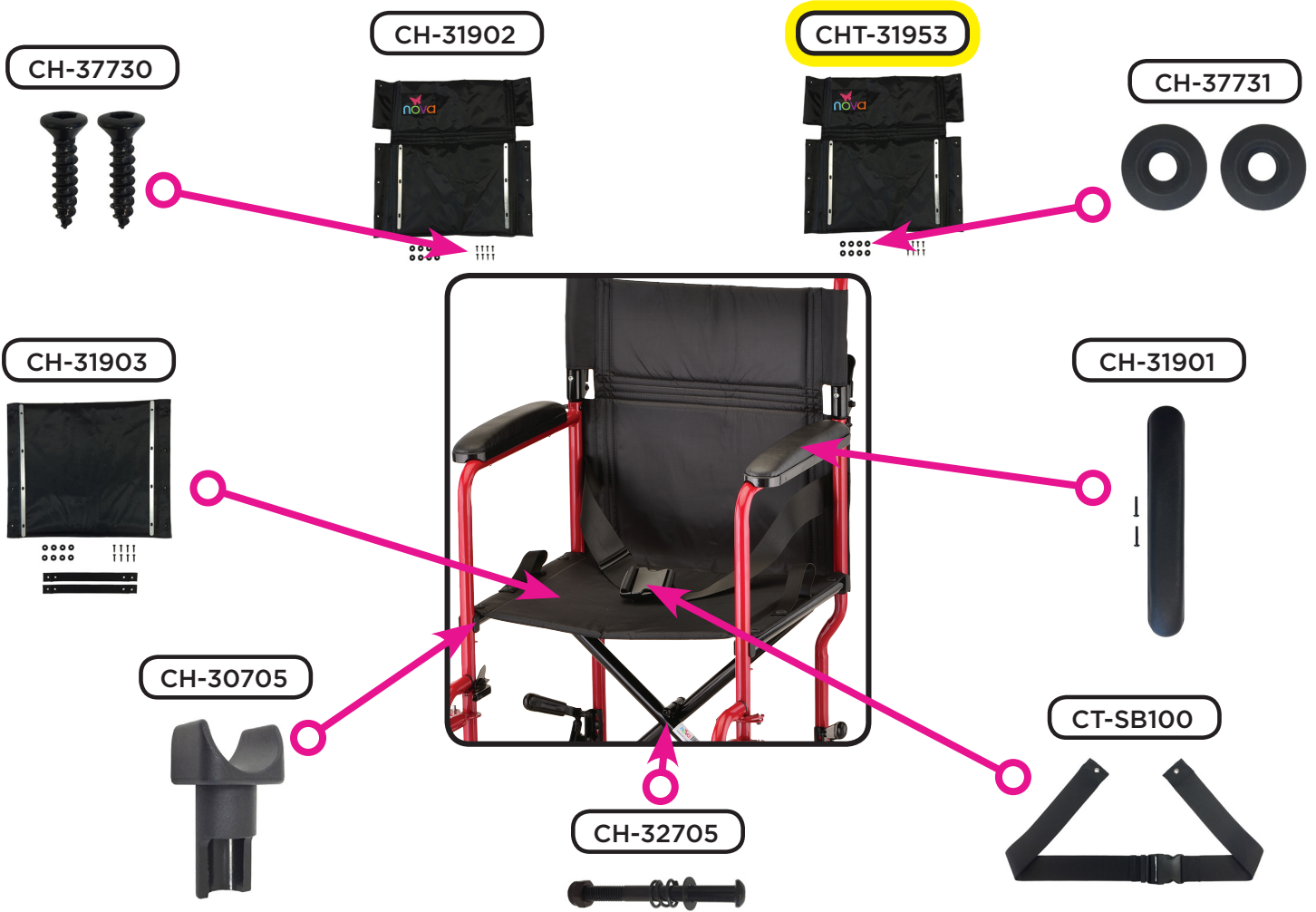


Footrest Assembly

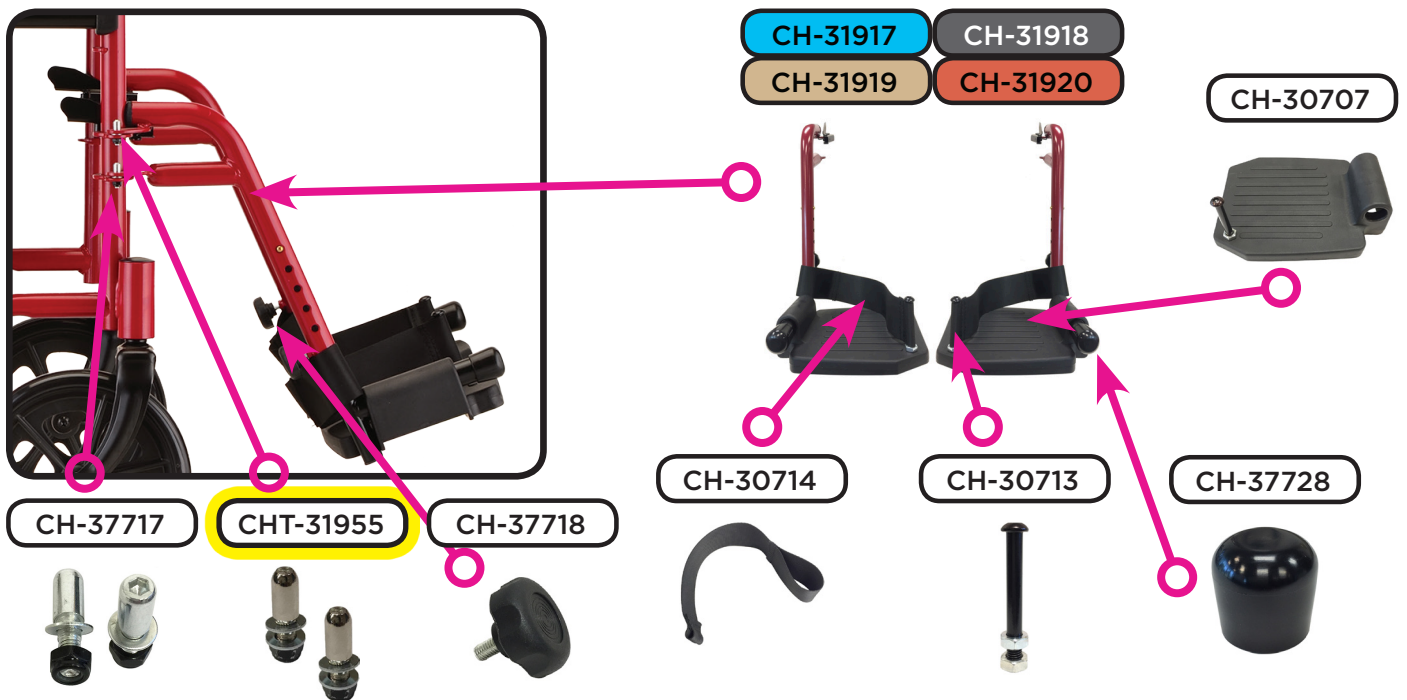

Front Wheel Assembly

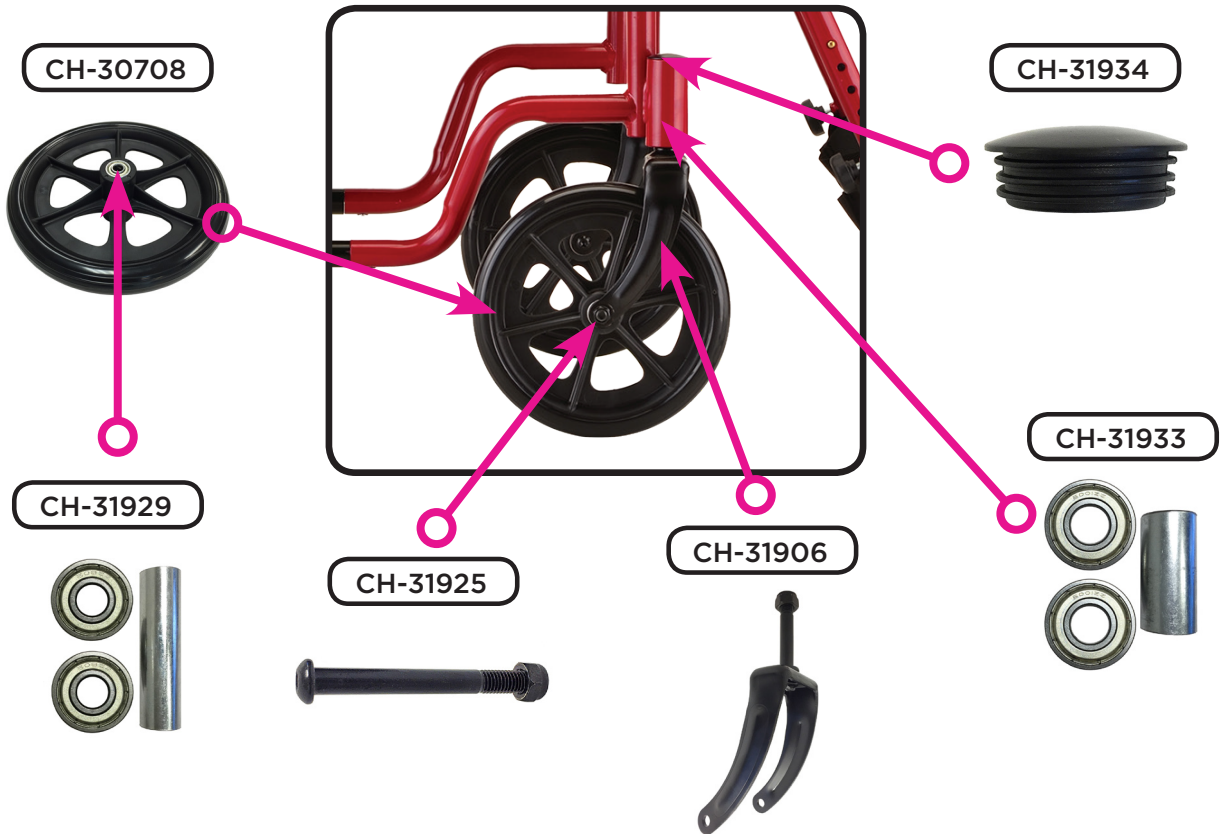

Rear Wheel Assembly

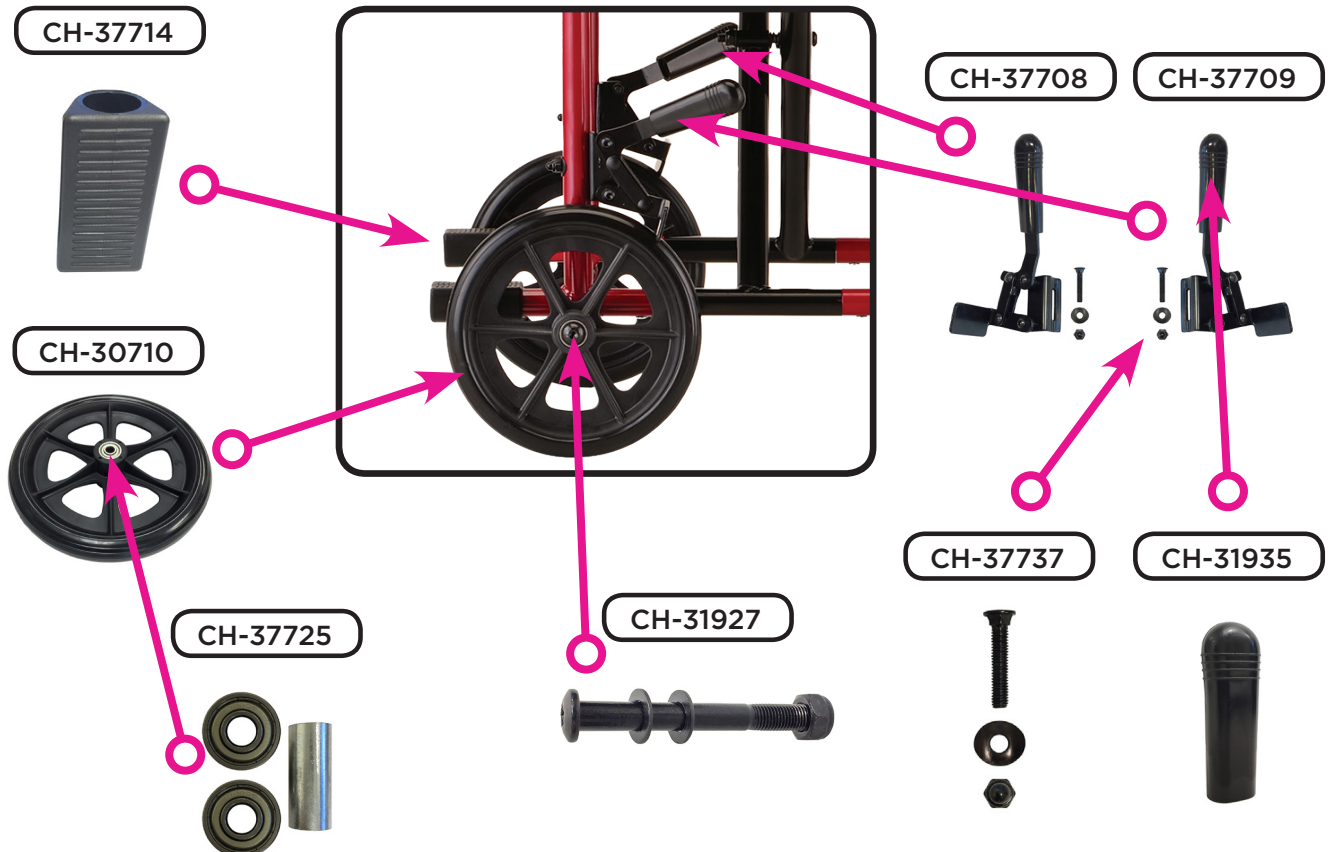



Item Number 319

319 Transport Chair

SERIAL # BEGINNING WITH "CHT" & "CH"

ITEM NUMBER	DESCRIPTION
CH-31901	ARM PAD FOR 309/319/327/329/330
CH-31925	AXLE SET FOR FRONT WHEEL 309/3
CH-31927	AXLE SET FOR REAR WHEEL 309/31
CH-31929 (SET)	BEARING & SPACER FOR FRONT WHEEL
CH-37725	BEARING & SPACER FOR REAR WHEEL
CH-31933 (SET)	BEARINGS FOR FRONT FORK 307/30
CH-37708	BRAKE LEFT 307/309/319/327/329
CH-37709	BRAKE RIGHT 307/309/319/327/32
CH-37728	CAP FOR FOOT REST
CH-37710	FOLDING MECHANISM
* CHT-31954	FOLDING MECHANISM (UPPER) FOR 319/327/329/330/377/379
CH-30707	FOOT PLATE (INCLUDES SCREW FOR HEEL STRAP)
CH-31917 (PAIR)	FOOT REST (PAIR) FOR 319B
CH-31918 (PAIR)	FOOT REST (PAIR) FOR 319BK/329
CH-31919 (PAIR)	FOOT REST (PAIR) FOR 319C SN:
CH-31920 (PAIR)	FOOT REST (PAIR) FOR 319R SN:
CH-31906	FORK FOR FRONT WHEEL 319/327/3
CH-37713	GRIP FOR HANDLE
CH-31941	HANDLE LEFT FOR 319 BLACK
* CHT-31949	HANDLE LEFT FOR 319 BLACK
CH-31939	HANDLE LEFT FOR 319 BLUE
* CHT-31945	HANDLE LEFT FOR 319 BLUE
CH-31943	HANDLE LEFT FOR 319 CHAMPAGNE
* CHT-31951	HANDLE LEFT FOR 319 CHAMPAGNE
CH-31937	HANDLE LEFT FOR 319 RED
* CHT-31947	HANDLE LEFT FOR 319 RED
CH-31942	HANDLE RIGHT FOR 319 BLACK
* CHT-31950	HANDLE RIGHT FOR 319 BLACK
CH-31940	HANDLE RIGHT FOR 319 BLUE
* CHT-31946	HANDLE RIGHT FOR 319 BLUE
CH-31944	HANDLE RIGHT FOR 319 CHAMPAGNE
* CHT-31952	HANDLE RIGHT FOR 319 CHAMPAGNE
CH-31938	HANDLE RIGHT FOR 319 RED
* CHT-31948	HANDLE RIGHT FOR 319 RED
CH-30714	HEEL STRAP
CH-37718	KNOB FOR FOOT REST

Item Number 319	
319 Transport Chair	
SERIAL # BEGINNING WITH "CHT" & "CH"	
ITEM NUMBER	DESCRIPTION
CH-37724	LEVER FOR FOLDING MECHANISM
CHT-31955 (PAIR)	PIN (PAIR) FOR FOOTREST FOR 319/327/329/330/377/379
* CH-37717 (PAIR)	PIN FOR FOOT REST
CH-31934	PLUG FOR NUT FOR FRONT FORK 30
CH-37729 (SET)	RELEASE PIN FOR FOLDING MECHANISM
CH-37737	SCREW FOR BRAKE
CH-32705	SCREW FOR CROSSBAR (INCLUDES WASHER, NUT, & SPRING)
CH-30713	SCREW FOR FOOT PLATE/HEEL STRAP
CH-37715 (PAIR)	SCREW FOR HANDLE & FOLDING MECHANISM
CH-37730 (PAIR)	SCREW FOR UPHOLSTERY
CT-SB100	SEAT BELT
CH-30705	SEAT GLIDE FOR FRONT & REAR
CH-31936	SLEEVE FOR FOLDING MECHANISM 3
CH-31935	TIP FOR BRAKE HANDLE 307/309/3
CH-37714	TOE KICK 307/309/319/327/329/3
CH-31902	UPHOLSTERY BACK FOR 309/319/32
* CHT-31953	UPHOLSTERY BACK FOR 319/329
CH-31903	UPHOLSTERY SEAT FOR 309/319/32
CH-37731 (PAIR)	WASHER FOR UPHOLSTERY
CH-30708	WHEEL FRONT FOR 307/309/319/32
CH-30710	WHEEL REAR FOR 307/309/319/327

* USE "CHT" PARTS FOR SERIAL NUMBERS STARTING WITH "CHT"	
OR	
ITEM #	USE "CHT" PARTS FOR SERIAL NUMBERS STARTING WITH:
319B	CH319BGJ0001
319BK	CH319BKGJ0001
319C	CH319CGM0001
319R	CH319RGH0065

CONTACT US!
We're here to help



info@novajoy.com



1 800 557 6682

CARE • QUALITY • INNOVATION



THE NOVA PROMISE