



nova

327

Parts Breakdown
*For models: 327B, 327R
Serial # Starting "CHT" & "CH"*



An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your transport chair corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (**see Figure 1**).

If you need further assistance, please call our customer service team at **(800) 557-6682**.

Thank you!



Figure 1

Handle and Folding Mechanism Components

CH-32703



CH-32704



CH-37713



CH-32701



CH-32702



CH-31936



CH-37724



CH-37729



CH-37710



CHT-31954



CH-37715



CHT-32706



CHT-32707




CHT-32708




CHT-32709




Main Body Components

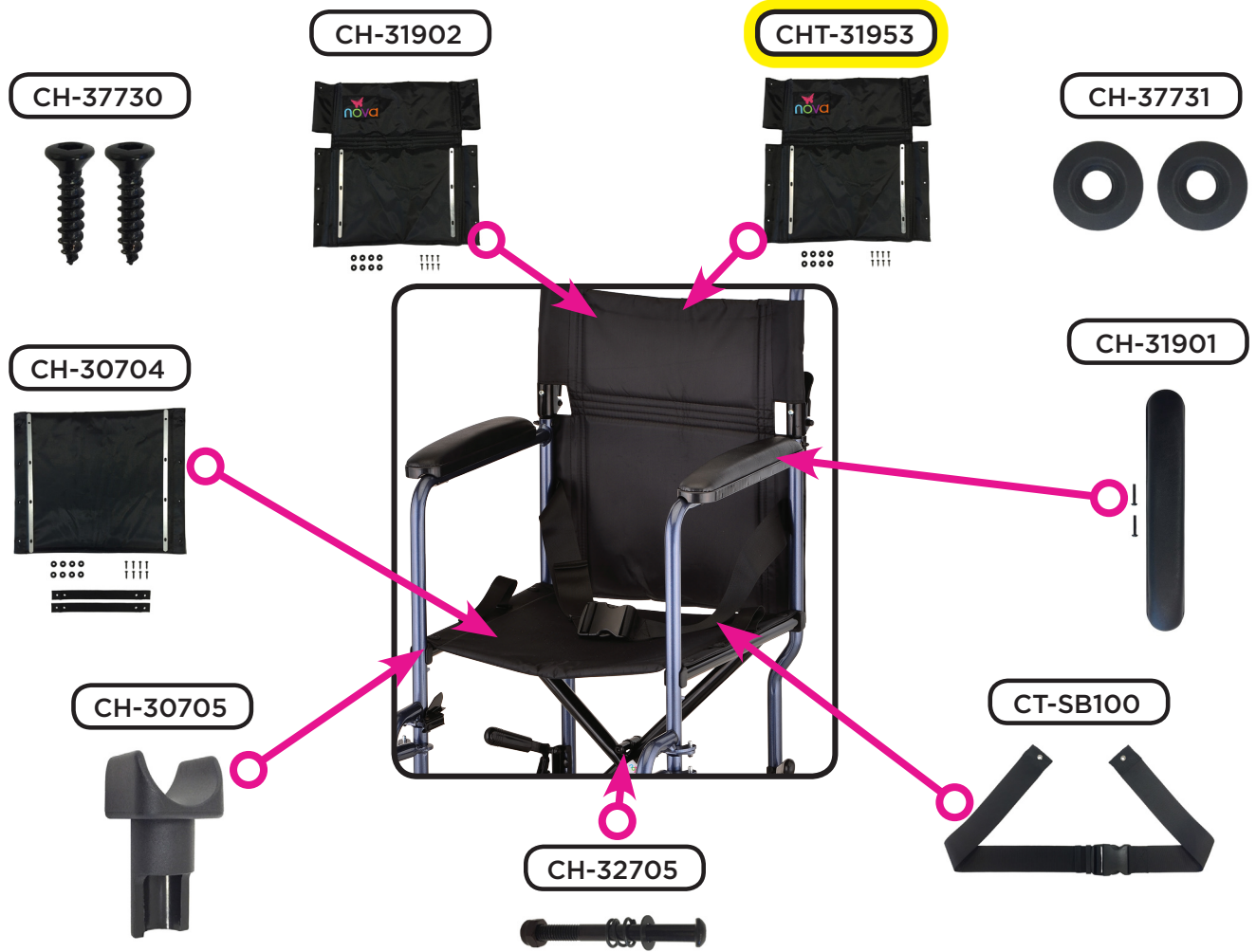


Footrest Assembly

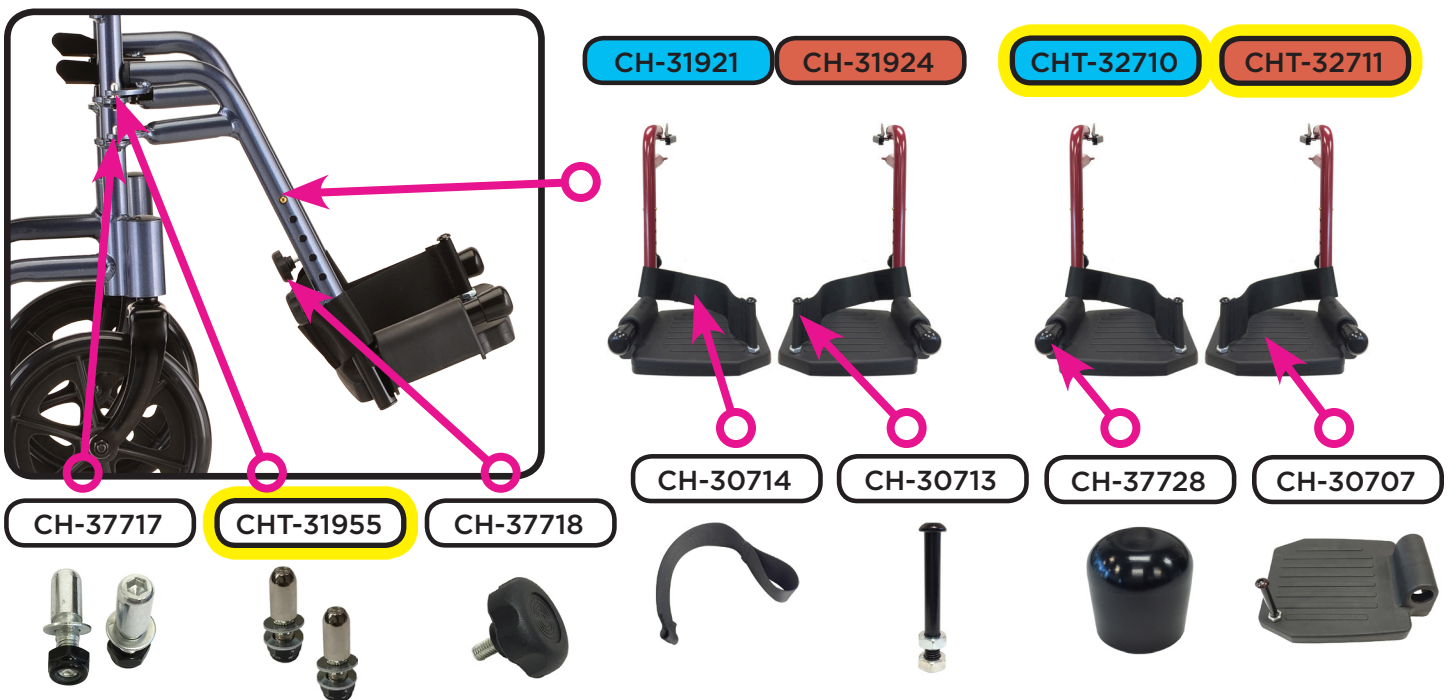

Front Wheel Assembly

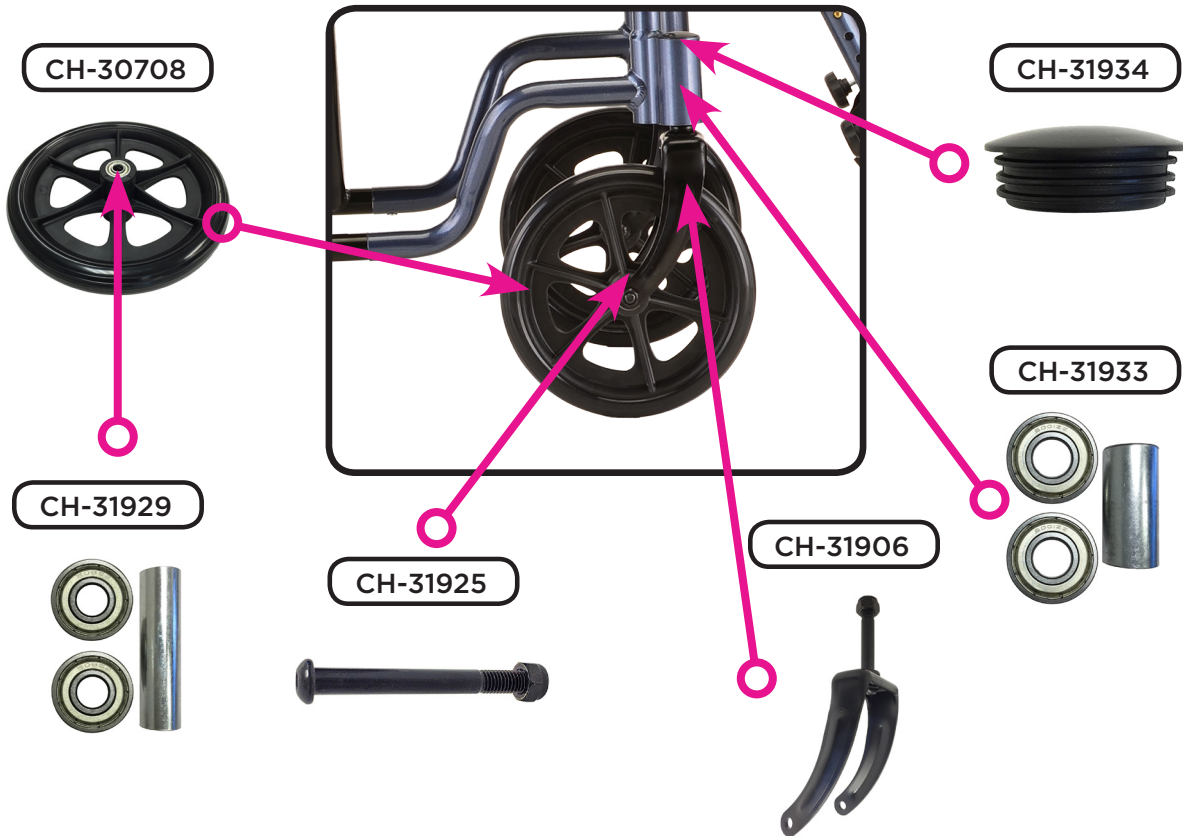

Rear Wheel Assembly

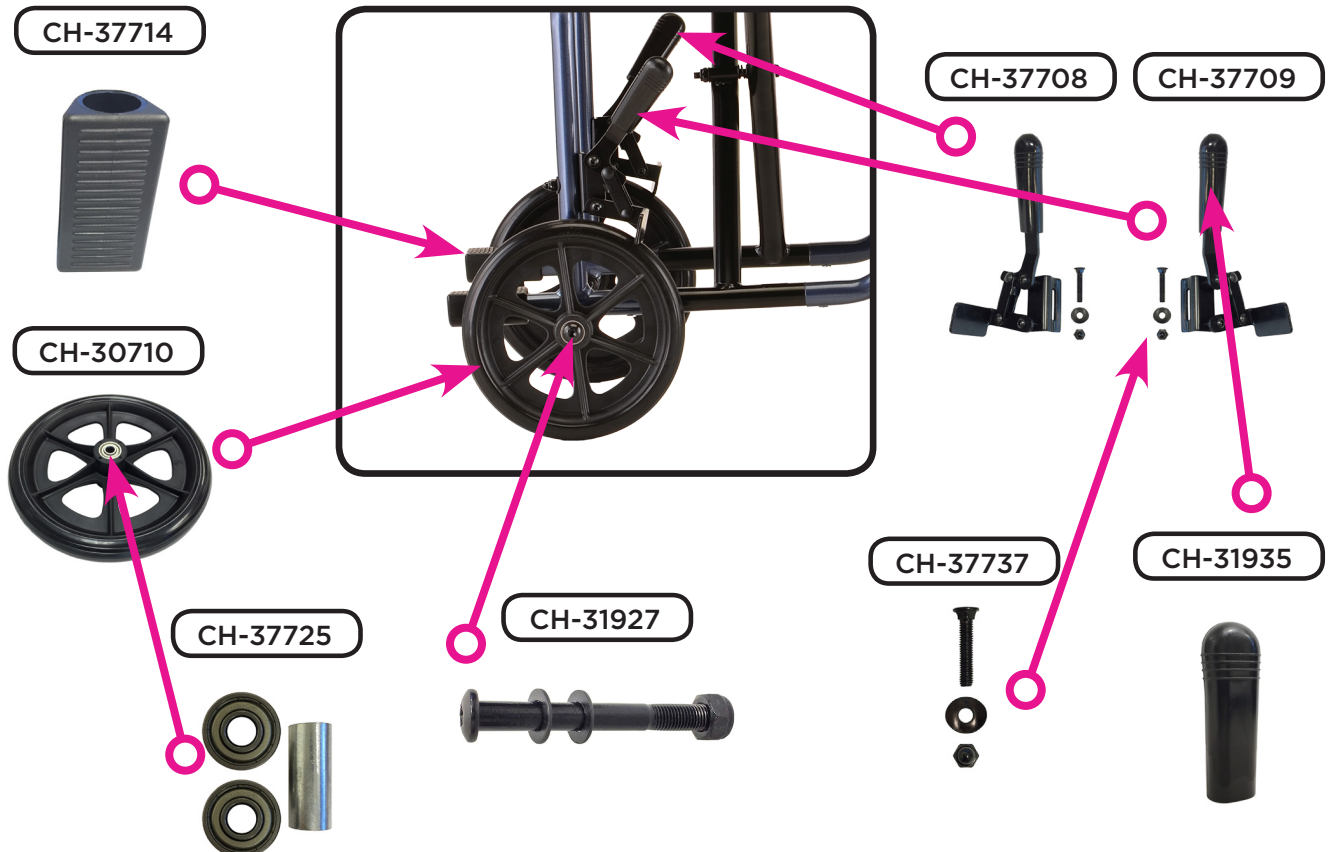




Item Number 327

327 Transport Chair

SERIAL # BEGINNING WITH "CHT" & "CH"

ITEM NUMBER	DESCRIPTION
CH-31901	ARM PAD (INCLUDES SCREWS)
CH-31925	AXLE FOR FRONT WHEEL (INCLUDES NUT)
CH-31927	AXLE FOR REAR WHEEL (INCLUDES WASHERS & NUT)
CH-30703	BACK UPHOLSTERY BLACK (INCLUDES SCREWS)
CH-31933	BEARING & SPACER FOR FRONT SWIVEL FORK
CH-31929	BEARING & SPACER FOR FRONT WHEEL
CH-37725	BEARING & SPACER FOR REAR WHEEL
CH-37708	BRAKE LEFT (INCLUDES SCREW, WASHER & NUT)
CH-37709	BRAKE RIGHT (INCLUDES SCREW, WASHER & NUT)
CH-37728	CAP FOR FOOT REST
CH-31934	COVER FOR FRONT SWIVEL FORK NUT/BEARINGS
CH-37710	FOLDING MECHANISM
* CHT-31954	FOLDING MECHANISM (UPPER)
CH-30707	FOOT PLATE (INCLUDES SCREW FOR HEEL STRAP)
CH-31921 (PAIR)	FOOT REST BLUE (PAIR)
CH-31924 (PAIR)	FOOT REST RED (PAIR)
* CHT-32710 (PAIR)	FOOTRESTS BLUE FOR 327/329/330 (PAIR)
* CHT-32711 (PAIR)	FOOTRESTS RED FOR 327/329/330 (PAIR)
CH-31906	FORK FOR FRONT WHEEL 319/327/3
CH-37713 (PAIR)	GRIP FOR HANDLE BAR (PAIR)
CH-32703	HANDLE LEFT BLUE
* CHT-32706	HANDLE LEFT FOR 327/329 BLUE
CH-32701	HANDLE LEFT FOR 327/329 RED
* CHT-32708	HANDLE LEFT FOR 327/329 RED
CH-32704	HANDLE RIGHT BLUE
* CHT-32707	HANDLE RIGHT FOR 327/329 BLUE
CH-32702	HANDLE RIGHT FOR 327/329 RED
* CHT-32709	HANDLE RIGHT FOR 327/329 RED
CH-30714	HEEL STRAP
CH-37718	KNOB FOR FOOT REST
CH-37724	LEVER FOR FOLDING MECHANISM
* CHT-31955 (PAIR)	PIN (PAIR) FOR FOOTREST
CH-37717 (PAIR)	PIN FOR FOOT REST (PAIR)
CH-37729 (SET)	RELEASE PIN FOR FOLDING MECH
CH-37730 (PAIR)	SCREW - UPHOLSTERY/HAND BRAKE (PAIR)

	Item Number 327
	327 Transport Chair
	SERIAL # BEGINNING WITH "CHT" & "CH"
ITEM NUMBER	DESCRIPTION
CH-37737	SCREW FOR BRAKE
CH-32705	SCREW FOR CROSSBAR
CH-37715 (PAIR)	SCREW FOR FOLDING MECHANISM & HANDLE (PAIR)
CH-30713	SCREW FOR FOOT PLATE/HEEL STRAP
CT-SB100	SEAT BELT
CH-30705	SEAT GLIDE FOR FRONT & REAR
CH-30704	SEAT UPHOLSTERY BLACK (INCLUDES SCREWS)
CH-31936	SLEEVE FOR FOLDING MECHANISM
CH-31935	TIP FOR BRAKE HANDLE
CH-37714	TOE KICK
* CHT-32712	UPHOLSTERY BACK FOR 327
CH-37731	WASHER FOR UPHOLSTERY
CH-30708	WHEEL FRONT
CH-30710	WHEEL REAR

*

USE "CHT" PARTS FOR SERIAL NUMBERS STARTING WITH "CHT"	
AND	
USE "CHT" PARTS FOR "CH" SERIAL NUMBERS STARTING WITH:	
ITEM NUMBER	SERIAL NUMBER
327B	CH327BGH0001
327R	CH327RGJ0001

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

novajoy.com

CARE • QUALITY • INNOVATION



THE NOVA PROMISE