



# 330

Parts Breakdown  
*For models: 330B, 330R*



## An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your transport chair corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (**see Figure 1**).

If you need further assistance, please call our customer service team at **(800) 557-6682**.

**Thank you!**



Figure 1

# Hand Brake Mechanism

CH-33015



P-CABLE



CH-33007



JN-423626



P40013



# Handle and Folding Mechanism Components

CH-33008



CH-33009



CH-33010



CH-33011



CH-33016



CH-31936



CH-37724



CH-37729



CH-37710



CHT-31954



CH-37715



CHT-33018



CHT-33019



CHT-33020



CHT-33021



# Main Body Components

CH-37730



CH-33001



CHT-33022



CH-37731



CH-33002



CH-33017



CH-30705



CH-31901




CH-32705



CT-SB100



## Footrest Assembly

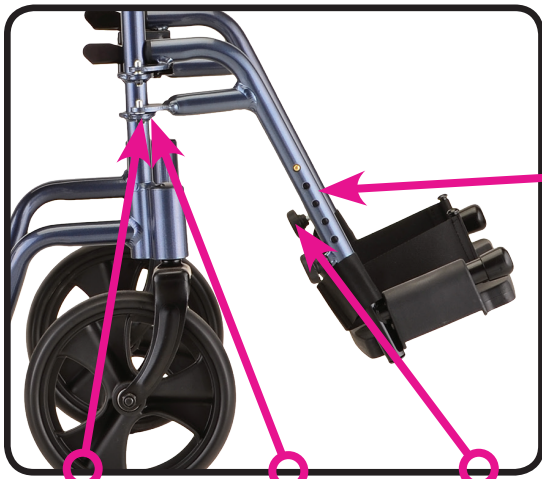

CH-31921

CH-31924

CHT-32710

CHT-32711



CH-30714

CH-30713

CH-37728

CH-30707

CH-37717


CHT-31955

CH-37718






## Front Wheel Assembly

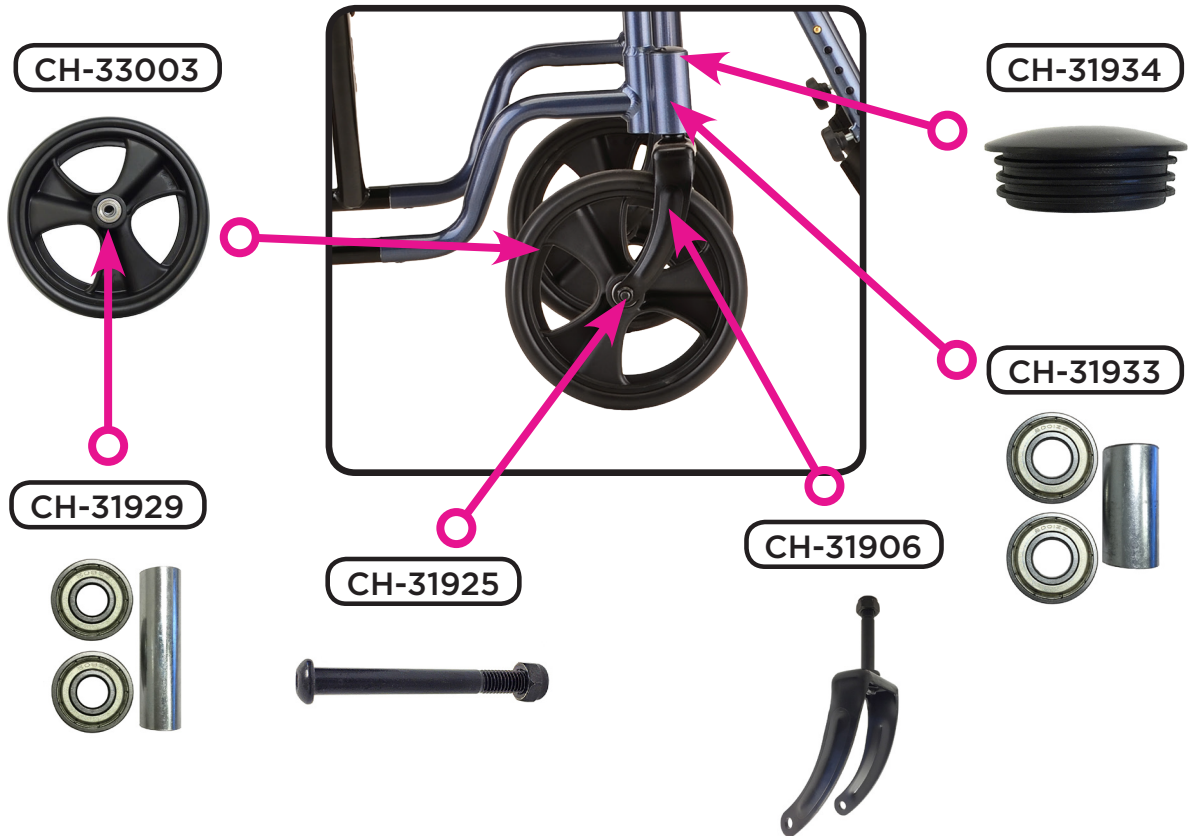

## Rear Wheel Assembly

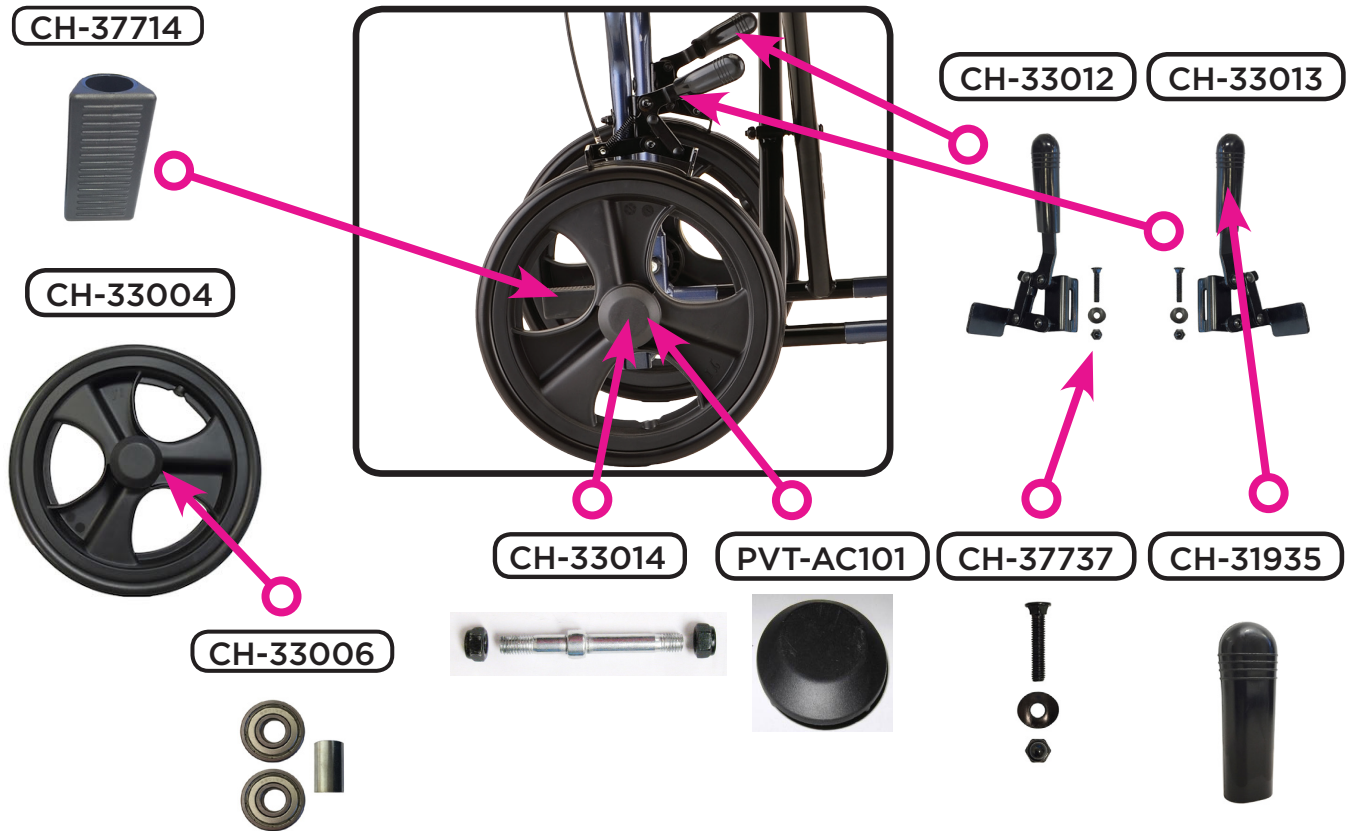



Item Number 330

### 330 Transport Chair

SERIAL # BEGINNING WITH "CHT" & "CH"

ITEM NUMBER	DESCRIPTION
CH-31901	ARM PAD FOR 309/319/327/329/330
CH-33014	AXLE & NUT FOR REAR WHEEL
CH-31925	AXLE SET FOR FRONT WHEEL 309
CH-33006 (SET)	BEARING & SPACER (SET) FOR REAR
CH-31929	BEARING & SPACER FOR FRONT WHEEL
CH-31933	BEARINGS FOR FRONT FORK 307
P-CABLE (PAIR)	BRAKE CABLE (PAIR)
CH-33012	BRAKE LEFT FOR 330 SN:CH
CH-33013	BRAKE RIGHT FOR 330 SN:CH
CH-33017	CABLE GUIDE FOR 330 SN: CH
CH-37728	CAP FOR FOOT REST SN: CH
CH-33007	COVER FOR BRAKE CABLE (27")
PVT-AC101	COVER FOR REAR WHEEL AXLE (330)
P40013 (PAIR)	CRIMP FOR STEEL CABLE (PAIR)
* CHT-31954	FOLDING MECHANISM (UPPER)
CH-37710	FOLDING MECHANISM FOR HANDLE
CH-30707 (PAIR)	FOOT PLATE (PAIR) FOR 307/309
CH-31921 (PAIR)	FOOT REST (PAIR) BLUE FOR 327B/329B
CH-31924 (PAIR)	FOOT REST RED (PAIR) FOR 327/329/330
* CHT-32710 (PAIR)	FOOTRESTS BLUE (PAIR) FOR 327/329/330
* CHT-32711 (PAIR)	FOOTRESTS RED (PAIR) FOR 327/329/330
CH-31906	FORK FOR FRONT WHEEL 319/327/330
CH-33016 (PAIR)	GRIP (PAIR) FOR 330 SN: CH
CH-33015	HAND BRAKE W/CABLE FOR 330
CH-33008	HANDLE BAR LEFT BLUE FOR 330B
CH-33010	HANDLE BAR LEFT RED FOR 330R
CH-33009	HANDLE BAR RIGHT BLUE FOR 330B
CH-33011	HANDLE BAR RIGHT RED FOR 330R
* CHT-33018	HANDLE LEFT FOR 330 BLUE
* CHT-33020	HANDLE LEFT FOR 330 RED
* CHT-33019	HANDLE RIGHT FOR 330 BLUE
* CHT-33021	HANDLE RIGHT FOR 330 RED
CH-30714	HEEL STRAP SN: CH
CH-37718	KNOB WITH SCREW FOR FOOTREST
CH-37724	LEVER FOR FOLDING MECHANISM
* CHT-31955 (PAIR)	PIN (PAIR) FOR FOOTREST

	Item Number 330
	<b>330 Transport Chair</b>
	SERIAL # BEGINNING WITH "CHT" & "CH"
ITEM NUMBER	DESCRIPTION
CH-37717 (PAIR)	PIN FOR FOOTREST
CH-31934	PLUG FOR NUT FOR FRONT FORK
CH-37729	RELEASE PIN FOR FOLDING MECH
CH-37737	SCREW FOR BRAKE SHOE
CH-32705	SCREW FOR CROSSBAR
CH-37715	SCREW FOR FOLDING MECHANISM
CH-30713	SCREW FOR HEEL STRAP SN: CH
CH-37730	SCREW FOR UPHOLSTERY/HAND BRAKE
CT-SB100	SEAT BELT FOR ALL TRANSPORT CHAIRS
CH-30705	SEAT GLIDE FOR FRONT & REAR
CH-31936	SLEEVE FOR FOLDING MECHANISM 3
JN-423626	TENSION SCREW FOR 4204/4205
CH-31935	TIP FOR BRAKE HANDLE 307/309/3
CH-37714	TOE KICK 307/309/319/327/329/3
* CHT-33022	UPHOLSTERY BACK FOR 330
CH-33001	UPHOLSTERY BACK FOR 330 SN:CH
CH-33002	UPHOLSTERY SEAT FOR 330 SN:CH
CH-37731	WASHER FOR UPHOLSTERY SCREW
CH-33003	WHEEL FRONT FOR 330 SN:CH
CH-33004	WHEEL REAR FOR 330 SN:CH

\*

USE "CHT" PARTS FOR SERIAL NUMBERS STARTING WITH "CHT"	
AND	
USE "CHT" PARTS FOR "CH" SERIAL NUMBERS STARTING WITH:	
ITEM NUMBER	SERIAL NUMBER
330B	CH330BGJ0001
330R	CH330RGH0001

**CONTACT US!**  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)  
 1 800 557 6682

[novajoy.com](http://novajoy.com)

CARE • QUALITY • INNOVATION



THE NOVA PROMISE