



377

Parts Breakdown
For models: 377B-R



An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your transport chair corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (**see Figure 1**).


If you need further assistance, please call our customer service team at **(800) 557-6682**.

Thank you!



Figure 1


Handle and Folding Mechanism Components

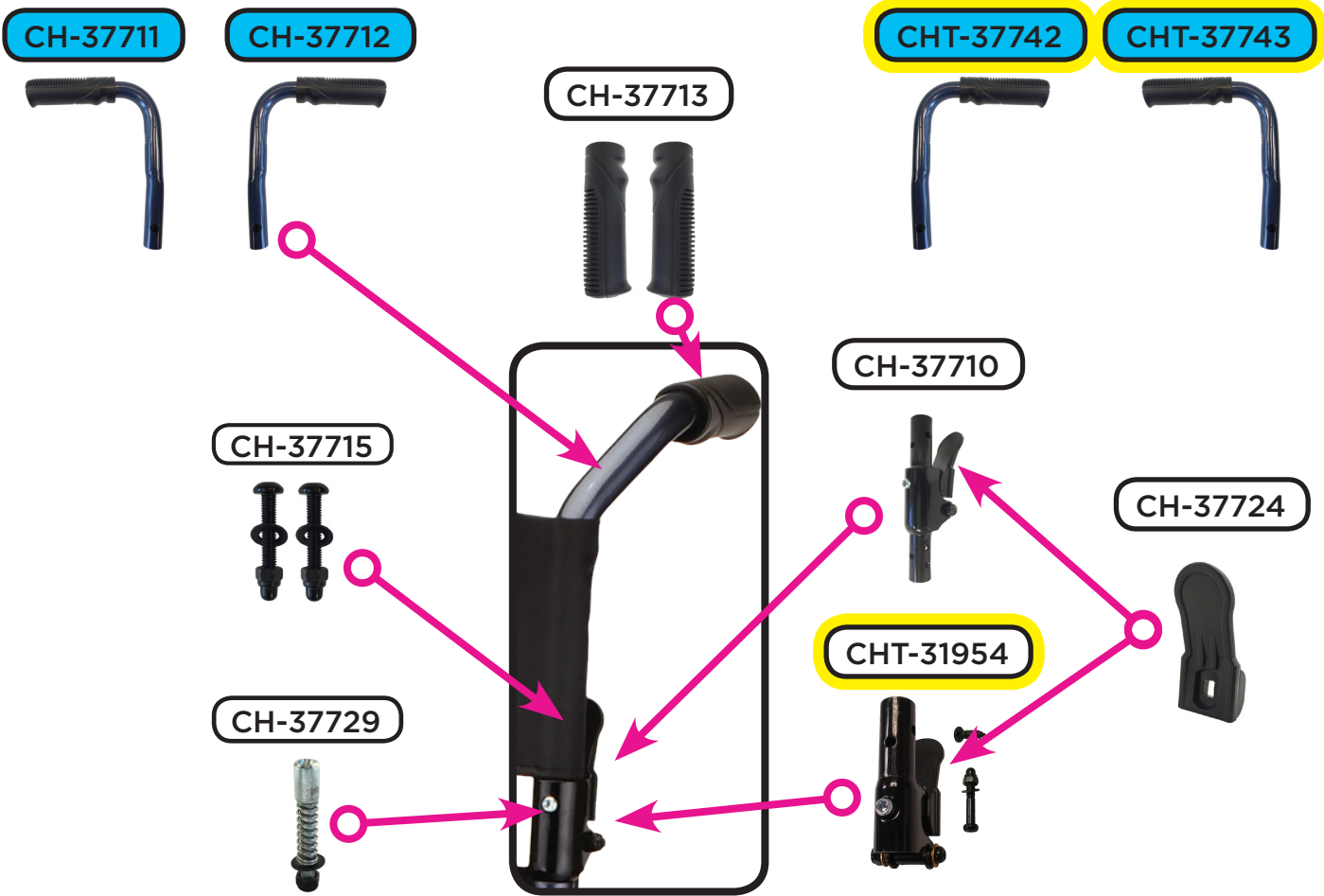

Lower Body Components




Main Body Components

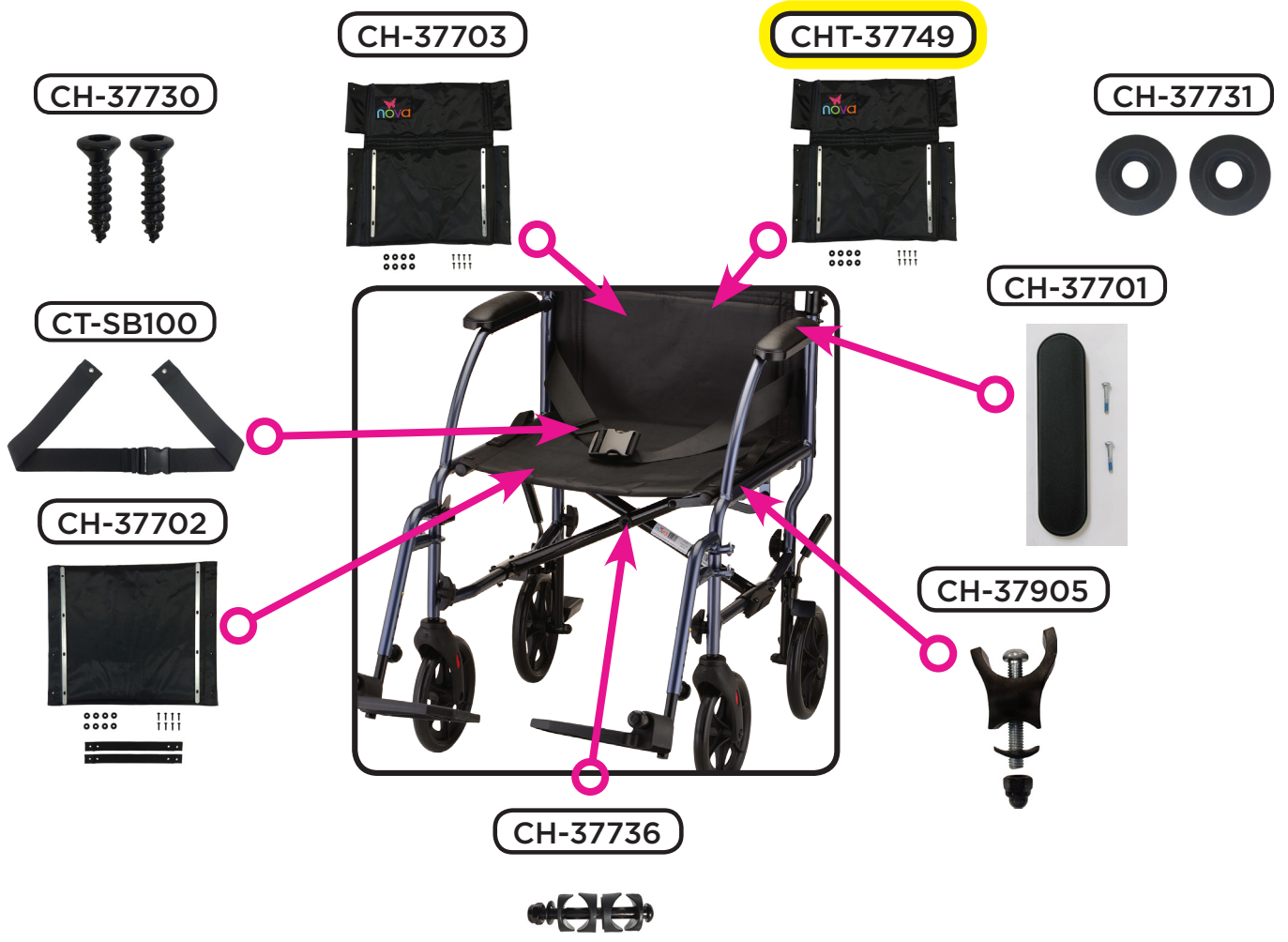


Footrest Assembly



Front Wheel Assembly

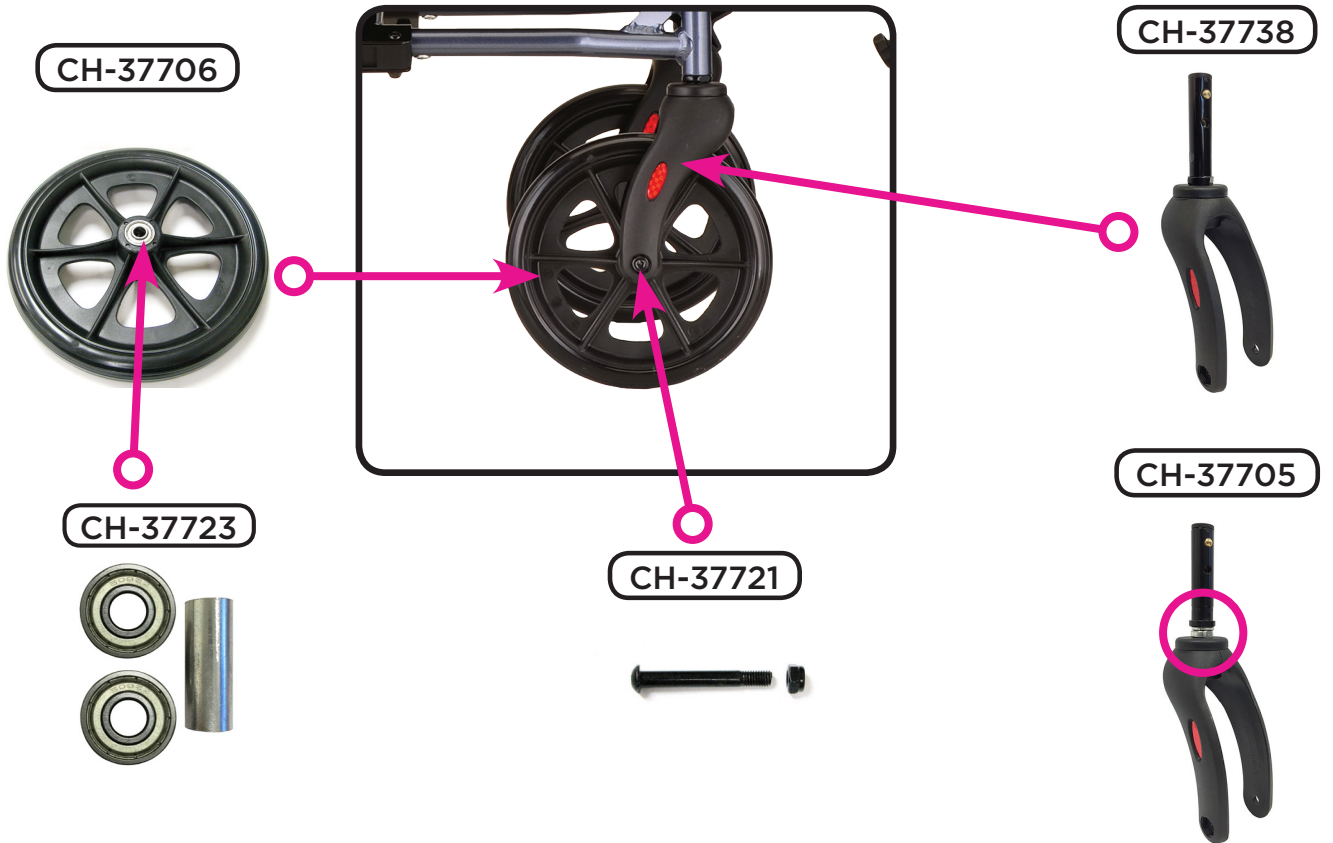

Rear Wheel Assembly

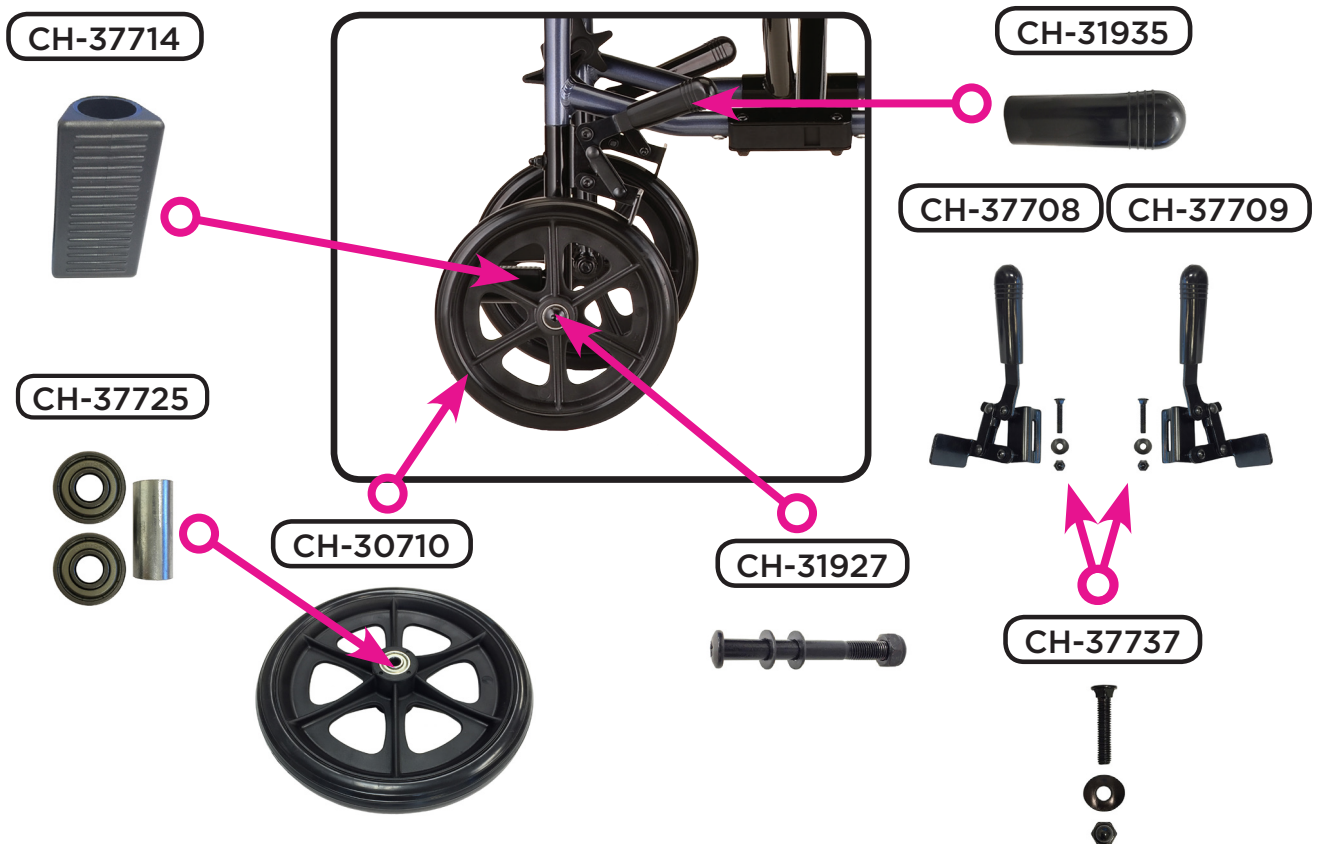




Item Number 377

377 Transport Chair

SERIAL # BEGINNING WITH "CHT" & "CH"

ITEM NUMBER	DESCRIPTION
CH-37701	ARM PAD FOR 377/379 SN:CH
CH-37721 (PAIR)	AXLE FOR FRONT WHEEL 377/379
CH-31927	AXLE SET FOR REAR WHEEL 309/31
CH-37723	BEARING & SPACER FOR FRONT WHEEL
CH-37708	BRAKE LEFT 307/309/319/327/329
CH-37709	BRAKE RIGHT 307/309/319/327/329
CH-37728	CAP FOR FOOT REST SN: CH
CH-37905	CRADLE FOR SEAT FOR 377/379
CH-37734	CROSSBAR SUPPORT FOR 377/379
* CHT-31954	FOLDING MECHANISM (UPPER)
CH-37710	FOLDING MECHANISM FOR HANDLE
CH-37716	FOOT PLATE FOR 377/379 SN:CH/
CH-37719	FOOT REST FOR 377B/379B
* CHT-37748	FOOTRESTS (PAIR) FOR 377/379
CH-37738	FORK FRONT FOR 377 NEW STEM WITHOUT SILVER NUT
CH-37705	FORK FRONT FOR 377 OLD STEM WITH SILVER NUT
CH-37713 (PAIR)	GRIP (PAIR) FOR HANDLE BAR
CH-37711	HANDLE BAR LEFT FOR 377B/379B
CH-37712	HANDLE BAR RIGHT FOR 377B/379B
* CHT-37742	HANDLE LEFT FOR 377/379 BLUE
* CHT-37743	HANDLE RIGHT FOR 377/379 BLUE
CH-37704	KNOB WITH SCREW FOR 377 LEG
CH-37718	KNOB WITH SCREW FOR FOOTREST
CH-37740	LEG LEFT REAR FOR 377
CH-37741	LEG RIGHT REAR FOR 377
CH-37724	LEVER FOR FOLDING MECHANISM
* CHT-31955	PIN (PAIR) FOR FOOTREST FOR 31
CH-37717 (PAIR)	PIN (PAIR) FOR FOOTREST
CH-37739	PLUG, SPRING & PIN FOR 377 FRONT LEG
CH-37729	RELEASE PIN FOR FOLDING MECH
CH-37730	SCREW - UPHOLSTERY/HAND BRAKE
CH-37737	SCREW FOR BRAKE
CH-37736	SCREW FOR CROSSBAR 377/379
CH-37715 (PAIR)	SCREW FOR FOLDING MECHANISM & ARM PAD (PAIR)
CH-37735	SCREWS FOR CROSSBAR SUPPORT

	Item Number 377
	377 Transport Chair
	SERIAL # BEGINNING WITH "CHT" & "CH"
ITEM NUMBER	DESCRIPTION
CT-SB100	SEAT BELT
CH-31935	TIP FOR BRAKE HANDLE 307/309
CH-37714	TOE KICK 307/309/319/327/329
* CHT-37749	UPHOLSTERY BACK FOR 377/379
CH-37703	UPHOLSTERY BACK FOR 377/379
CH-37702	UPHOLSTERY SEAT FOR 377/379
CH-37731	WASHER FOR UPHOLSTERY SCREW
CH-37706	WHEEL FRONT FOR 377/379 SN:CH
CH-30710	WHEEL REAR FOR 307/309/319/327

*	USE "CHT" PARTS FOR SERIAL NUMBERS STARTING WITH "CHT"	
	AND	
	USE "CHT" PARTS FOR "CH" SERIAL NUMBERS STARTING WITH:	
	ITEM NUMBER	SERIAL NUMBER
	377B-R	CH377BGJ0001

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

novajoy.com

CARE • QUALITY • INNOVATION



THE NOVA PROMISE