



nova

352

Parts Breakdown
For models: 352B, 352R
Serial # Starting "NV"



CARE • QUALITY • INNOVATION




THE NOVA PROMISE

Revised on November 21, 2023

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

novajoy.com

An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your transport chair corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (see Figure 1).

If you need further assistance, please call our customer service team at (800) 557-6682.

Thank you!

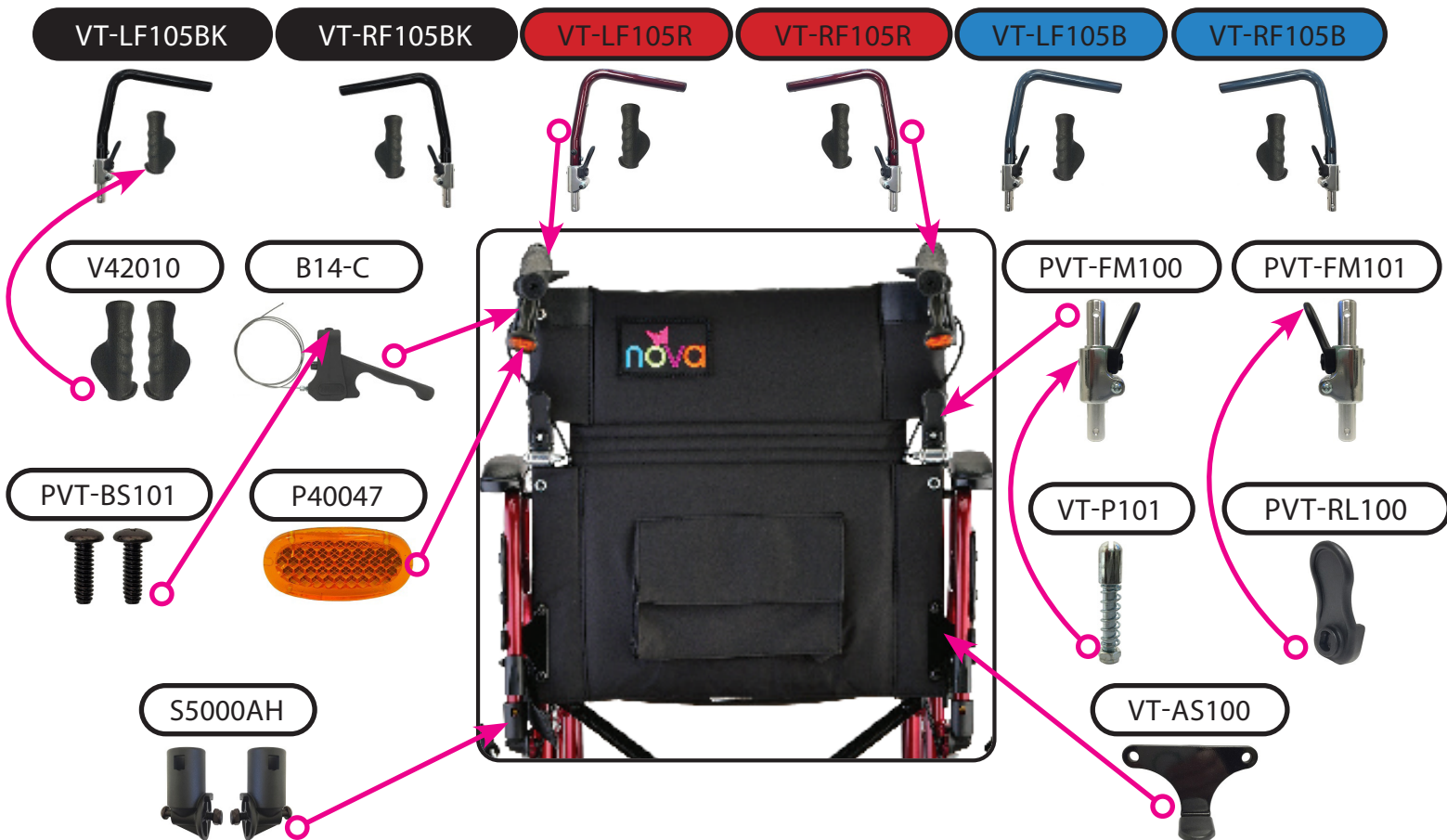


Figure 1

Main Body Components




Handle and Folding Mechanism Components




Detachable Arm Components



Footrest Assembly

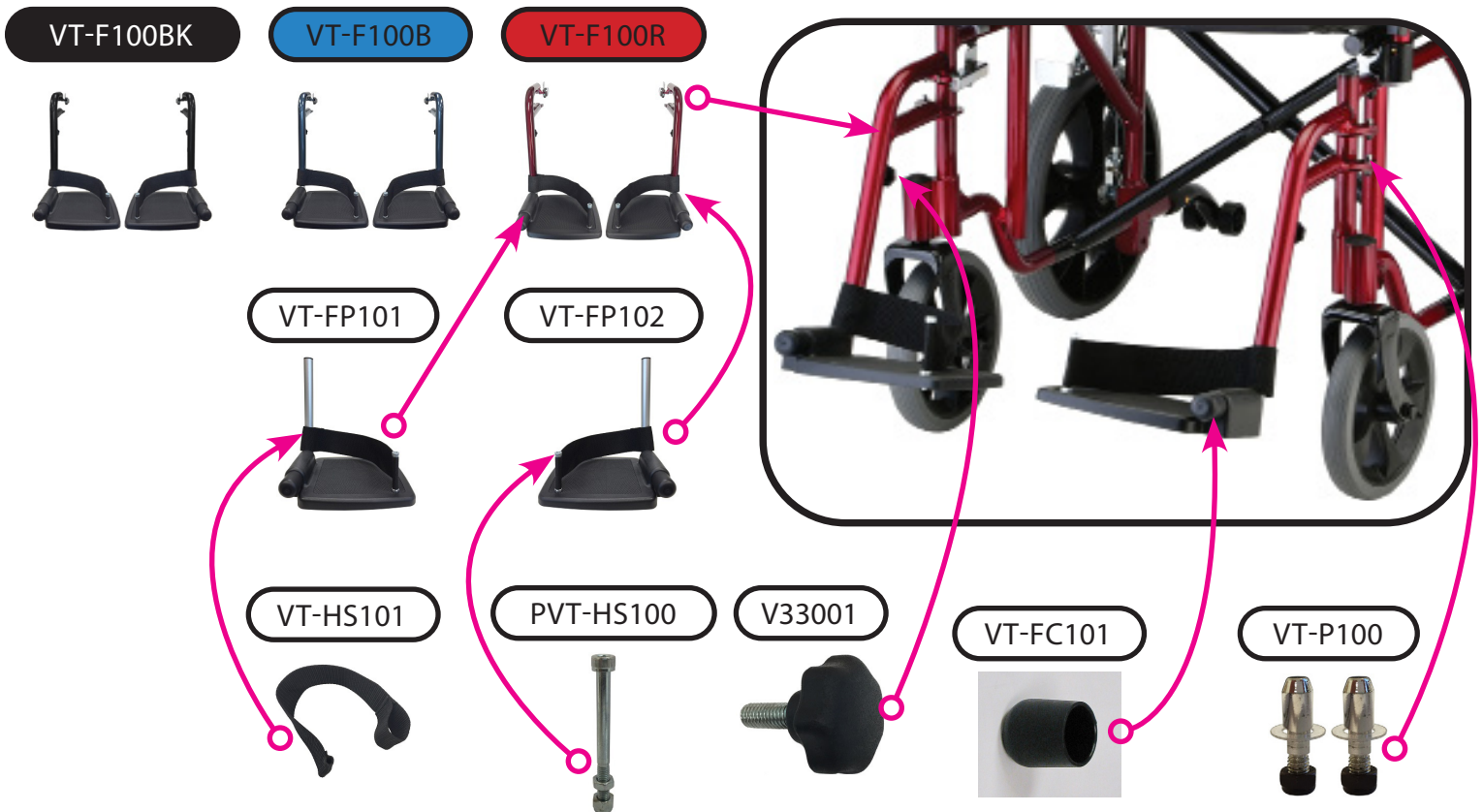

Front And Rear Wheel Assembly Components

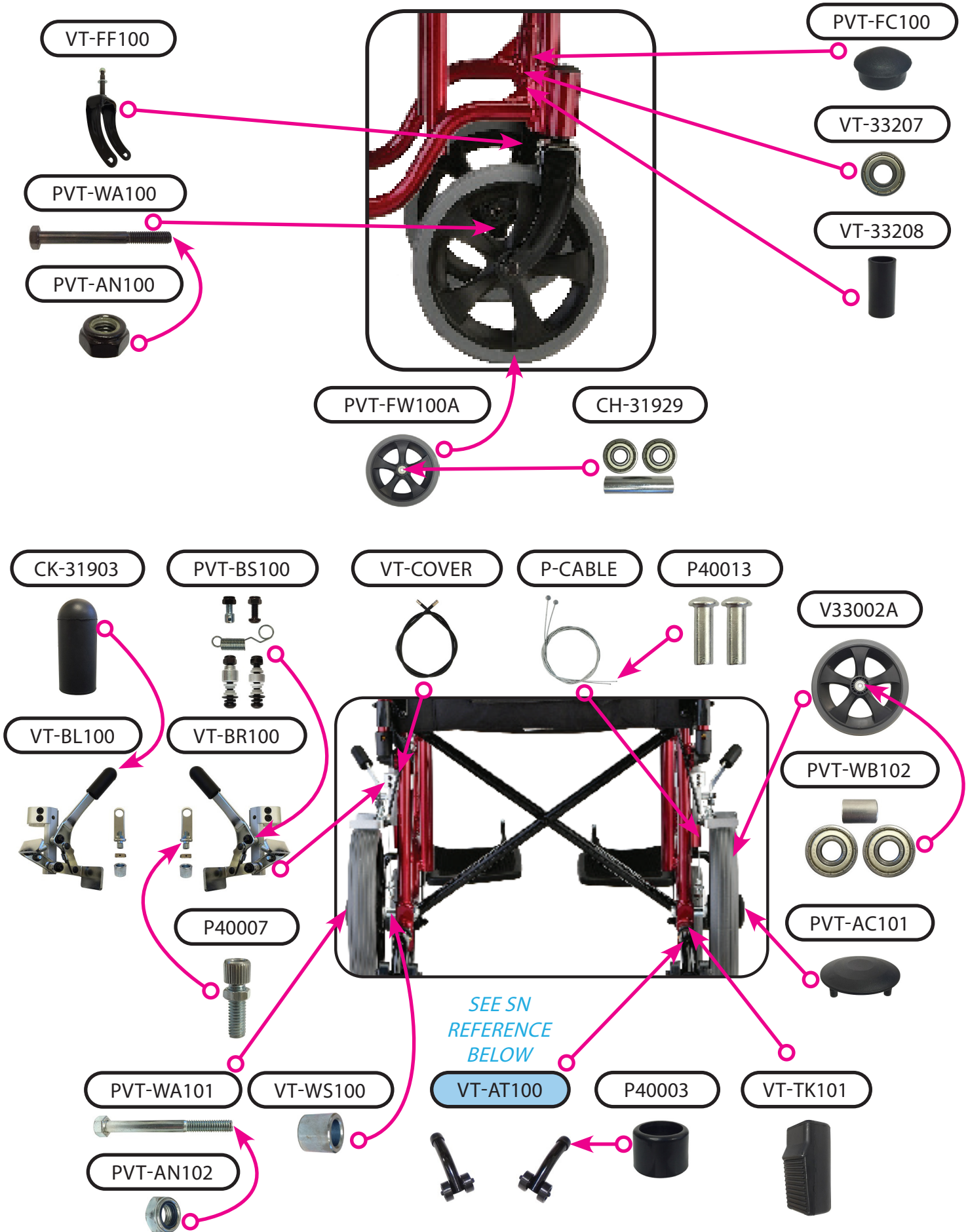



Item Number 352B, 352R

352 TRANSPORT CHAIR

SERIAL # BEGINNING WITH "NV"

ITEM NUMBER	DESCRIPTION	UOM
VT-AT100	ANTI-TIPPER (PAIR) BLACK <i>SEE SN REFERENCE BELOW</i>	PAIR
PVT-33215	ARM PAD DESK LENGTH	EACH
VT-AS100	ARM SUPPORT BRACKET	EACH
PVT-WA100	AXLE FOR FRONT WHEEL	EACH
PVT-WA101	AXLE FOR REAR WHEEL	EACH
CH-31929	BEARING & SPACER FOR FRONT WHEEL	SET
PVT-WB102	BEARING FOR REAR WHEEL	EACH
VT-33207	BEARING FOR SWIVEL FRONT FORK	EACH
VT-COVER	BLACK CABLE COVER	EACH
VT-33201	BLUE LEFT DETACHABLE ARM	EACH
VT-33202	BLUE RIGHT DETACHABLE ARM	EACH
P-CABLE	BRAKE CABLE (PAIR)	PAIR
VT-BL100	BRAKE FOR 12" LEFT WHEEL	EACH
VT-BR100	BRAKE FOR 12" RIGHT WHEEL	EACH
VT-FC101	COVER FOR FOOTREST	EACH
PVT-FC100	COVER FOR FRONT FORK SWIVEL SCREW	EACH
PVT-AC101	COVER FOR REAR WHEEL AXLE	EACH
V-33004	CRADLE FOR FRONT SEAT	EACH
P40013	CRIMP FOR STEEL CABLE (PAIR)	PAIR
VT-AP100	DESK ARM PANEL	EACH
VT-FP102	FOOT PLATE FOR LEFT FOOTREST	EACH
VT-FP101	FOOT PLATE FOR RIGHT FOOTREST	EACH
VT-F100B	FOOT REST BLUE	PAIR
VT-F100R	FOOT REST RED	PAIR
VT-F100BK	FOOT REST FOR BLACK	PAIR
VT-FF100	FORK FOR FRONT WHEEL	EACH
VT-SG100	GLIDE FOR SEAT	PAIR
B14-C	HAND BRAKE WITH STEEL CABLE	EACH
V42010	HAND GRIP ANATOMICAL	PAIR
VT-HS101	HEEL LOOP FOR FOOT REST	EACH
V33001	KNOB FOR ADJUSTABLE FOOTREST	EACH
PVT-FM100	LEFT FOLDING MECHANISM	EACH
PVT-RL100	LEVER TO RELEASE FOLD DOWN ARM	EACH
PVT-AN102	NUT FOR REAR AXLE	EACH
PVT-AN100	NUT FOR AXLE - FRONT/REAR	EACH




Item Number 352B, 352R

352 TRANSPORT CHAIR

SERIAL # BEGINNING WITH "NV"


ITEM NUMBER	DESCRIPTION	UOM
VT-P101	PIN FOR FOLDING MECHANISM	EACH
VT-P100	PIN FOR FOOTREST	PAIR
PVT-PL101	PLUG FOR FRAME	PAIR
PVT-PL100	PLUG FOR SEAT BAR	PAIR
VT-LF105B	PUSH HANDLE LEFT - BLUE	EACH
VT-RF105B	PUSH HANDLE RIGHT - BLUE	EACH
VT-RF105R	PUSH HANDLE- RIGHT - RED	EACH
VT-LF105R	PUSH HANDLE LEFT - RED	EACH
S5000AH	RECEIVER FOR FLIP BACK ARM-REAR	PAIR
VT-33203	RED LEFT DETACHABLE ARM	EACH
VT-33204	RED RIGHT DETACHABLE ARM	EACH
P40047	REFLECTOR FOR BRAKE HANDLE	EACH
PVT-RL101	RELEASE LEVER ON FLIP BACK ARM	EACH
PVT-FM101	RIGHT FOLDING MECHANISM	EACH
VT-TK101	RUBBER TOE KICK	EACH
PVT-BS101	SCREW FOR HAND BRAKE	EACH
VT-SU100	SCREW FOR SEAT UPHOLSTERY	EACH
PVT-CS102	SCREW FOR CROSSBAR (INCLUDES NUT, SPACER & WASHERS)	EACH
VT-33206	SCREW FOR FRONT SEAT CRADLE	EACH
PVT-BS100	SCREW SET FOR WHEEL BRAKE	SET
P40007	SCREW TO ADJUST CABLE TENSION	EACH
V33003	SCREW TO ATTACH DESK ARM PAD	EACH
PVT-HS100	SCREW TO ATTACH HEEL LOOP TO FOOT REST	EACH
VT-33205	SCREW TO ATTACH SIDE PANEL	EACH
VT-SU101	SCREWS FOR BACK UPHOLSTERY	SET
CT-SB100	SEAT BELT	EACH
P40003	SILENCER FOR ADJ. HANDLE	EACH
VT-WS100	SPACER FOR REAR WHEEL AXLE	EACH
VT-33208	SPACER FOR SWIVEL FRONT FORK	EACH
CK-31903	TIP FOR BRAKE HANDLE	EACH
VT-UB104	UPHOLSTERY 19" BACK	EACH
VT-US100	UPHOLSTERY 19" SEAT	EACH
V33002A	WHEEL 12" GREY REAR	EACH
PVT-FW100A	WHEEL 8" GREY FRONT	EACH
VT-33216	BLACK LEFT D/A	EACH

	Item Number 352B, 352R	
	352 TRANSPORT CHAIR	
	SERIAL # BEGINNING WITH "NV"	
ITEM NUMBER	DESCRIPTION	UOM
VT-33217	BLACK RIGHT D/A	EACH
VT-LF105BK	BLACK LEFT STRAIGHT PUSH HANDLE	EACH
VT-RF105BK	BLACK RIGHT STRAIGHT PUSH HANDLE	EACH

SERIAL NUMBER (SN) REFERENCE	
ITEM NUMBER	<i>POST JULY 2018 MODELS</i>
352B	SN AFTER: NV352BHE0001
352R	SN AFTER: NV352RHF0001

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

novajoy.com

CARE • QUALITY • INNOVATION



THE NOVA PROMISE