



# nova

# EXPRESS

Parts Breakdown  
For models: 4328BK, 4328BL, 4328PL, 4328RD  
Serial # Starting "HT"



CARE • QUALITY • INNOVATION




THE NOVA PROMISE

Revised on June 19, 2023

CONTACT US!  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)

## An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your rolling walker corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (see Figure 1).

If you need further assistance, please call our customer service team at (800) 557-6682.

Thank you!



Figure 1

# Upper Assembly Components

SEE BOTTOM FOR SN REFERENCE



HCT-432845


HCT-432846



HCT-480113

HCT-432815

HCT-432814

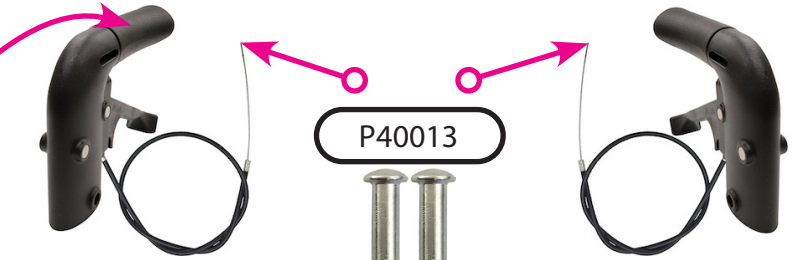


P40013

HCT-480107

HCT-480219

HCT-432834

HCT-432835

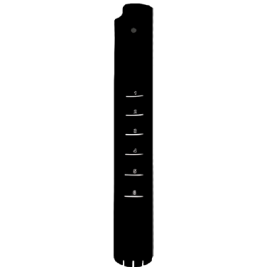

## Seat & Backrest

HCT-432836

HCT-432823



HCT-432812




HCT-432830


HCT-432804




# Folding Mechanism Components

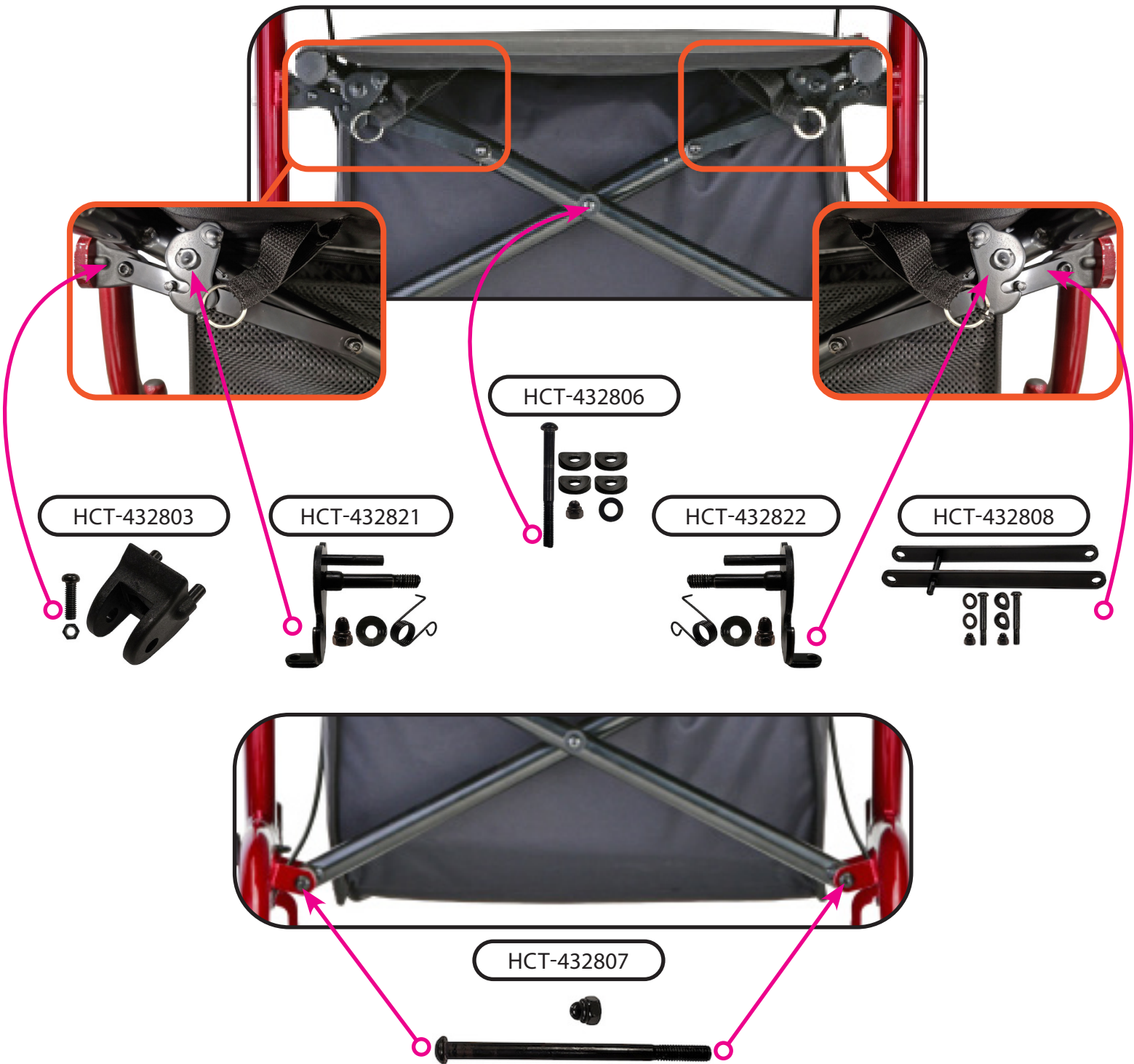

## Fork Assembly Components

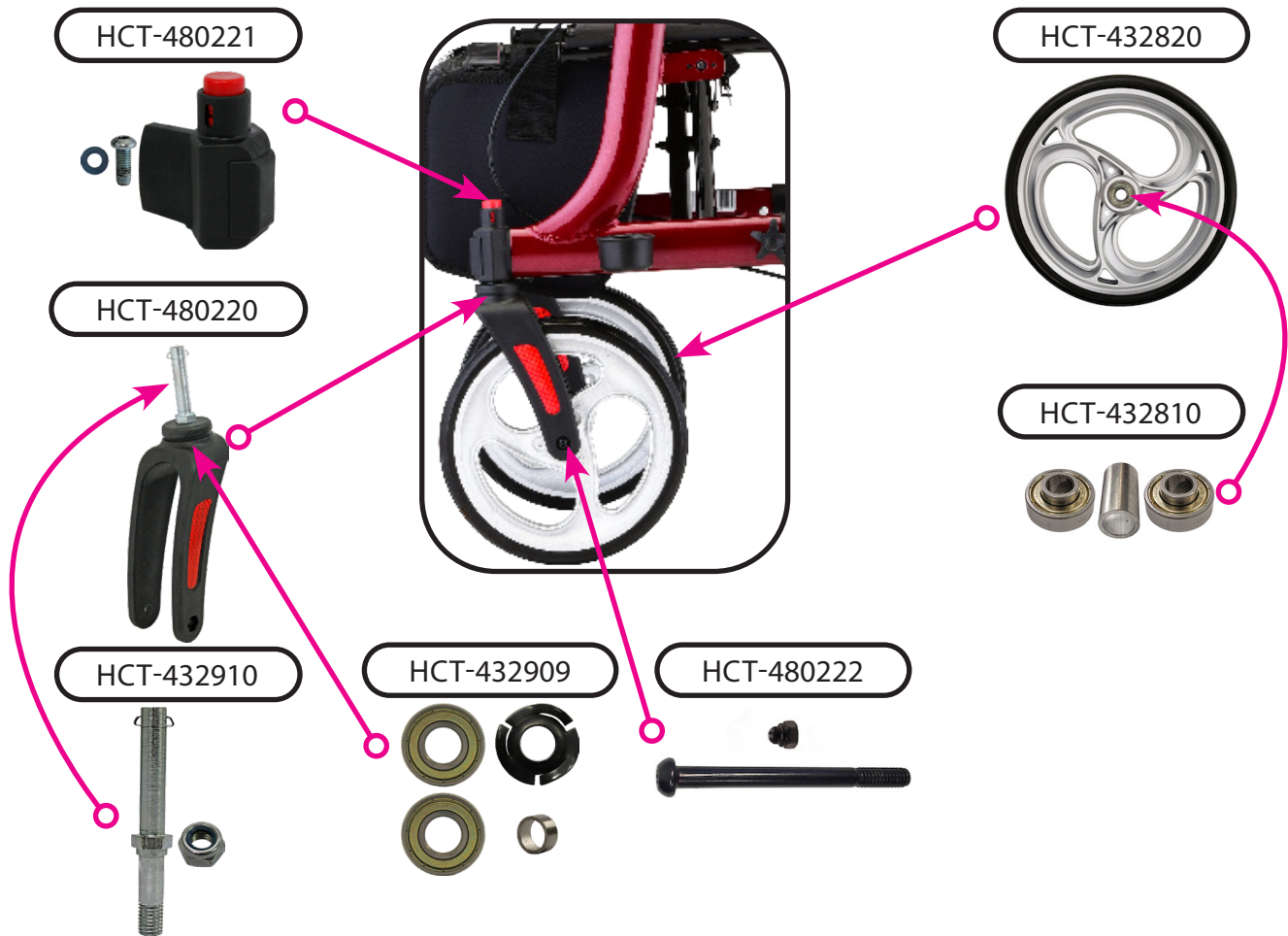

## Rear Wheel Assembly Components

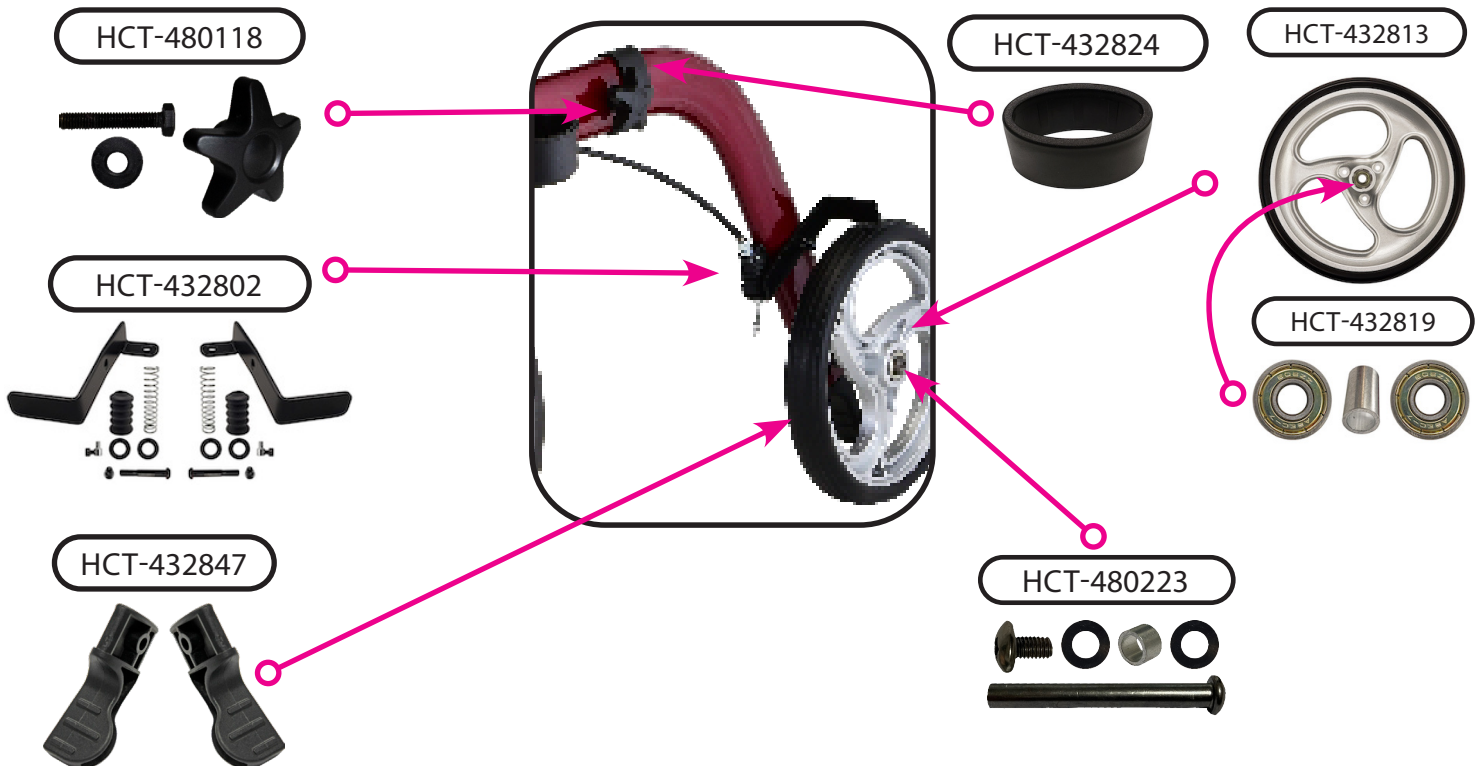





Item Number 4328

**EXPRESS**

SERIAL # BEGINNING WITH "HT "

ITEM NUMBER	DESCRIPTION	UOM
HCT-432836	BACKREST	EACH
HCT-432830	BAG	EACH
HCT-432802	BRAKE SHOE SET	SET
HCT-432804	CANE HOLDER TRAY	EACH
HCT-432805	COVER FOR FRONT FORK	EACH
P40013	CRIMP	PAIR
HCT-432806	CROSSBAR CENTER BOLT	EACH
HCT-432807	CROSSBAR SIDE BOLT	EACH
HCT-432803	CROSSBAR SIDE BRACKET	EACH
HCT-432808	CROSSBAR SUPPORT	EACH
HCT-480220	FORK	EACH
HCT-480221	FORK ATTACHMENT	EACH
HCT-480222	FRONT AXLE	EACH
HCT-432810	FRONT BEARING SET	SET
HCT-480219	HAND GRIPS	PAIR
HCT-432845	HANDLE ASSEMBLY RIGHT FOR 4328	EACH
HCT-432846	HANDLE ASSEMBLY LEFT FOR 4328	EACH
HCT-432814	HANDLE LEFT	EACH
HCT-432834	HANDLE LOWER BAR LEFT	EACH
HCT-432835	HANDLE LOWER BAR RIGHT	EACH
HCT-432815	HANDLE RIGHT	EACH
HCT-432829	HOLDER FOR BRAKE COVER	EACH
HCT-480118	KNOB W/SCREW & WASHER FOR REAR LEG	SET
HCT-480223	REAR AXLE	EACH
HCT-432819	REAR BEARING SET	SET
HCT-432821	RELEASE LEVER LEFT	EACH
HCT-432822	RELEASE LEVER RIGHT	EACH
HCT-432812	SEAT PLUG FRONT	EACH
HCT-480113	SEAT PLUG REAR	PAIR
HCT-432823	SEAT WITH HANDLE	EACH
HCT-432824	SILENCER FOR LEG	EACH
HCT-432910	STEM AND NUT FOR FRONT FORK	EACH
HCT-432909	SWIVEL BEARING SET FRONT FORK	SET
HCT-432847	TOE KICK	PAIR
HCT-432820	WHEEL FRONT 10"	EACH
HCT-432813	WHEEL REAR 8"	EACH

SERIAL NUMBER (SN) REFERENCE	
ITEM NUMBER	<i>POST SEPT. 2021 MODELS</i>
4328BK	SN AFTER: HT4328BKMG0001
4328BL	SN AFTER: HT4328BLMG0001
4328PL	SN AFTER: HT4328PLMG0002
4328RD	SN AFTER: HT4328RDMG0001

SERIAL NUMBER (SN) REFERENCE	
ITEM NUMBER	<i>POST JAN. 2023 MODELS</i>
4328BL	SN AFTER: HT4328BLNM0001
4328PL	SN AFTER: HT4328PLNM0001
4328RD	SN AFTER: HT4328RDNM0001

**CONTACT US!**  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)  
 1 800 557 6682

[novajoy.com](http://novajoy.com)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE