



nova
Monarch
Open & Fly Rollator

Parts Breakdown
For models: 4329BL, 4329PL, 4329RD
Serial # Starting "HT"



CARE • QUALITY • INNOVATION



THE NOVA PROMISE

Revised on June 15, 2023

CONTACT US!
We're here to help

✉ info@novajoy.com

☎ 1 800 557 6682

novajoy.com

An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your rolling walker corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (see Figure 1).

If you need further assistance, please call our customer service team at (800) 557-6682.

Thank you!



Figure 1



Upper Assembly Components



Seat & Backrest




Folding Mechanism Components

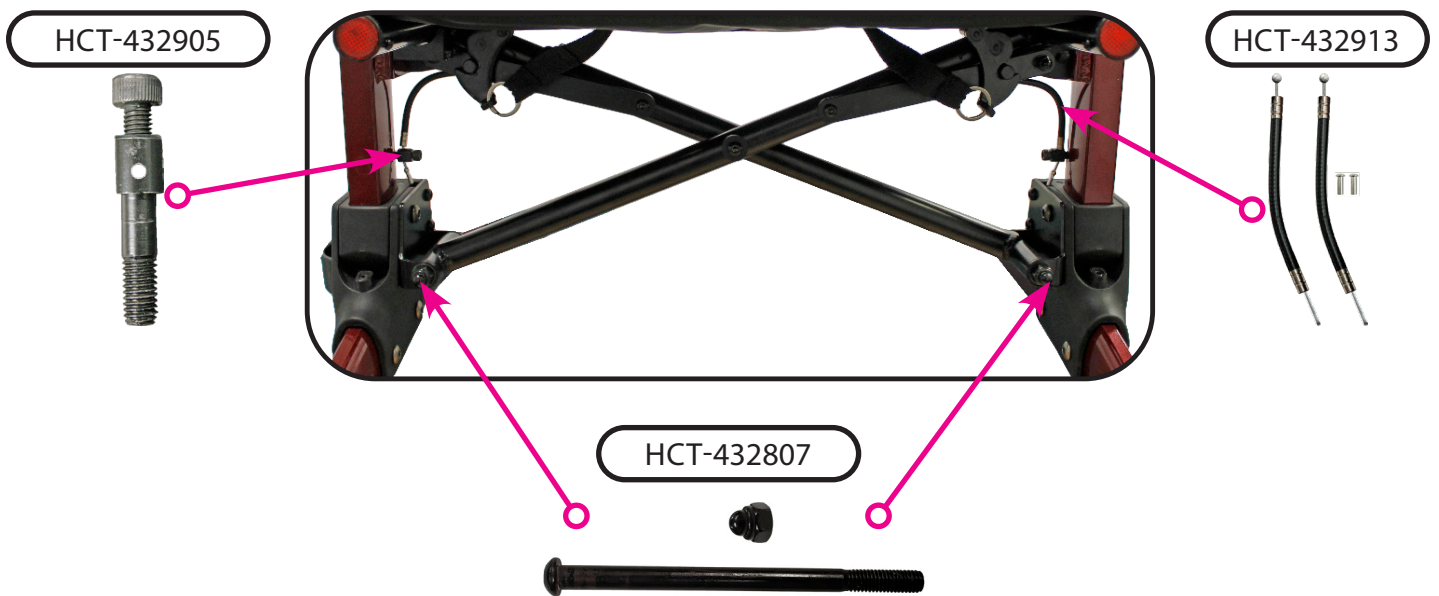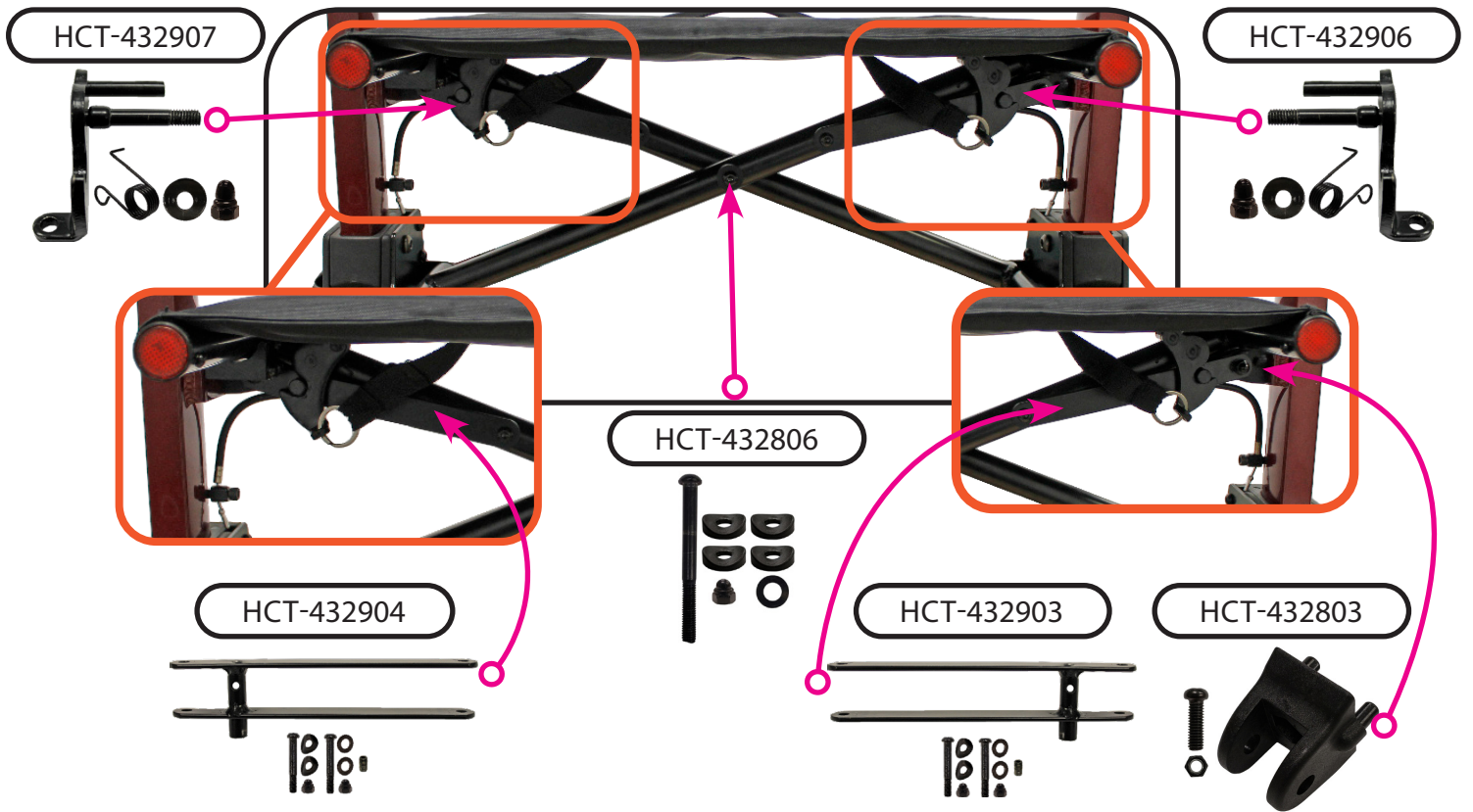

Fork Assembly Components



Rear Wheel Assembly Components

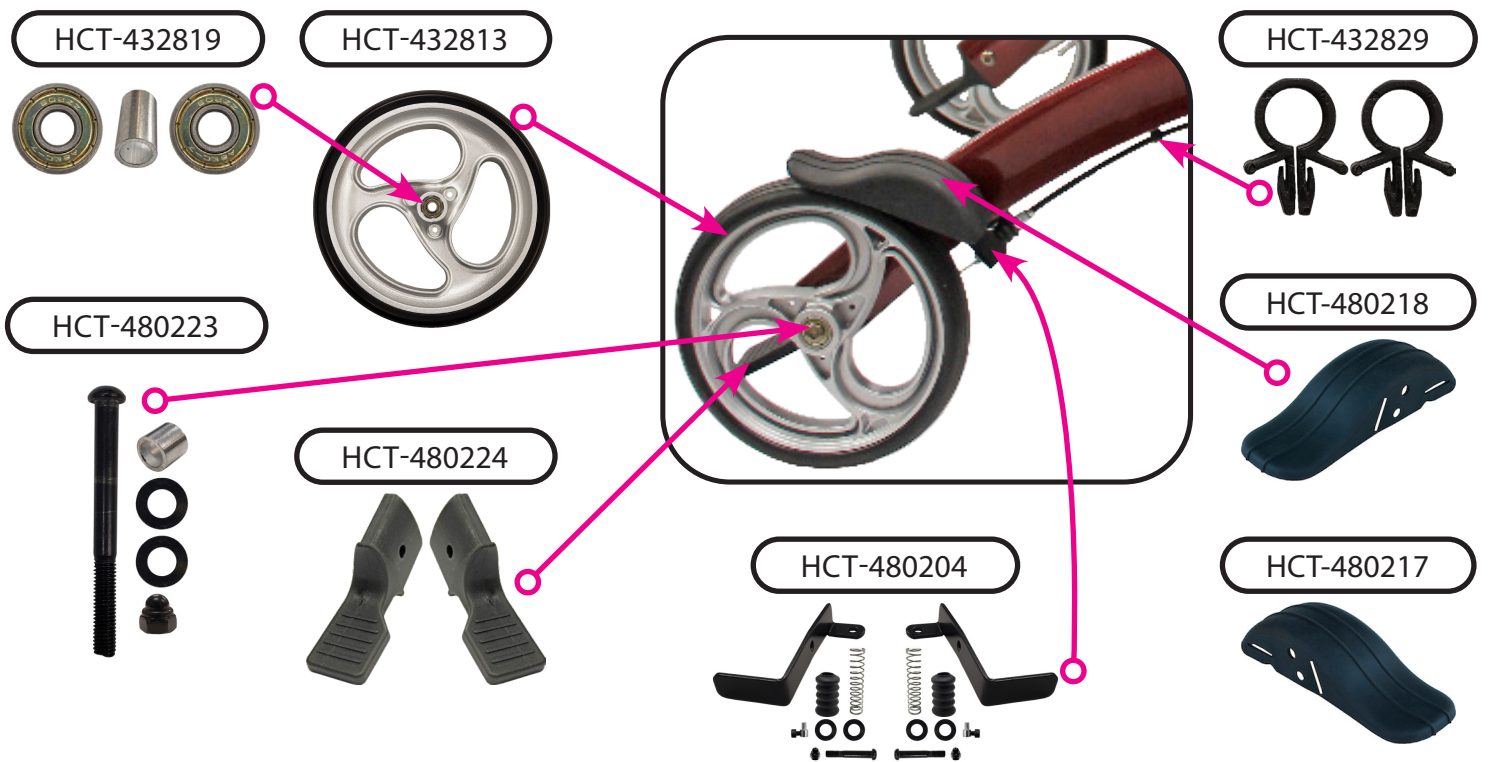

	Item Number 4329	
		
	SERIAL # BEGINNING WITH "HT"	
ITEM NUMBER	DESCRIPTION	UOM
HCT-480209	ADJUSTMENT RELEASE BUTTON	EACH
HCT-432905	ADJUSTMENT SCREW FOR SEAT CABLE & COVER	EACH
HCT-480222	AXLE FRONT	EACH
HCT-480223	AXLE REAR	EACH
HCT-432908	BACKREST	EACH
HCT-432830	BAG	EACH
HCT-480204	BRAKE SHOE (SET)	SET
HCT-432804	CANE HOLDER TRAY	EACH
HCT-480217	COVER FOR LEFT BRAKE	EACH
HCT-480218	COVER FOR RIGHT BRAKE	EACH
HCT-432806	CROSSBAR CENTER BOLT	EACH
HCT-432807	CROSSBAR SIDE BOLT	EACH
HCT-432803	CROSSBAR SIDE BRACKET	EACH
HCT-432903	CROSSBAR SUPPORT LEFT	EACH
HCT-432904	CROSSBAR SUPPORT RIGHT	EACH
HCT-480220	FORK	EACH
HCT-480221	FORK ATTACHMENT	EACH
HCT-432810	FRONT BEARING SET	SET
HCT-480205	FRONT SEAT PLUG	SET
HCT-480219	HAND GRIPS FOR TOP HANDLE BAR	PAIR
HCT-432902	HANDLE BAR RIGHT WITH HAND BRAKE	EACH
HCT-432901	HANDLE BAR WITH HAND BRAKE LEFT	EACH
HCT-432829	HOLDER FOR BRAKE COVER	EACH
HCT-432819	REAR BEARING SET	SET
HCT-432906	RELEASE LEVER LEFT	EACH
HCT-432907	RELEASE LEVER RIGHT	EACH
HCT-432913	SEAT CABLE & COVER	SET
HCT-480113	SEAT PLUG REAR	PAIR
HCT-432823	SEAT WITH HANDLE	EACH
HCT-432910	STEM AND NUT FOR FRONT FORK	EACH
HCT-432909	SWIVEL BEARING SET FRONT FORK	SET
HCT-480224	TOE KICK	PAIR
HCT-432911	WHEEL FRONT 8"	EACH
HCT-432813	WHEEL REAR 8"	EACH