



20" Reclining Wheelchair

Parts Breakdown
*For models: 6200S
Serial # Starting "YH"*



CARE • QUALITY • INNOVATION



THE NOVA PROMISE

Revised on April 13, 2021

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

novajoy.com

An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your wheelchair corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame near the front wheel (see **Figure 1**).


If you need further assistance, please call our customer service team at **(800) 557-6682**.

Thank you!




Figure 1


Main Body Components

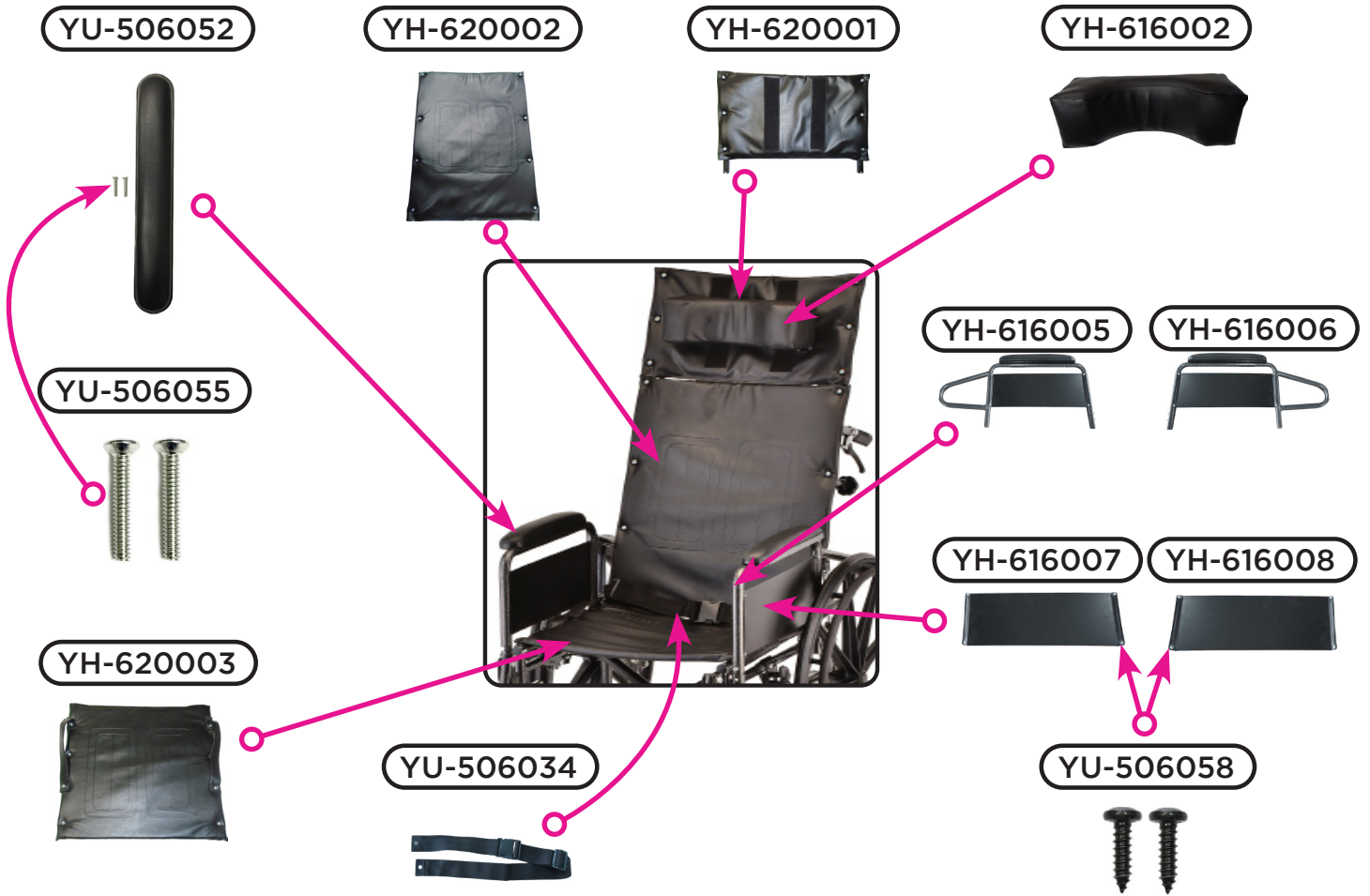


Footrest & Front Wheel Components

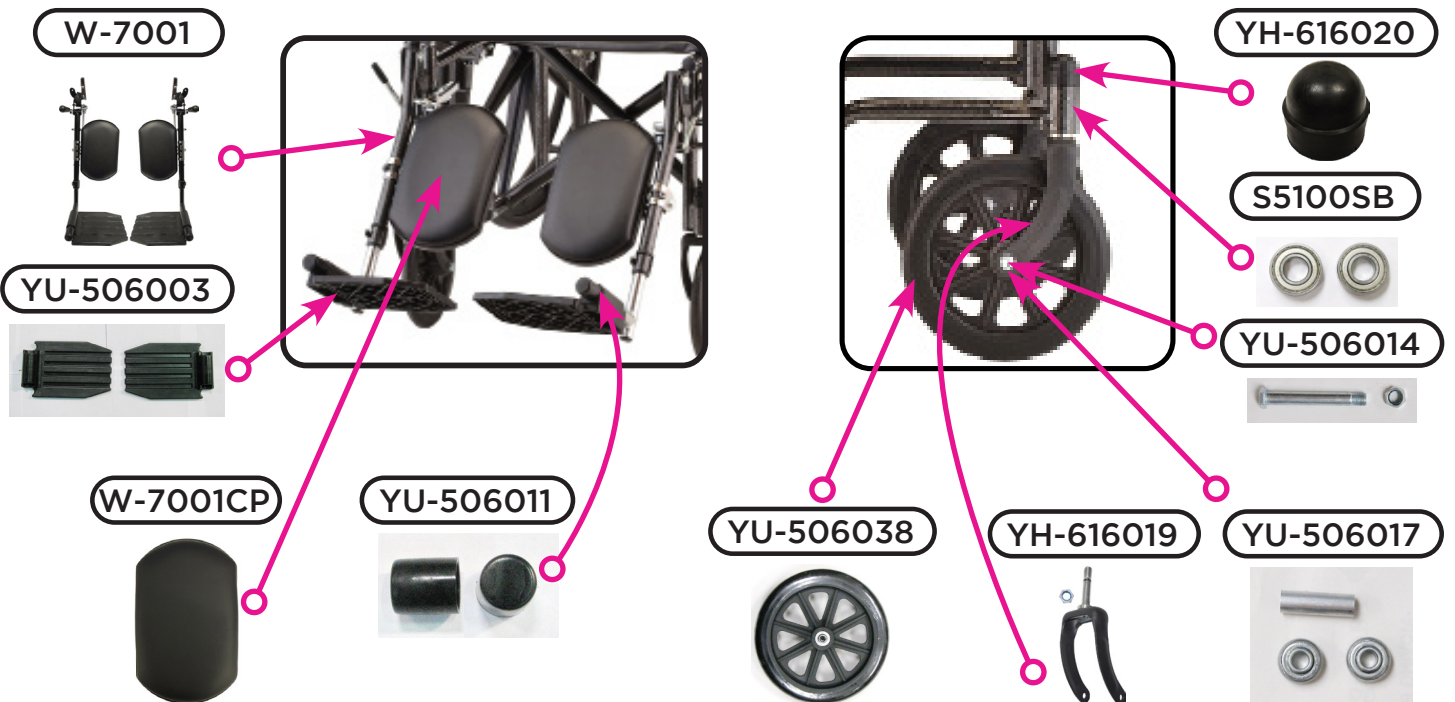

Rear Assembly

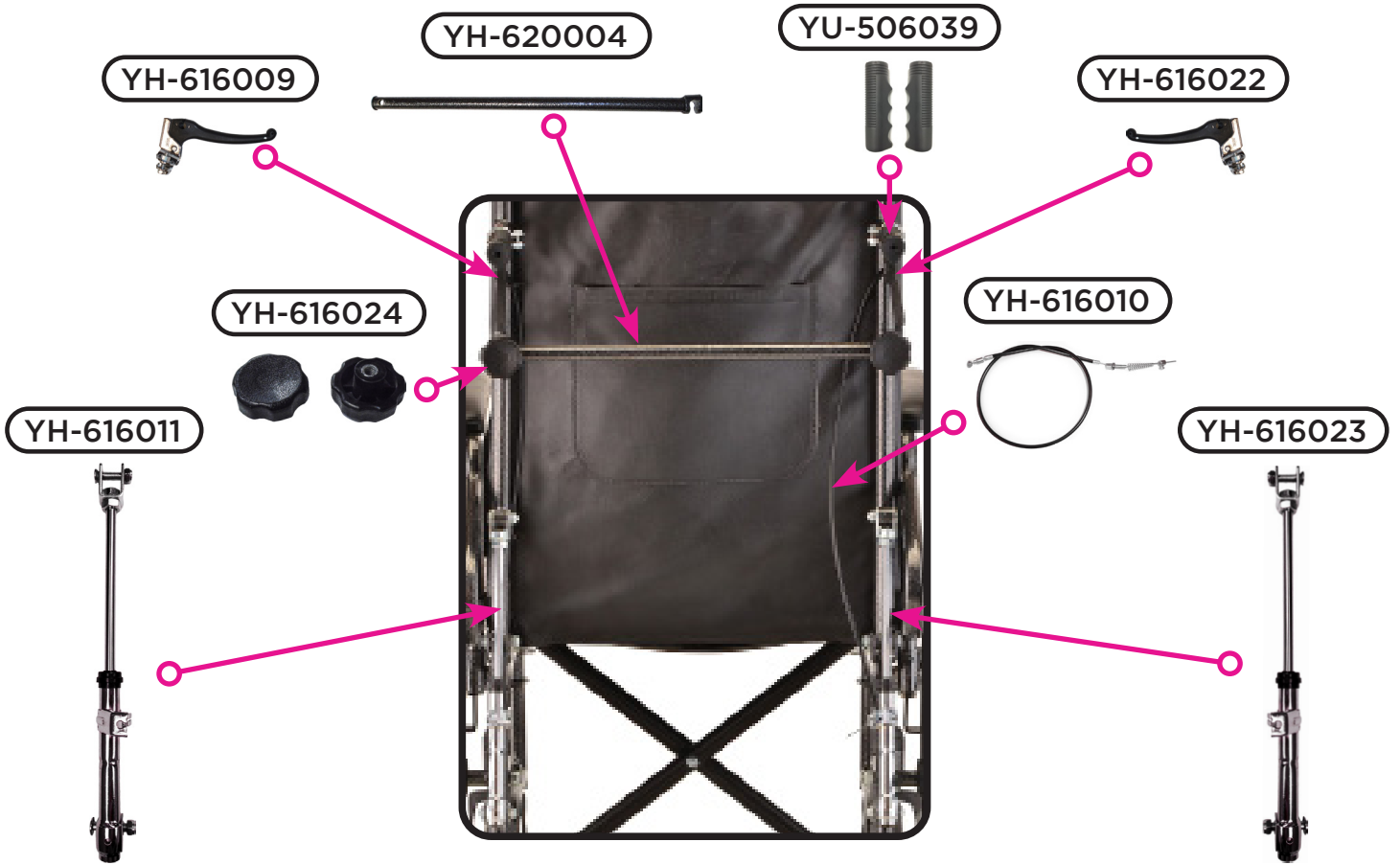

Rear Wheel Assembly

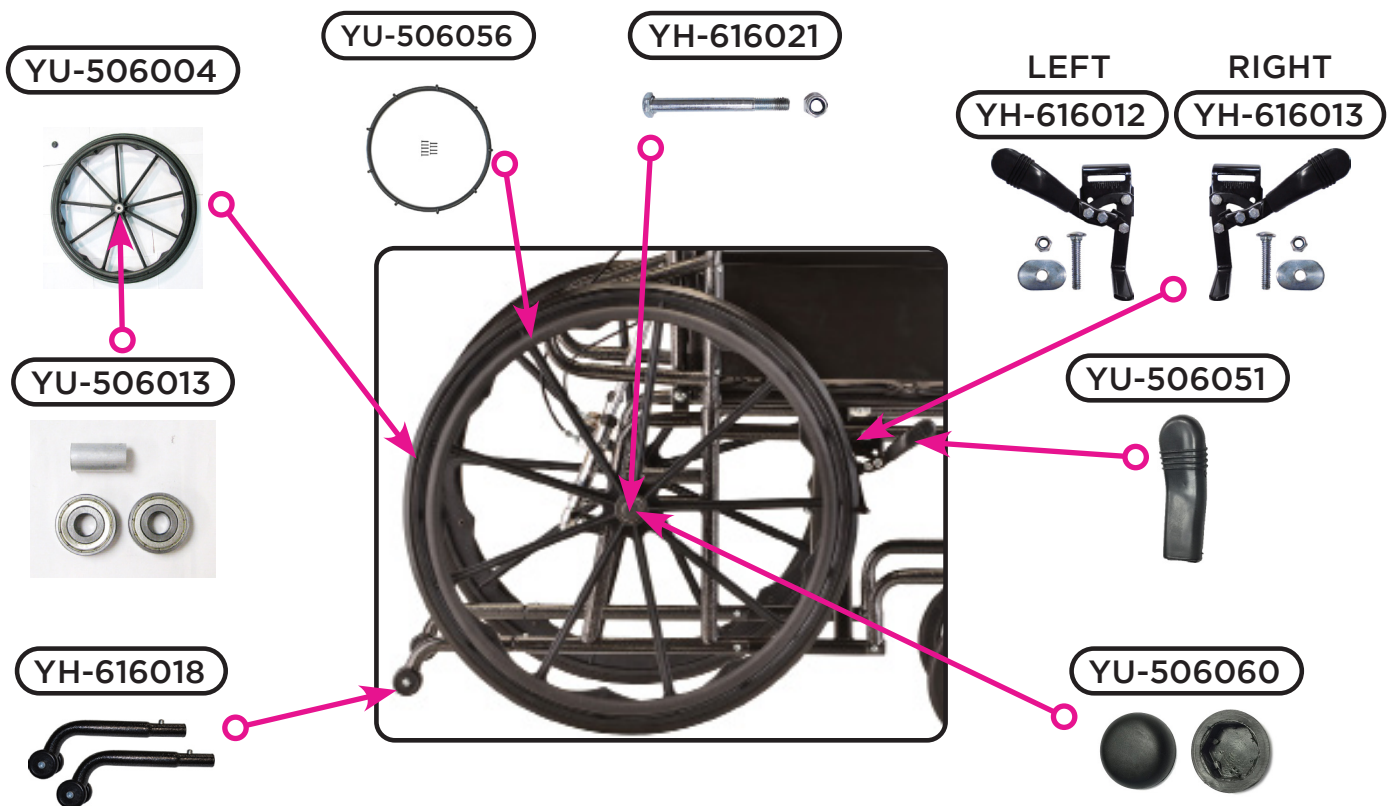



Item Number 6200S

6200 Steel Wheelchair

SERIAL # BEGINNING WITH "YH"

ITEM NUMBER	DESCRIPTION	UOM
YH-616018	ANTI-TIPPER (PAIR)	PAIR
YU-506052	ARM PAD FULL LENGTH WITH SCREW	EACH
YH-616005	ARMREST FULL LEFT	EACH
YH-616006	ARMREST FULL RIGHT	EACH
YH-616021	AXLE REAR	EACH
YU-506014	AXLE SET FOR 8" WHEEL	EACH
YH-620004	BAR SUPPORT FOR BACK	EACH
YU-506017	BEARING & SPACER SET	SET
YU-506013	BEARING & SPACER SET FOR 24" WHEEL	SET
YH-616012	BRAKE LEFT	EACH
YH-616013	BRAKE RIGHT	EACH
YH-616010	CABLE ASSEMBLY RECLINING	EACH
W-7001CP	CALF PAD FOR ELR (ELEVATING LEG RESTS)	EACH
YU-506011	CAP FOR FOOTREST (PAIR)	PAIR
YH-616020	CAP FOR SWIVEL FORK	EACH
W-7001	ELEVATING LEG RESTS	PAIR
YU-506003	FOOT PLATE (PAIR)	PAIR
YH-616019	FORK FRONT	PAIR
YU-506039	GRIP (PAIR)	PAIR
YU-506056	HAND RIM FOR 24" WHEEL	EACH
YH-616009	HANDLE LEFT FOR RECLINING MECH	EACH
YH-616022	HANDLE RIGHT FOR RECLINING MECH	EACH
YH-620001	HEADREST EXTENSION	EACH
YU-506060	HUB CAP FOR 24" WHEEL	PAIR
YH-616024	KNOB (PAIR) FOR BAR SUPPORT	PAIR
YH-616007	PANEL LEFT FOR FULL ARM	EACH
YH-616008	PANEL RIGHT FOR FULL ARM	EACH
YH-616002	PILLOW FOR HEADREST	EACH
YH-616011	RECLINING MECHANISM LEFT	EACH
YH-616023	RECLINING MECHANISM RIGHT	EACH
YU-506055	SCREW FOR ARM PAD (2 PCS)	PAIR
YU-506058	SCREW FOR PANEL/SKIRT GUARD	PAIR



Item Number 6200S


6200 Steel Wheelchair

SERIAL # BEGINNING WITH "YH"

ITEM NUMBER	DESCRIPTION	UOM
YU-506034	SEAT BELT FOR WHEELCHAIR	EACH
S5100SB	SWIVEL BEARING (PAIR)	PAIR
YH-616014	TIP FOR BRAKE	EACH
YH-620002	UPHOLSTERY 20" BACK	EACH
YH-620003	UPHOLSTERY 20" SEAT	EACH
YU-506004	WHEEL 24" REAR	EACH
YU-506038	WHEEL 8" FRONT	EACH

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

novajoy.com

CARE • QUALITY • INNOVATION



THE NOVA PROMISE