



# no<sup>v</sup>o<sup>d</sup>

# 7180

Parts Breakdown  
*For models: 7180L, 7180LE, 7181L, 7181SE  
Serial # Starting "YH"*



CARE • QUALITY • INNOVATION




THE NOVA PROMISE

*Revised on July 14, 2023*

**CONTACT US!**  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)

## An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your wheelchair corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame near the front wheel (see Figure 1).


If you need further assistance, please call our customer service team at (800) 557-6682.

Thank you!



Figure 1


# Main Body Components

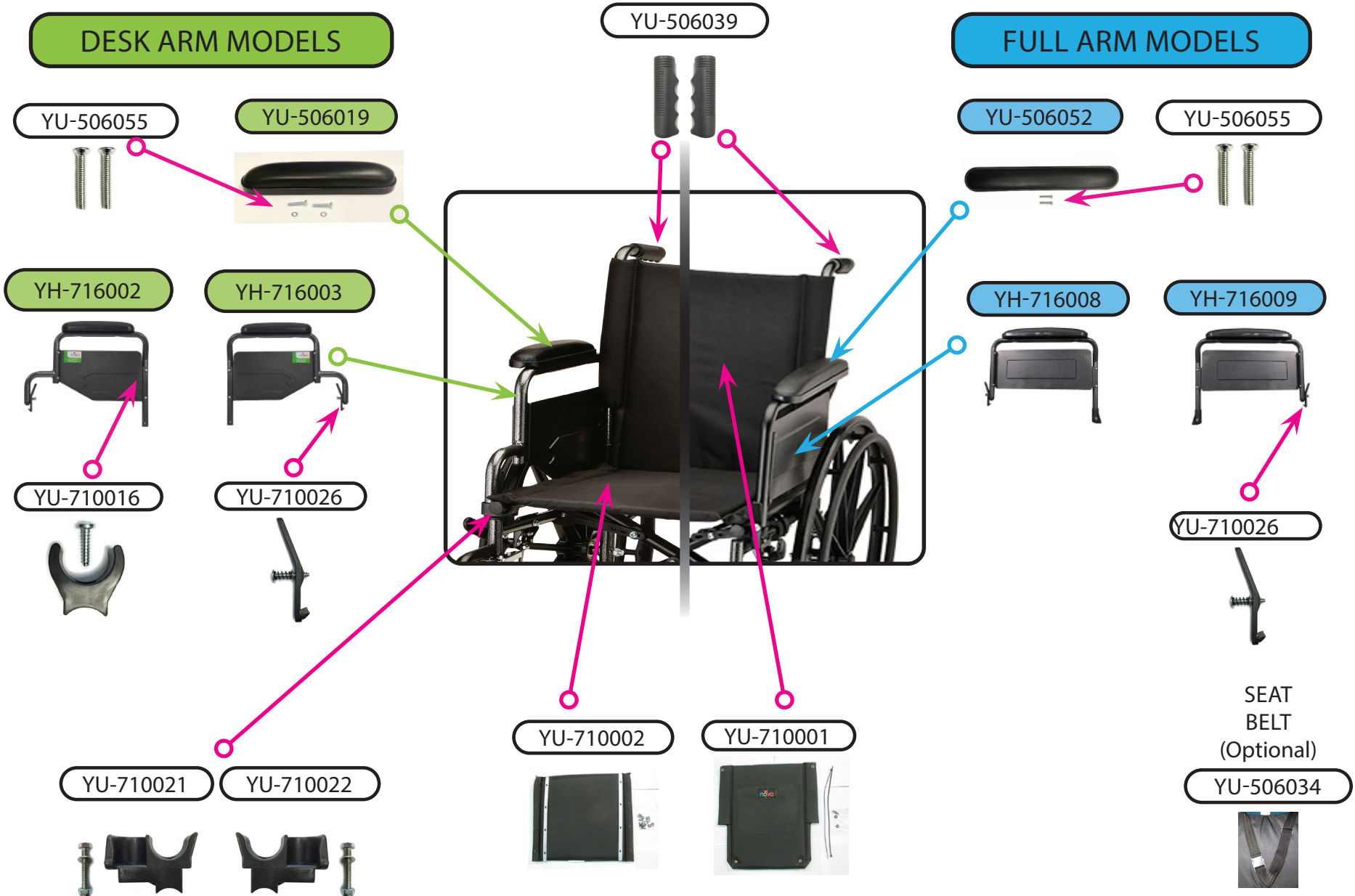

## Rear Frame Components




## Footrest & Lower Frame Components

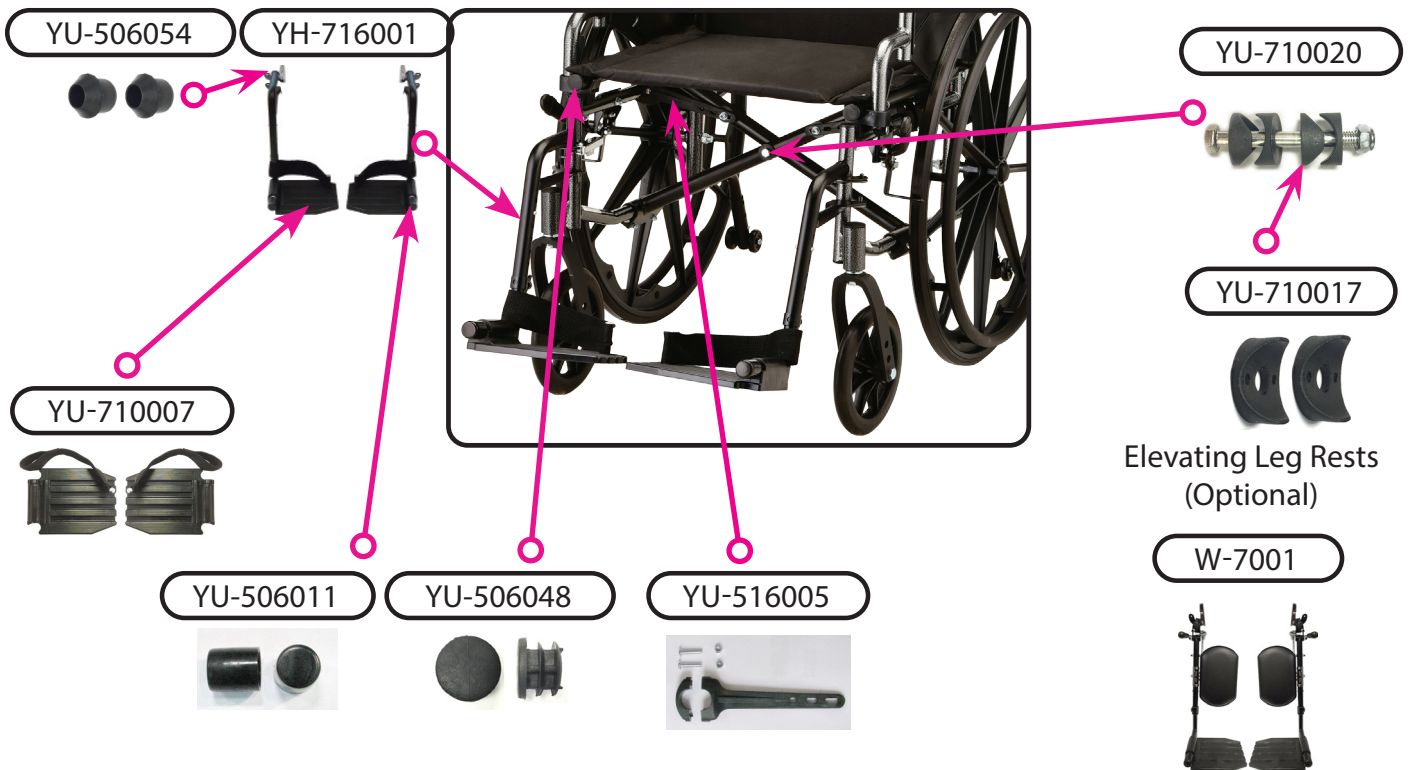




# Front Wheel Assembly

SEE SERIAL NUMBER REFERENCE AT THE BOTTOM

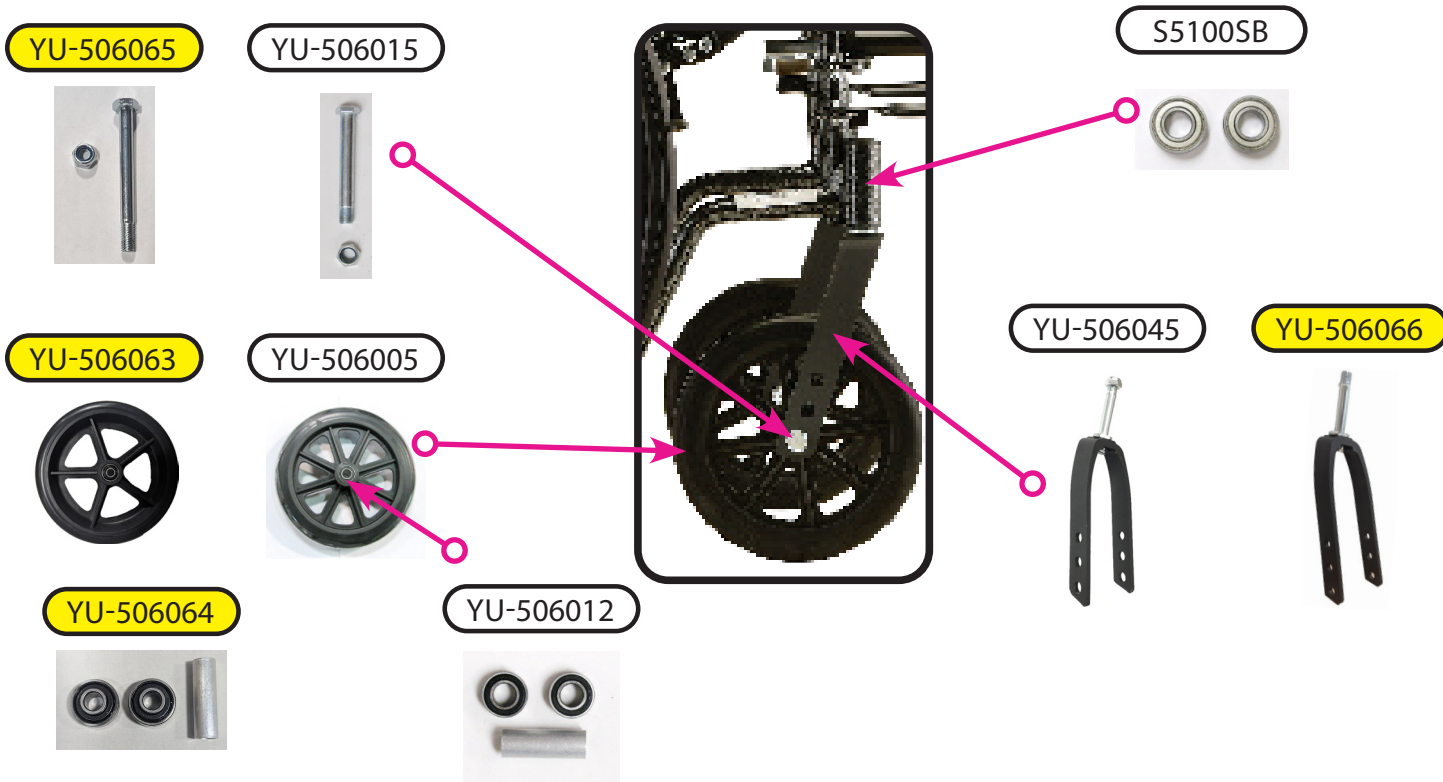

# Rear Wheel Assembly

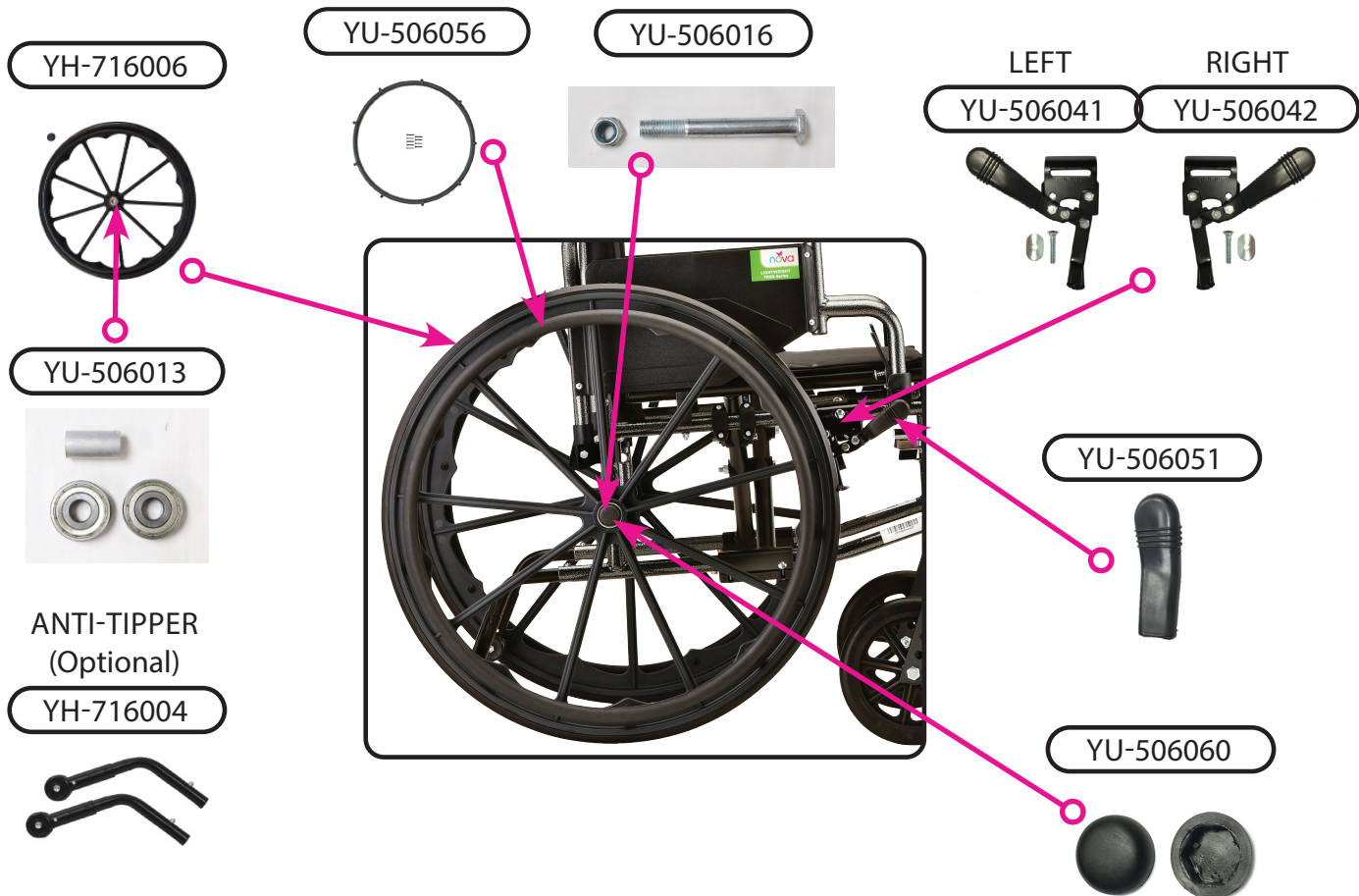




Item Number 7180L/7180LE/7181L/7181LE

**7180 Lightweight Wheelchair**

SERIAL # BEGINNING WITH "YH"

ITEM NUMBER	DESCRIPTION	UOM
YU-710016	ANTI-RATTLE CLAMP AND SCREW	EACH
YH-716004	ANTI-TIPPER BLACK ADJ (PAIR)	PAIR
YU-506019	ARM PAD DESK ARM WITH SCREWS	EACH
YU-506052	ARM PAD FULL LENGTH WITH SCREW	EACH
YU-506015	AXLE SET FOR 8" WHEEL	SET
YU-506013	BEARING & SPACER SET FOR 24" WHEEL	SET
YU-506012	BEARING & SPACER SET FOR 8" WHEEL	SET
YU-506041	BRAKE LEFT	EACH
YU-506042	BRAKE RIGHT	EACH
YU-506011	CAP FOR FOOTREST (PAIR)	PAIR
S5100SC	CRADLE FOR REAR SEAT	EACH
YH-716002	DESK ARM LEFT ALUM H/T	EACH
YH-716003	DESK ARM RIGHT ALUM H/T	EACH
W-7001	ELEVATING LEG RESTS	PAIR
YU-710007	FOOT PLATE (PAIR) WITH HEEL LOOP	PAIR
YH-716001	FOOTRESTS BLACK (PAIR)	PAIR
YU-506045	FORK FOR FRONT WHEEL	EACH
YH-716008	FULL ARM LEFT ALUM H/T	EACH
YH-716009	FULL ARM RIGHT ALUM H/T	EACH
YU-506039	GRIP (PAIR)	PAIR
YU-506056	HAND RIM FOR 24" WHEEL - INCLUDES 9 SCREWS	EACH
YH-716005	HANDLE BAR H/T	EACH
YU-710019	HEEL LOOP	EACH
YU-506060	HUB CAP FOR 24" WHEEL	PAIR
YU-710023	KNOB & SCREW FOR HANDLE BAR	EACH
YU-710026	LEVER TO RELEASE FLIP BACK ARM	EACH
YU-506054	PLUG (PAIR) FOR FOOT REST	PAIR
YU-710021	RECEIVER & CRADLE FRONT LEFT	EACH
YU-710022	RECEIVER & CRADLE FRONT RIGHT	EACH
YU-710024	RECEIVER REAR FOR DESK ARM	EACH
YU-506055	SCREW FOR ARM PAD (2 PCS)	PAIR
YU-710020	SCREW FOR CENTER OF CROSSBAR	EACH
YU-710018	SCREW FOR HEEL LOOP	EACH
YU-506034	SEAT BELT FOR WHEELCHAIR	EACH
YU-710017	SPACER CONTOURED	PAIR
YU-516005	SUPPORT FOR CROSS BAR	EACH
S5100SB	SWIVEL BEARING (PAIR)	PAIR
YU-506051	TIP FOR BRAKE	EACH


	Item Number 7180L/7180LE/7181L/7181LE	
	<b>7180 Lightweight Wheelchair</b>	
	SERIAL # BEGINNING WITH "YH"	
ITEM NUMBER	DESCRIPTION	UOM
YU-710001	UPHOLSTERY BACK FOR 7180 NYLON	EACH
YU-710002	UPHOLSTERY SEAT FOR 7180 NYLON	EACH
YU-506005	8" WHEEL FRONT	EACH
YH-716006	24"WHEEL REAR WITH CAP	EACH

SERIAL NUMBER FOR REFERENCE	
7180L	SN STARTING FROM: YH7180LNK0215

ITEM NUMBER	DESCRIPTION	UOM
YU-506065	AXLE SET FOR 8" WHEEL	SET
YU-506064	BEARING & SPACER SET FOR 8" WHEEL	SET
YU-506066	FORK FOR FRONT WHEEL	EACH
YU-506063	WHEEL 8" FRONT	EACH

**CONTACT US!**  
We're here to help

 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)

CARE • QUALITY • INNOVATION



THE NOVA PROMISE