

# HOW TO ASSEMBLE YOUR



## EXERCISE PEDDLER

Item # 6002-R



NO tools required  
Quick assembly

Easy to follow  
step-by-step  
instructions

### WHAT'S IN THE BOX

- a. (1) Peddler
- b. (1) Digital display
- c. (1) Tension control knob
- d. (1) AAA battery



a.



b.



c.



d.

## 1 Instructions for use

Remove unit from all packaging. Open unit by unfolding peddler legs. An audible “click” will be heard when push buttons are engaged.

**Caution:** Take care not to pinch fingers where unit unfolds and locks into place.

Place exercise peddler level on floor to use with legs or other flat surface to use with arms. Position peddler at a comfortable distance and rotate pedals, ensuring straps are used appropriately.

## 2 Adjusting the tension control knob

The tension control knob may be tightened or loosened to increase or decrease the tension required to use the product.

## 3 Digital Display

For units with a digital display, the display will automatically power on when you start to exercise. When the exercise peddler is not used for 5 seconds, a “STOP” indicator in the top left corner of the window will appear, and the timer will also stop. The timer will resume when you continue exercising.

The digital display will power off after 5 minutes of inactivity. To reset the digital display, hold the red button for 10 seconds.

### Readout

- A) Exercise time
- B) Revolution count from last reset
- C) Total revolutions for all sessions
- D) Calories burned from last reset

To select the information to display, press red button, and indicator arrow will point to the desired data.

Selecting “SCAN” will automatically prompt the digital display to cycle through all data in the above order.



**NOTE:** If numbers begin to fade or show unusual characters, replace the battery.

## 4 To install battery

This unit uses one 1.5V (AAA) battery (included)

Unscrew and remove tension control knob, and lift the digital display off of exercise peddler. Depress tab on bottom of digital display, and gently pull apart and open the case. Install or replace battery. Close the case, and replace display on peddler. Screw on tension control knob.



\*Revised February 19, 2019

## **WARNING**

Read all instructions carefully and consult with a qualified professional before assembling to avoid possible injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper assembly or misuse of this product.**

- Check push buttons before each use
- Use on a level surface
- **DO NOT** attempt to stand while using exercise peddler

## **CARE AND MAINTENANCE**

Periodically check the tension control knob to ensure the desired tension is engaged. Check all push buttons are engaged and that straps are fully secure on each pedal before each use. Wipe down with damp, non-abrasive cloth to clean. Ensure battery is in working condition and that no acid build-up is visible.

## **WARRANTY INFORMATION**

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	<b>Limited Lifetime Warranty</b>
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	<b>One Year Limited Warranty</b>

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.


**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

### **IMPORTANT WARNING**

**FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

**CONTACT US!**  
We're here to help


 [info@novajoy.com](mailto:info@novajoy.com)


 1 800 557 6682


[novajoy.com](http://novajoy.com)


*Connect with us!*

[novajoy.com](http://novajoy.com)

 [novamedicalprod](https://www.facebook.com/novamedicalprod)

 [novamedicalprod](https://twitter.com/novamedicalprod)

 [novawalkerstalk](https://twitter.com/novawalkerstalk)

 [NOVA Medical Products](https://www.youtube.com/NOVAMedicalProducts)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE