

# HOW TO ASSEMBLE YOUR



## I-V POLE STAND - 2 HOOK

ITEM # 6081



QUICK ASSEMBLY

Follow these SIMPLE STEPS


**1** Remove all parts from box. (Be sure all parts are included before assembling)

- A. (1) I-V pole with extension
- B. (1) Bottom base with guide pin (labeled)
- C. (1) Top base with guide pin hole (labeled)
- D. (1) I-V hook


**TOOLS NEEDED:  
WRENCH**




**2** Unscrew and remove the bolt and washer from the bottom of the I-V pole.

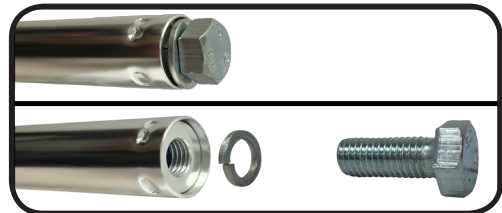


**3** Assemble top and bottom base pieces perpendicularly by aligning center holes and the guide pin to the smaller hole. Insert the bolt through the small lock washer and through base pieces. Hold I-V pole with one hand and tighten screw with the appropriate wrench. **DO NOT** tighten solely by hand.

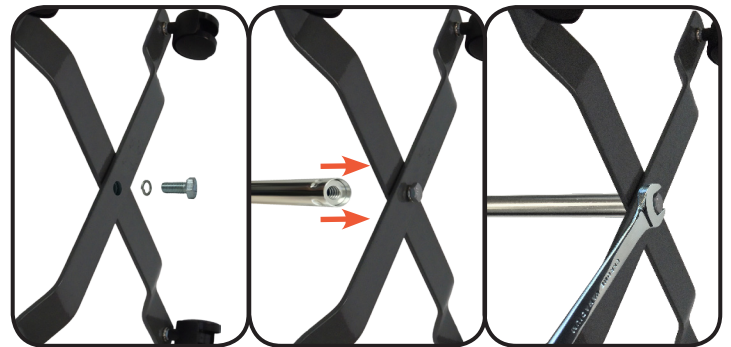

**NOTE:** Sometimes, guide pin may loosen while tightening. Tap the pin appropriately with a hammer or mallet to reset.

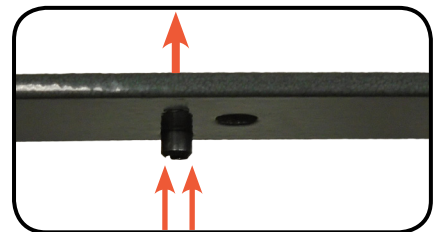

**4** Screw on the I-V hook clockwise onto the top of the I-V pole. Tighten by hand until it is secure.

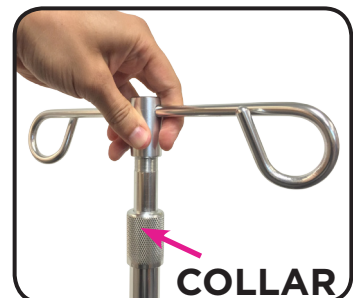

**5** To adjust the I-V pole, loosen the collar by turning it counter-clockwise. Extend or lower the top half of the pole. Once a desired height is found, turn the collar clockwise until it is tightened and the top half of the pole is secure.

## WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

Regular cleaning is recommended. Use a non abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.

**IMPORTANT WARNING**  
**FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

- Use on a level surface
- **DO NOT** use the I-V pole for support, a stand assist, or as a walking aid
- Check for stability before each use

**CONTACT US!**  
**We're here to help**

 [info@novajoy.com](mailto:info@novajoy.com)  
 1 800 557 6682

CARE • QUALITY • INNOVATION



THE NOVA PROMISE

Connect with us!  
novajoy.com  
facebook: novamedicalprod  
twitter: novamedicalprod  
twitter: novawalkerstalk  
YouTube: NOVA Medical Products