

## HOW TO INSTALL YOUR



### RAISED TOILET SEAT

Item # 8340/8340-R

Capacity not to exceed 300 lb



NO tools required.  
Quick installation.

Easy to follow  
step-by-step  
instructions

#### WHAT'S IN THE BOX

- a. (1) Raised Toilet Seat



a.

### Pre-Installation Instructions


- 1 Make sure the toilet is clean and sanitized. Lift up toilet seat and lid.

**NOTE:** You may also need to remove the entire seat and lid from the toilet (varies depending on the size of the toilet).

### Installation Instructions

- 1 Set the Raised Toilet Seat onto the toilet (**Figure 1**). Ensure the raised lip underneath the riser sits inside the toilet bowl.

**NOTE:** The 8340 Raised Toilet Seat does not lock onto the toilet.

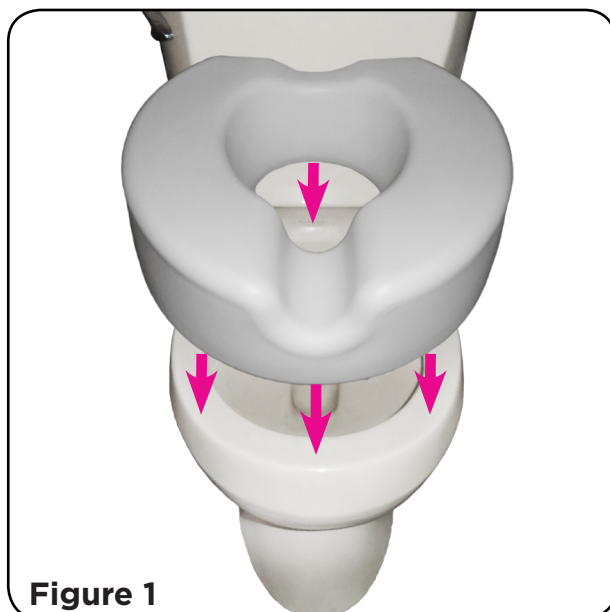

**Figure 1**

#### **WARNING**

Read all instructions carefully before use. Failure to do so may result in serious injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper use of this product.** This product fits most standard sized toilets.

#### **CARE AND MAINTENANCE**

Check to make sure the Raised Toilet Seat is set on the toilet evenly before using. Clean regularly with a toilet sanitizer or cleanser.

## WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

|   |                           |
|---|---------------------------|
| Frame   | Limited Lifetime Warranty |
| Non-Metal/Metal parts<br>(rubber tips, plastic or metal hardware) | One Year Limited Warranty |

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.



For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.


**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

.....  
Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and its parts for any corrosion, breakage, compression or wear.  
.....

**IMPORTANT WARNING**  
**FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

**CONTACT US!**  
We're here to help  
 [info@novajoy.com](mailto:info@novajoy.com)  
 1 800 557 6682  
[novajoy.com](http://novajoy.com)

*Connect with us!*  
novajoy.com  
 novamedicalprod  
 novamedicalprod  
 novawalkerstalk  
 NOVA Medical Products

CARE • QUALITY • INNOVATION  
    
THE **NOVA** PROMISE