

HOW TO INSTALL YOUR

**nova**

Toilet Seat Riser


Item #'s 8341-R, 8342-R
Capacity not to exceed 300 lbs.



**NO tools required.
Quick assembly.**

**Follow these
SIMPLE STEPS**


WHAT'S IN THE BOX

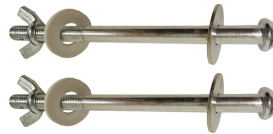

- 1** Remove all parts from box:
 - (1) toilet seat
 - (2) mounting screws
 - (2) metal washers
 - (2) plastic washers
 - (2) wing nutsClean and sanitize toilet seat.
- 2** Remove the existing toilet seat and lid from the toilet bowl by unscrewing the seat screws located at the rear of the seat.
- 3** Place the toilet seat riser on the toilet bowl. The mounting holes on the rear of the riser should line up with the holes on the rear of the toilet bowl.
- 4** Replace the toilet seat and lid so the holes of the toilet seat, the riser, and the bowl line up. Place metal washer onto screw.

Insert the mounting bolts with washer through top of the seat and lid, seat riser, and toilet bowl.
- 5** Place plastic washers at end of screws and then attach the provided wing nuts to each mounting screw. Do not tighten completely yet.

Be sure the toilet seat is straight and sits flush on the seat and then fully tighten

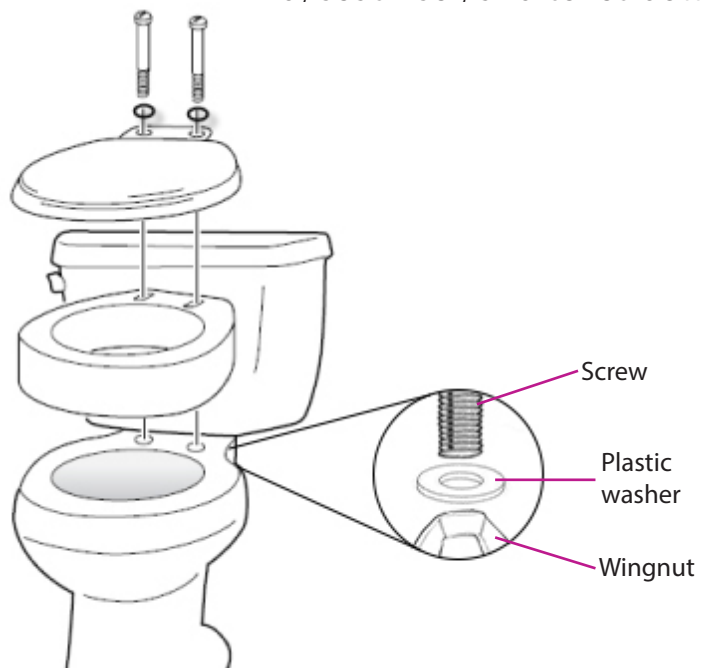

WARNING

Read all instructions carefully and consult with a qualified professional before installing to avoid possible injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper installations or misuse of this product.**

- Ensure seat is secure before each use
- DO NOT stand on the seat
- Supervision or assistance when using this product is recommended for users with limited physical capabilities

HOW TO USE

Back up to Toilet Seat Riser until legs touch, then slowly lower yourself onto seat. Apply weight evenly to riser.

CARE AND MAINTENANCE

Check regularly to make sure the Toilet Seat Riser is securely mounted to the toilet bowl. Use a mild detergent and warm water to clean.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and its parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE

Connect with us!
novajoy.com
facebook: novamedicalprod
twitter: novamedicalprod
twitter: novawalkerstalk
YouTube: NOVA Medical Products