

HOW TO ASSEMBLE YOUR



DROP ARM COMMODE

Item # 8900W

Capacity not to exceed 300 lb

WHAT'S INCLUDED

- a. (1) Commode frame with seat and lid
- b. (1) Backrest
- c. (1) Bucket with lid
- d. (1) Splash guard



a.



b.



c.



d.

NO tools required  
Quick assembly

Follow these  
SIMPLE STEPS

- 1** Place commode on front side. Depress push buttons on backrest, and insert into the backrest receivers located on the bottom of the seat platform. Ensure push buttons are fully engaged and check by pulling upward (see Figure 1).

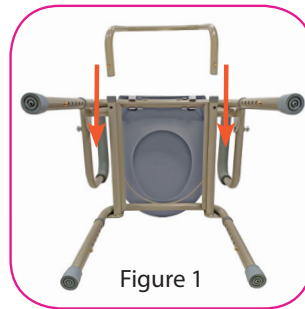


Figure 1

- 2** Adjust the height of the legs by depressing the push button on each leg and sliding the legs up or down to desired height (see Figure 2).



Figure 2

NOTE: Legs must be adjusted to same height so commode sits level.

- 3** To lower the drop arms, push in the chrome spring-loaded release button and swing drop arm down (see Figure 3).

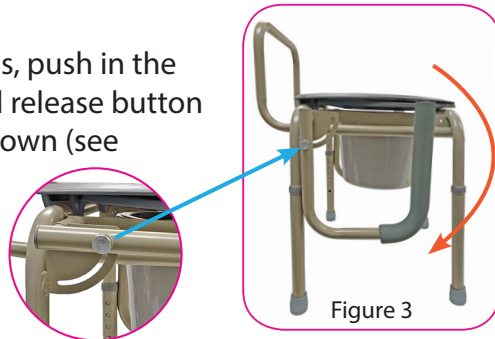


Figure 3

- 4** To raise the drop arms, swing drop arm up until the red spring-loaded release button completely pops out and is locked (see Figure 4).



Figure 4

- 5** Lift seat & lid up and place the bucket on top of the bucket hanger then slide the bucket so it locks in place. (see Figure 5 & 6).



Figure 5

**Installing the splash guard:** Repeat step 5 (see Figure 7 & 8).



Figure 6

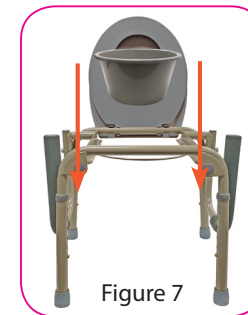


Figure 7

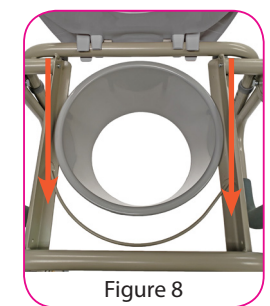


Figure 8

## WARNING

Read all instructions carefully and consult with a qualified professional before assembling to avoid possible injury. NOVA Medical Products shall have no liability for any injury resulting from improper assembly or misuse of this product.

- Ensure ALL leg/backrest push buttons are engaged before using
- All legs MUST be adjusted to the same height to ensure level seating and safety
- Use EXTREME caution on wet tub or floor surfaces
- DO NOT stand on commode
- DO NOT use if commode is damaged. Check frequently for cracks, damage, or looseness.

## HOW TO USE

Lift the seat and place the bucket in the center of the commode in between the bucket hangers. When using the commode over a toilet, replace the bucket with the splash guard. The commode should be low enough to ensure that your feet touch the floor when sitting on it, but high enough to enable easier sitting and standing. After use, remove the bucket, discard waste and sanitize.

## CARE AND MAINTENANCE

Check regularly to make sure all arms and legs on the commode are securely locked. Use a mild detergent and warm water to clean. Dry with a soft cloth to avoid moisture build-up.

## WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts	One Year Limited Warranty (rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.


THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

### IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

### CONTACT US! We're here to help


 [info@novajoy.com](mailto:info@novajoy.com)

 1 800 557 6682

[novajoy.com](http://novajoy.com)

### Connect with us!

[novajoy.com](http://novajoy.com)

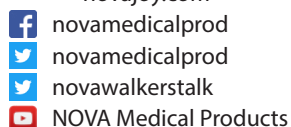

CARE • QUALITY • INNOVATION



THE NOVA PROMISE