

HOW TO ASSEMBLE YOUR

nova

Drop Arm Commode

Item # 8901W
Capacity not to exceed 300 lb



QUICK ASSEMBLY

Follow these
SIMPLE STEPS




Bucket & Lid



Splash Guard



Back Rest

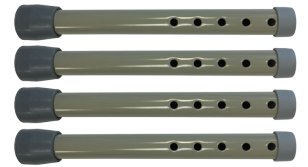

Leg Extensions

- 1** Remove all parts from box:
1 drop arm commode with seat and lid, 1 bucket with lid, 1 splash guard, 1 backrest, and 4 leg extensions.
- 2** To adjust the legs and seat height, place the seat face down and depress the silver push button on one of the legs of the commode and slide a leg extension over it. Lengthen or shorten the leg extension to the desired position. Once the desired position is found, ensure that the silver push button protrudes from the adjustment hole of the leg extension and is fully engaged so that it does not move. Adjust the other three legs to the same length, and make sure that the commode sits level.
- 3** Lay the front end of the commode face down and insert the backrest into the tube slots under the seat. Depress the silver push buttons on each side of the backrest until they protrude from the adjustment hole and are fully engaged into the user's desired setting. Ensure the silver push buttons are fully engaged by pulling on the back.
- 4** Lift the seat and place the bucket in the center of the commode in between the bucket hangers. When using the commode over a toilet, replace the bucket with the splash guard.



HOW TO USE YOUR Drop Arm Commode

- 1** **Lowering the drop arms:** On the side of the arms, push in the RED spring loaded release button and push down on the drop arm.
Raising the drop arms: Pull up on the arms until the RED spring loaded release button completely pops out and is locked.
- 2** The commode should be low enough to ensure that your feet touch the floor when sitting on it, but high enough to enable easier sitting and standing.
- 3** After use, remove the bucket, discard waste and sanitize.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

| | |
|---|---------------------------|
| Metal Frame | Limited Lifetime Warranty |
| Non-Metal/Metal parts (rubber tips, plastic or metal hardware) | One Year Limited Warranty |

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

**READ THESE DIRECTIONS CAREFULLY BEFORE USE.
FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

- Be sure seat is in place and secure before each use.
- Do not stand on seat.

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

CARE • QUALITY • INNOVATION



THE NOVA PROMISE

Connect with us!

novamedicalproducts.com
facebook: [novamedicalprod](#)
twitter: [novamedicalprod](#)
twitter: [novawalkertalk](#)
pinterest: [novamedicalprod](#)