

# HOW TO ASSEMBLE YOUR



DELUXE SHOWER CHAIR/COMMUNE

Item # 9072/9072-R

Capacity not to exceed 300 lb

## WHAT'S INCLUDED

- |                |                                  |   |
|----------------|----------------------------------|---|
| a. (1) Frame A | c. (1) Backrest & Extension Bars | e. (2) Spacer                                 |
| b. (1) Frame B | d. (1) Handle                    | f. (2) Knobs                                  |
|                |                                  | g. (2) Legs w/extensions & large suction tips |
|                |                                  | h. (2) Legs w/extensions & small suction tips |
- 

NO tools required  
Quick assembly

Follow these  
SIMPLE STEPS

- Place both transfer bench frames upside-down on a flat surface. Insert Frame A into Frame B (see Figure 1). Secure both frames by inserting the spacer and knob in the hole next to the backrest slots and turn the knob clockwise until both sides are secure (see Figure 2)

Figure 1


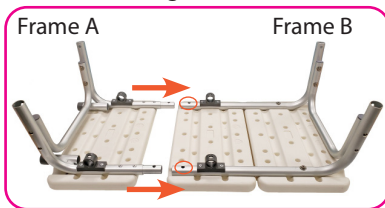


Figure 2

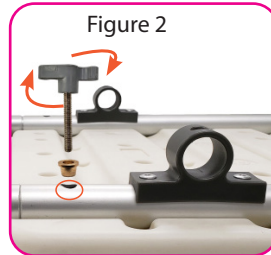

- Depress push buttons on each adjustable suction leg and insert leg onto the side with handle bracket. (see Figure 3). Repeat step 2 on the opposite side with the small suction tip legs (see Figure 4). Adjust all legs so the bench is parallel to the floor, and ensure all push buttons are engaged before using (see Figure 5).

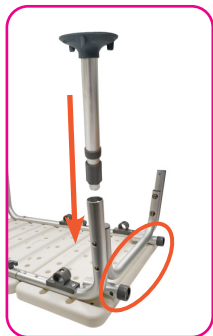


Figure 3

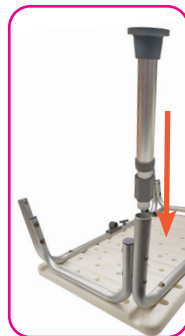


Figure 4



Figure 5

NOTE: Suction leg side of bench goes into the bathtub. Leg heights may vary depending on the difference in height of bathtub and the floor.

- Insert the two backrest extension bars into the backrest tubes until the push button "clicks" into locked position (see Figure 6).



Figure 6

- Turn the bench on its side and insert the backrest into the 4 tubing slots under the seat. Depress the push button on the backrest and ensure that the push buttons "click" and lock into position from the top tubing slots (see Figure 7).

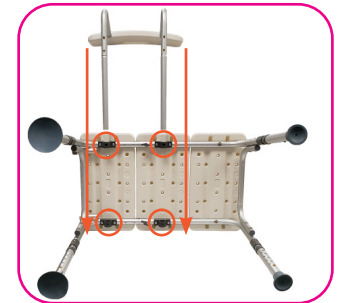


Figure 7

NOTE: The backrest can slide on either side of the bench depending on the direction the you will be sitting.

- Set transfer bench upright on its legs. Depress push buttons on handle, and insert into the slots on the side of the bench (see Figure 8).



Figure 8

## **WARNING**

Read all instructions carefully and consult with a qualified professional before assembling to avoid possible injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper assembly or misuse of this product.**

- Ensure ALL leg push buttons are engaged before using
- All legs **MUST** be adjusted to the same height to ensure level seating and safety
- Use **EXTREME** caution on wet tub or floor surfaces
- **DO NOT** stand on transfer bench
- **DO NOT** use if transfer bench is damaged. Check frequently for cracks, damage, or looseness.
- This product is rated for persons weighing up to **300 lb** when properly assembled

## **HOW TO USE**

Place suction tip legs into bathtub. Ensure the bench is stable and not wobbly. Adjust leg heights if necessary. Slowly seat yourself onto edge of transfer bench, and carefully slide towards inside of bathtub, using handle to stabilize as needed.

## **CARE AND MAINTENANCE**

Check regularly to make sure all legs on the transfer bench are securely locked. Use a mild detergent and warm water to clean. Dry with a soft cloth to avoid moisture build-up.

## **WARRANTY INFORMATION**

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under **NORMAL** use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame

Non-Metal/Metal parts

**Limited Lifetime Warranty**

**One Year Limited Warranty**

(Rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.


**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

### **IMPORTANT WARNING**

**FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

**CONTACT US!**  
We're here to help


 [info@novajoy.com](mailto:info@novajoy.com)


 1 800 557 6682


[novajoy.com](http://novajoy.com)


**Connect with us!**

[novajoy.com](http://novajoy.com)

 [novamedicalprod](https://www.facebook.com/novamedicalprod)

 [novamedicalprod](https://twitter.com/novamedicalprod)

 [novawalkerstalk](https://twitter.com/novawalkerstalk)

 [NOVA Medical Products](https://www.youtube.com/channel/UC...)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE