

HOW TO INSTALL YOUR



SWIVEL CUP HOLDER FOR STAR SERIES

Item # CH-3200R



CONTACT US!
We're here to help

✉ info@novajoy.com

☎ 1 800 557 6682

TOOLS NEEDED



Phillips Screwdriver

**Quick installation
with step-by-step
instructions**

WHAT'S INCLUDED

a. (1) Cup holder



a.

b. (1) Attachment mount



b.


c. (2) Phillips screws



c.



- 1** **To install:**
Using a Phillips screwdriver, unscrew attachment mount from cup holder. Attach cup holder to rear right leg in desired position. Cup holder should face away from frame and swivel back and around the leg.

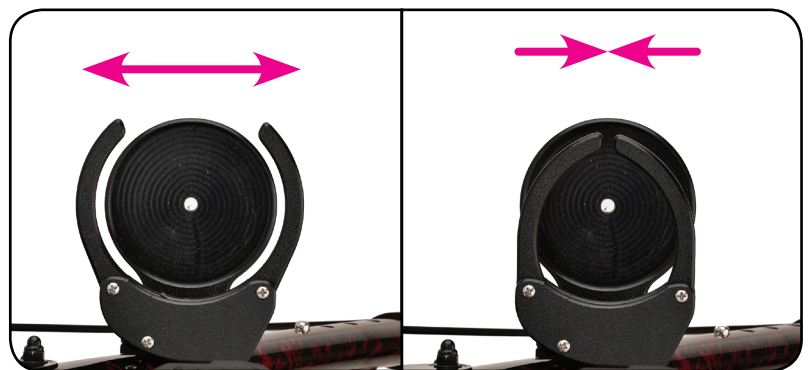

- 2** **To use:**
Carefully swing cup holder towards rear of walker to swivel out of the way. Swing back, when desired, until magnets engage. Pull arms apart or push arms together to adjust to different cup sizes.