



Spring 2021

nova

Rolling Walkers





What makes NOVA Rolling Walkers so unique?

1. Our patented locking hand brakes



NOVA's patented hand brakes are designed for maximum comfort and control. They allow users to easily control the speed by pulling up and engage the parking brake by pushing down. Users with weak or small hands need not worry - our hand brakes are the **best and easiest to use**. Most come with reflectors for safety and good looks!

2. NOVA Quality

At NOVA, we stand by the quality of our products. That's why each NOVA rolling walker comes with a **lifetime warranty** on the frame.

3. Excellence in Service

When you purchase a NOVA Rolling Walker, you are also receiving the **best in customer service**. Our NOVA hand brakes come with a **5 year warranty**, and our dedicated customer service representatives are standing by to answer any questions you may have.

Tips for Choosing Your NOVA Rolling Walker

Walkers are NOT one size fits all

To determine the correct seat height - sit on the seat with your back to the backrest. Your feet should both be flat on the ground with your thighs resting comfortably on the seat. Make sure your weight is below the listed weight capacity and you can sit comfortably between the handles.

Choose the features and options that go with your lifestyle

With your NOVA Walker you will now be on the go! - so choose the features that compliment and support your independence. How about storage for your personal belongings? What size wheels are best for your walking terrain? Be sure to consider great options like a cup holder, flashlight, or mobility clutch!


Get your Style ON


Choose the walker color and designer mobility bags to compliment your personal style. You can choose for YOU and walk with pride and style.

Before you take your NOVA for a test drive - be sure to:

- Make sure the backrest is attached.
- Adjust the handles to your height. Adjust the hand grips to your wrist for a slight 30 degree bend at the elbow.
- Read and understand all user warnings.
- Practice engaging the hand brakes and locking the hand brakes prior to use.

Learn More Online

-  (NOVA Medical Products)
- NOVA Website: novajoy.com

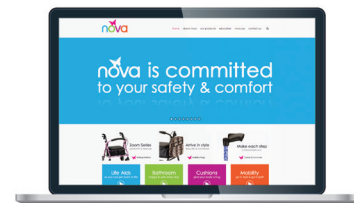

STAR FEATURES

NOVA's STAR series rollators offer some new and exciting features ...



LOCK & LIFT

Keeps rollator locked in a folded position for easy loading and unloading




Scan here to watch the How to Use Video!



HANDLE FOR EASY FOLDING

Built-in handle for easy folding

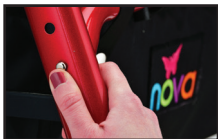

PUSH-BUTTONS

Quick-Fit push-buttons to adjust and remove handles, adjust seat height and remove legs



FOOT PEDALS

Built-in Foot Pedals to aid with curbs, terrain and folding



PADDED SEAT AND UNDER-SEAT POUCH

Super thick and comfortable padded seat and large under-seat pouch with removable plastic liner



Get the Perfect-Fit!

- Standard & Petite models
- Adjustable seat heights & handle heights
- Accommodates a wide range of user heights

The NOVA STAR shines with more features, options, colors and style. Its *Quick-Fit* adjustable design gives the user a perfect fit in just seconds, also making it so easy to pack up for travel, adventure and storage.



Scan here to watch our quick video and learn more!

19.5", 20.5",
21.5" & 22.5"
Adjustable
Seat Height

COMES STANDARD

- Lock & Lift
- Locking Hand Brakes
- Quick-Fit Push-button Adjustable
- Padded Seat w/ Under Seat Pouch
- Built-in Foot Pedals
- Oval-shaped Tubing
- Extended Folding Handle
- 6" Wheels

OPTIONS

- Cup Holder (CH-3200R)
- Cane Holder (CH-4200R)
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

Colors: Black, Blue, Diamond Sky Blue, Purple, Red

WEIGHT CAPACITY: 300 lb

WEIGHT: 17.5 lb

SEAT HEIGHT: 19.5", 20.5", 21.5" & 22.5"

SEAT DIMENSIONS: 14.5"w x 13.5"d

HANDLE HEIGHT ADJ: 32" - 37"

WIDTH BETWEEN HANDLES: 18.75"

OVERALL DIMENSIONS (HIGH): 24"w x 28.5"d

APPROX. USER HEIGHT: 5'4" - 6'2"

POUCH DIMENSIONS: 14.5"w x 8"d x 6.5"h

DIM. WHEN FOLDED: 24.75"w x 11.5"d x 42.75"h



STAR 6 Petite

4276

The NOVA STAR shines with more features, options, colors and style. Its *Quick-Fit* adjustable design gives the user a perfect fit in just seconds; also making it so easy to pack up for travel, adventure and storage.



Scan here to watch our quick video and learn more!

17", 18" & 19"

Adjustable
Seat Height

COMES STANDARD

- Lock & Lift
- Locking Hand Brakes
- Quick-Fit* Push-button Adjustable
- Padded Seat w/ Under Seat Pouch
- Built-in Foot Pedals
- Oval-shaped Tubing
- Extended Folding Handle
- 6" Wheels

OPTIONS

- Cup Holder (CH-3200R)
- Cane Holder (CH-4200R)
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

Colors: Blue, Purple & Red

WEIGHT CAPACITY: 300 lb

SEAT HEIGHT: 17", 18" & 19"

SEAT DIMENSIONS: 14.5"w x 13.5"d

HANDLE HEIGHT ADJ: 28.5" - 32"

WIDTH BETWEEN HANDLES: 18.5"

OVERALL DIMENSIONS (HIGH): 24.25"w x 25"d

APPROX. USER HEIGHT: 4'10" - 5'4"

POUCH DIMENSIONS: 14.5"w x 8"d x 6.5"h

DIM. WHEN FOLDED: 24.75"w x 11.5"d x 37.25"h

WEIGHT: 15.5 lb



4288

STAR 8

The NOVA STAR shines with more features, options, colors and style. Its *Quick-Fit* adjustable design gives the user a perfect fit in just seconds, also making it so easy to pack up for travel, adventure and storage.



Scan here to watch our quick video and learn more!

20", 21" & 22"

Adjustable
Seat Height

COMES STANDARD

- Lock & Lift
- Locking Hand Brakes
- Quick-Fit* Push-button Adjustable
- Padded Seat w/ Under Seat Pouch
- Built-in Foot Pedals
- Oval-shaped Tubing
- Extended Folding Handle
- 8" Wheels

OPTIONS

- Cup Holder (CH-3200R)
- Cane Holder (CH-4200R)
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

Colors: Black, Blue, Diamond Sky Blue, Purple, Red

WEIGHT CAPACITY: 300 lb

SEAT HEIGHT: 20", 21" & 22"

SEAT DIMENSIONS: 14.5"w x 13.5"d

HANDLE HEIGHT ADJ: 31.75" - 38"

WIDTH BETWEEN HANDLES: 18.5"

OVERALL DIMENSIONS (HIGH): 24.5"w x 28"d

APPROX. USER HEIGHT: 5'6" - 6'2"

POUCH DIMENSIONS: 14.5"w x 8"d x 6.5"h

DIM. WHEN FOLDED: 24.5"w x 12"d x 44"h

WEIGHT: 18 lb



STAR 8 Petite

4278

The NOVA STAR shines with more features, options, colors and style. Its *Quick-Fit* adjustable design gives the user a perfect fit in just seconds; also making it so easy to pack up for travel, adventure and storage.



Scan here to watch our quick video and learn more!

18", 19" & 20"
Adjustable
Seat Height

COMES STANDARD

Lock & Lift
Locking Hand Brakes
Quick-Fit Push-button Adjustable
Padded Seat w/ Under Seat Pouch
Built-in Foot Pedals
Oval-shaped Tubing
Extended Folding Handle
8" Wheels

Colors: Blue, Purple & Red

OPTIONS

Cup Holder (CH-3200R)
Cane Holder (CH-4200R)
Designer Mobility Bags
Back and Seat Covers
Travel Bag

WEIGHT CAPACITY: 300 lb
SEAT HEIGHT: 18", 19" & 20"
SEAT DIMENSIONS: 14.5"w x 13.5"d
HANDLE HEIGHT ADJ: 29.5" - 33.25"
WIDTH BETWEEN HANDLES: 18.5"
OVERALL DIMENSIONS (HIGH): 24"w x 26"d
APPROX. USER HEIGHT: 4'11" - 5'5"
POUCH DIMENSIONS: 14.5"w x 8"d x 6.5"h
DIM. WHEN FOLDED: 24.5"w x 12.25"d x 38.5"h

WEIGHT: 17 lb



STAR HD

4289

The NOVA STAR HD comes standard with the same great features as the STAR 6 and STAR 8, plus a **large, sturdy seat** and a **450 lb weight capacity**.



Scan here to watch our quick video and learn more!

20", 21" & 22"
Adjustable
Seat Height

COMES STANDARD

Lock & Lift
Locking Hand Brakes
Quick-Fit Push-button Adjustable
Padded Seat w/ Under Seat Pouch
Built-in Foot Pedals
Oval-shaped Tubing
Extended Folding Handle
8" Wheels

Colors: Blue, Purple & Red

OPTIONS

Cup Holder (CH-3200R)
Cane Holder (CH-4200R)
Designer Mobility Bags
Back and Seat Covers
Travel Bag

WEIGHT CAPACITY: 450 lb
SEAT HEIGHT: 20", 21" & 22"
SEAT DIMENSIONS: 18.5"w X 15"d
HANDLE HEIGHT ADJ: 31.5" - 38"
WIDTH BETWEEN HANDLES: 22"
OVERALL DIMENSIONS (HIGH): 27.5"w x 28"d
APPROX. USER HEIGHT: 5'3" - 6'2"
POUCH DIMENSIONS: 17"w x 9"d x 6.5"h
DIM. WHEN FOLDED: 28"w x 13"d x 26.5" - 43.75"h

WEIGHT: 20 lb



STAR **HD** Petite

4279

The NOVA **STAR HD Petite** comes standard with the same great features as the **STAR 6** and **STAR 8**, plus a **large, sturdy seat** and a **450 lb weight capacity**.



Scan here to watch our quick video and learn more!

18", 19" & 20"
Adjustable
Seat Height

COMES STANDARD

Lock & Lift
Locking Hand Brakes
Quick-Fit Push-button
Adjustable
Padded Seat w/ Under
Seat Pouch
Built-in Foot Pedals
Oval-shaped Tubing
Extended Folding Handle
8" Wheels

Colors: Blue, Purple & Red

OPTIONS

Cup Holder
(CH-3200R)
Cane Holder
(CH-4200R)
Designer Mobility Bags
Back and Seat Covers
Travel Bag



WEIGHT CAPACITY: 450 lb

SEAT HEIGHT: 18", 19" & 20"

SEAT DIMENSIONS: 18.5"w x 15"d

HANDLE HEIGHT ADJ: 29.5" - 33.25"

WIDTH BETWEEN HANDLES: 22"

OVERALL DIMENSIONS (HIGH): 27.5"w x 26"h

APPROX. USER HEIGHT: 4'11" - 5'5"

POUCH DIMENSIONS: 17"w x 9"d x 6.5"h

DIM. WHEN FOLDED: 28"w x 12.25"d x 34.25" - 39"h



WEIGHT: 20 lb

4262

STAR **8** DX

The NOVA **STAR** shines with more features, options, colors and style. Its *Quick-Fit* adjustable design gives the user a perfect fit in just seconds, also making it so easy to pack up for travel, adventure and storage.



Scan here to watch our quick video and learn more!

**20", 21",
22" & 23"**
Adjustable
Seat Height

COMES STANDARD

Lock & Lift
Locking Hand Brakes
Quick-Fit Push-button
Adjustable
Seat w/ Built-in Seat Pad
Built-in Foot Pedals
Oval-shaped Tubing
Extended Folding Handle
8" Wheels

Colors: Blue, Purple & Red

OPTIONS

Cup Holder
(CH-3200R)
Cane Holder
(CH-4200R)
Designer Mobility Bags
Back and Seat Covers
Travel Bag

WEIGHT CAPACITY: 350 lb

SEAT HEIGHT: 20", 21", 22" & 23"

SEAT DIMENSIONS: 17"w x 12.75"d

HANDLE HEIGHT ADJ: 33" - 40.25"

WIDTH BETWEEN HANDLES: 18.75"

OVERALL DIMENSIONS (HIGH): 24"w x 29.25"d

APPROX. USER HEIGHT: 5'4" - 6'4"

POUCH DIMENSIONS: 16"w x 7.5"w x 6"h

DIM. WHEN FOLDED: 24"w x 12.5"d x 46.25"h



WEIGHT: 19 lb

The NOVA **STAR** shines with more features, options, colors and style. Its *Quick-Fit* adjustable design gives the user a perfect fit in just seconds; also making it so easy to pack up for travel, adventure and storage.



Scan here to watch our quick video and learn more!

20", 21",
22" & 23"

Adjustable
Seat Height

COMES STANDARD

- Lock & Lift
- Locking Hand Brakes
- Quick-Fit Push-button Adjustable
- Seat w/ Built-in Seat Pad
- Built-in Foot Pedals
- Oval-shaped Tubing
- Extended Folding Handle
- 8" Wheels

OPTIONS

- Cup Holder (CH-3200R)
- Cane Holder (CH-4200R)
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

Colors: Blue, Purple & Red



WEIGHT CAPACITY: 450 lb

WEIGHT: 19 lb

SEAT HEIGHT: 20", 21", 22" & 23"

SEAT DIMENSIONS: 17"w x 12.75"d

HANDLE HEIGHT ADJ: 33" - 40.25"

WIDTH BETWEEN HANDLES: 20.75"

OVERALL DIMENSIONS (HIGH): 26.25"w x 29"d

APPROX. USER HEIGHT: 5'4" - 6'4"

POUCH DIMENSIONS: 16"w x 7.5"w x 6"h

DIM. WHEN FOLDED: 26.25"w x 12.5"d x 46.25"h



The **EXPRESS** takes you on an adventure ride with All Terrain 10" and 8" wheels, strong, super stable and sporty design - and the Stand & Fold feature that makes the **EXPRESS** easy to fold, roll, carry and store.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Locking hand brakes
- Large padded sling seat
- Padded backrest
- Removable front pouch
- Foldable
- Attached Cane Holder
- 10" front wheels & 8" rear wheels

OPTIONS

- Designer Mobility Bags
- Travel Bag

Stands alone when folded

Colors: Black, Blue, Purple & Red

WEIGHT CAPACITY: 300 lb

WEIGHT: 15.5 lb

SEAT HEIGHT: 21.25"

SEAT DIMENSIONS: 18"w x 10"d

HANDLE HEIGHT ADJ: 33.25" - 38.25"

WIDTH BETWEEN HANDLES: 19"

OVERALL DIMENSIONS (HIGH): 25.25"w x 27"d

APPROX. USER HEIGHT: 5'4" - 6'2"

POUCH DIMENSIONS: 15"w x 9"d x 10"h

DIM. WHEN FOLDED: 28.25"w x 11"d x 38.25"h



The **Forté** is about strength, expertise and conviction. We know NOVA is the best and our new **Forté** is strong, beautiful and for the intentional. It stands when folded, carries your things, comes in two seat heights and makes a bold statement. *Fortissimo!*

4320
20.5"
Seat Height

4323
23.5"
Seat Height



Scan here to watch our quick video and learn more!



Stands on its own!

COMES STANDARD

- Locking Hand Brakes
- Front Pouch
- Foldable
- 8" Wheels
- Stands When Folded

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Travel Bag

Colors: Blue, Red

WEIGHT CAPACITY: 300 lb **WEIGHT:** 15 lb
SEAT HEIGHT: 20.5" (4320) & 23.5" (4323)
SEAT DIMENSIONS: 15"w x 9"d (4320) & 17.25"w x 9.5"d (4323)
HANDLE HEIGHT ADJUSTMENT: 30" - 34" (4320) & 31" - 37" (4323)
WIDTH BETWEEN HANDLES: 16.25" (4320) & 18.25" (4323)
OVERALL DIMENSIONS (HIGH): 23"w x 24.25"d (4320) & 24.75"w x 26.5"d (4323)
APPROX. USER HEIGHT: 5'2" - 5'9" (4320) & 5'4" - 6'2" (4323)
BAG DIMENSIONS: 12"w x 6.75"d x 9"h (4320) & 14"w x 6.75"d x 11"h (4323)
DIM. WHEN FOLDED: 9.25"w x 24.25"d x 30" - 34"h (4320) & 9.5"w x 26.5"d x 33" - 39.5"h (4323)

The **Traveler** will take you many places with its unique, **three wheel** maneuverable design.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Locking Hand Brakes
- Tray
- Basket
- Pouch
- Foldable
- 8" Wheels

OPTIONS

- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Travel Bag

Colors: Black, Blue, Purple, Red, Diamond Sky Blue

WEIGHT CAPACITY: 250 lb **WEIGHT:** 16.5 lb
HANDLE HEIGHT ADJUSTMENT: 33.25" - 37"
WIDTH BETWEEN HANDLES: 12"
OVERALL DIMENSIONS (HIGH): 23.5"w x 24.5"d
APPROX. USER HEIGHT: 5'4" - 6'2"
BASKET DIMENSIONS: 9.5"w x 8"d x 9.5"h
BAG DIMENSIONS: 11.75"w x 9"d x 11.75"h
TRAY DIMENSIONS: 9.25"w x 7.75"d
DIM. WHEN FOLDED: 26.5"w x 9.25"d x 33" - 37.25"h



Get the perfect fit and features with the **Zoom**. Choose your perfect seat height from **18"**, **20"**, **22"** and **24"** for your height, from petite to tall. The **Zoom** has the best features with 8" wheels, a large padded seat, and an under seat pouch.



Scan here to watch our quick video and learn more!

← 18"
Seat Height

COMES STANDARD

- Locking Hand Brakes
- Large Padded Seat
- Under Seat Pouch
- Flip-up Back
- Foldable
- 8" Wheels
- Lock N Load
- Heavy Duty Cable Covers

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

Colors: Blue, Pink, Purple, Red, Diamond Sky Blue

WEIGHT CAPACITY: 300 lb **WEIGHT:** 15.6 lb
SEAT HEIGHT: 18" **SEAT DIMENSIONS:** 13.75" w x 12.75" d
HANDLE HEIGHT ADJUSTMENT: 27.75" - 31.25"
WIDTH BETWEEN HANDLES: 18"
OVERALL DIMENSIONS (HIGH): 24.5" w x 25.25" d
APPROX. USER HEIGHT: 4'10" - 5'4"
POUCH DIMENSIONS: 15.5" w x 9.25" d x 6.25" h
DIM. WHEN FOLDED: 24.5" w x 12" d x 33.5" - 37.5" h



Get the perfect fit and features with the **Zoom**. Choose your perfect seat height from **18"**, **20"**, **22"** and **24"** for your height, from petite to tall. The **Zoom** has the best features with 8" wheels, a large padded seat, and an under seat pouch.



Scan here to watch our quick video and learn more!

← 20"
Seat Height

COMES STANDARD

- Locking Hand Brakes
- Large Padded Seat
- Under Seat Pouch
- Flip-up Back
- Foldable
- 8" Wheels
- Lock N Load
- Heavy Duty Cable Covers

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

Colors: Black, Blue, Green, Pink, Purple, Red, Diamond Sky Blue

WEIGHT CAPACITY: 300 lb **WEIGHT:** 16 lb
SEAT HEIGHT: 20" **SEAT DIMENSIONS:** 13.75" w x 12.75" d
HANDLE HEIGHT ADJUSTMENT: 30.25" - 33.5"
WIDTH BETWEEN HANDLES: 18"
OVERALL DIMENSIONS (HIGH): 24.5" w x 26" d
APPROX. USER HEIGHT: 5'2" - 5'8"
POUCH DIMENSIONS: 15.5" w x 9.25" d x 6.25" h
DIM. WHEN FOLDED: 24.5" w x 12.25" d x 35" - 40.25" h



Get the perfect fit and features with the **Zoom**. Choose your perfect seat height from **18", 20", 22"** and **24"** for your height, from petite to tall. The **Zoom** has the best features with 8" wheels, a large padded seat, and an under seat pouch.



Scan here to watch our quick video and learn more!

← 22"
Seat Height

COMES STANDARD

Locking Hand Brakes
Large Padded Seat
Under Seat Pouch
Flip-up Back
Foldable
8" Wheels
Lock N Load
Heavy Duty Cable Covers

Colors: Black, Blue, Green, Pink, Purple, Red, Diamond Sky Blue

OPTIONS

Cup Holder
Flashlight Attachment
Cane Holder
Designer Mobility Bags
Back and Seat Covers
Travel Bag

WEIGHT CAPACITY: 300 lb

WEIGHT: 16.25 lb

SEAT HEIGHT: 22" **SEAT DIMENSIONS:** 13.75"w x 12.75"d

HANDLE HEIGHT ADJUSTMENT: 31.75" - 36"

WIDTH BETWEEN HANDLES: 18"

OVERALL DIMENSIONS (HIGH): 24.5"w x 26.25"d

APPROX. USER HEIGHT: 5'5" - 6'1"

POUCH DIMENSIONS: 15.5"w x 9.25"d x 6.25"h

DIM. WHEN FOLDED: 24.5"w x 12.5"d x 37" - 42"h



Get the perfect fit and features with the **Zoom**. Choose your perfect seat height from **18", 20", 22"** and **24"** for your height, from petite to tall. The **Zoom** has the best features with 8" wheels, a large padded seat, and an under seat pouch.



Scan here to watch our quick video and learn more!

← 24"
Seat Height

COMES STANDARD

Locking Hand Brakes
Large Padded Seat
Under Seat Pouch
Flip-up Back
Foldable
8" Wheels
Lock N Load
Heavy Duty Cable Covers

Colors: Blue, Red

OPTIONS

Cup Holder
Flashlight Attachment
Cane Holder
Designer Mobility Bags
Back and Seat Covers
Travel Bag

WEIGHT CAPACITY: 300 lb

WEIGHT: 16.25 lb

SEAT HEIGHT: 24" **SEAT DIMENSIONS:** 13.75"w x 12.75"d

HANDLE HEIGHT ADJUSTMENT: 33.5" - 38"

WIDTH BETWEEN HANDLES: 18"

OVERALL DIMENSIONS (HIGH): 24.5"w x 27.25"d

APPROX. USER HEIGHT: 5'5" - 6'2"

POUCH DIMENSIONS: 15.5"w x 9.25"d x 6.25"h

DIM. WHEN FOLDED: 24.5"w x 12.75"d x 39.25" - 44"h



The **GetGO Standard** (formerly the GetGO Classic) is a true classic with **excellent features** at a **nice value**. It is equipped with 6" wheels, a large padded seat and an under seat pouch.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Locking Hand Brakes
- Large Padded Seat
- Under Seat Pouch
- Flip-up Back
- Foldable
- 6" Wheels
- Lock N Load
- Heavy Duty Cable Covers

Colors: Black, Blue, Green, Pink, Purple, Red, Diamond Sky Blue

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

WEIGHT CAPACITY: 300 lb

WEIGHT: 14.1 lb

SEAT HEIGHT: 22.25" **SEAT DIMENSIONS:** 13.75"w x 12.75"d

HANDLE HEIGHT ADJUSTMENT: 32.25" - 36.5"

WIDTH BETWEEN HANDLES: 17.75"

OVERALL DIMENSIONS (HIGH): 24.25"w x 23"d

APPROX. USER HEIGHT: 5'4" - 6'1"

POUCH DIMENSIONS: 15.5"w x 9.25"d x 6.25"h

DIM. WHEN FOLDED: 24.5"w x 11.5"d x 37.5" - 42.25"h



We love the new **GetGO Petite** (formerly the GetGO Junior) because it has the same great features as the 4202C with a **lower 18.5" seat height**.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Locking Hand Brakes
- Large Padded Seat
- Under Seat Pouch
- Flip-up Back
- Foldable
- 6" Wheels
- Lock N Load
- Heavy Duty Cable Covers

Colors: Blue, Purple & Red

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

WEIGHT CAPACITY: 300 lb

WEIGHT: 13.75 lb

SEAT HEIGHT: 18.5" **SEAT DIMENSIONS:** 13.75"w x 12.75"d

HANDLE HEIGHT ADJUSTMENT: 28.5" - 32.75"

WIDTH BETWEEN HANDLES: 17.75"

OVERALL DIMENSIONS (HIGH): 24.25"w x 23.5"d

APPROX. USER HEIGHT: 4'10" - 5'4"

POUCH DIMENSIONS: 15.5"w x 9.25"d x 6.25"h

DIM. WHEN FOLDED: 24.25"w x 11.5"d x 34.5" - 39.5"h



GetGO Petite Narrow

4208C

The **GetGO Petite Narrow**, (formerly the GetGO Petite) ideal for petite users, will get you on the go with **great features** at a **great value**.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Locking Hand Brakes
- Large Padded Seat
- Under Seat Pouch
- Flip-up Back
- Foldable
- 6" Wheels
- Lock N Load
- Heavy Duty Cable Covers

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

Colors: Blue, Pink, Purple & Red

WEIGHT CAPACITY: 300 lb

SEAT HEIGHT: 18.5"

SEAT DIMENSIONS: 12"w x 12"d

HANDLE HEIGHT ADJUSTMENT: 29" - 33.25"

WIDTH BETWEEN HANDLES: 15.75"

OVERALL DIMENSIONS (HIGH): 22"w x 23.75"d

APPROX. USER HEIGHT: 4'10" - 5'4"

POUCH DIMENSIONS: 15.5"w x 9.25"d x 6.25"h

DIM. WHEN FOLDED: 22"w x 11.25"d x 35" - 40"h



4235

Vibe 6S (Steel Frame)

The **Vibe 6S** comes standard with all the **Vibe's** excellent features at a great value - and now with a sturdy, yet lightweight steel frame.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Steel Frame
- Locking Hand Brakes
- Large Padded Seat
- Under Seat Pouch
- Flip-up Back
- Foldable
- 6" Wheels

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

Colors: Black, Blue, Purple, Red

WEIGHT CAPACITY: 300 lb

SEAT HEIGHT: 22.25"

SEAT DIMENSIONS: 13.5"w x 12.5"d

HANDLE HEIGHT ADJUSTMENT: 32.5" - 38"

WIDTH BETWEEN HANDLES: 18"

OVERALL DIMENSIONS (HIGH): 24"w x 26.5"d

APPROX. USER HEIGHT: 5'4" - 6'1"

POUCH DIMENSIONS: 14"w x 8.25"d x 6.25"h

DIM. WHEN FOLDED: 24"w x 12.5"d x 38.5" - 44.25"h



Journey

4205

Life's a trip! Let the Journey take you on the ride of your life. This walker truly celebrates personal mobility with dual height adjustments for a custom fit and color choices that are super cool and sizzling hot.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Steel Frame
- Locking Hand Brakes
- Large Padded Seat
- Under Seat Pouch
- Flip-up Back
- Height Adjustable
- Foldable
- 6" Wheels

Colors: Black, Blue, Purple, Red

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

WEIGHT CAPACITY: 275 lb

WEIGHT: 18.5 lb

SEAT HEIGHT: 18.75" & 21.5"

SEAT DIMENSIONS: 13.75"w x 12.5"d

HANDLE HEIGHT ADJ: 29.75" - 35" (low)
32.5" - 37.75" (high)

WIDTH BETWEEN HANDLES: 18"

OVERALL DIMENSIONS (HIGH): 24"w x 27.75"d

APPROX. USER HEIGHT: 4'9" - 6'0"

POUCH DIMENSIONS: 14"w x 8.5"d x 6.5"h

DIM. WHEN FOLDED: 24"w x 12.25"d x 35.75" - 44.75"h



4216

Mighty Mack

The **strongest of them all** is the **Mighty Mack** with a **500 pound weight capacity** and a standard seat height for **average to tall** users. Now with more durable, longer lasting wheels.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Locking Hand Brakes
- Extra Wide Plastic Seat with built-in rubber seat pad
- Front Wire Basket
- Removable Back
- Foldable
- 8" Wheels

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Tray
- Travel Bag

Color: Blue



WEIGHT CAPACITY: 500 lb

WEIGHT: 24.2 lb

SEAT HEIGHT: 22"

SEAT DIMENSIONS: 19.75"w x 13.25"d

HANDLE HEIGHT ADJUSTMENT: 30.25" - 36.25"

WIDTH BETWEEN HANDLES: 24"

OVERALL DIMENSIONS (HIGH): 28"w x 26.25"d

APPROX. USER HEIGHT: 5'5" - 6'0"

BASKET DIMENSIONS: 17.25"w x 10"d x 6.5"h

DIM. WHEN FOLDED: 28.75"w x 26.5" - 27.5"d x 31" - 36.75"h



FEATURES

NOVA's **Phoenix** rollator offers some exciting features ...



BUILT-IN CANE HOLDER

Attached cane holder keeps canes conveniently close



CONTOURED PADDED ARMRESTS

Padded armrests provide stability and support



QUICK-ADJUST BUTTON

Orange *Quick-Adjust* Button for easy adjustment of handle height



STABILIZATION STRAP

Secures the handle bars



FOOT PEDALS

Built-in Foot Pedals to aid with curbs, terrain and folding



PADDED SLING SEAT WITH PADDED SEAT HANDLES

Comfortable padded sling seat (with built-in handle for easy folding) and padded seat handles provide support while seated

We love the new **Phoenix Rise UP Rollator** because it gives your body the vehicle to Rise Up & Look Forward. The unique design makes walking happen ... with an upright, forward looking posture, optimum control and stability and much reduced body strain and fatigue.



Scan here to watch our quick video and learn more!

COMES STANDARD

Contoured armrests
Locking hand brakes
Large padded sling seat
Quick-Adjust handle height
Removable front pouch
Foldable
Attached Cane Holder
10" front wheels & 8" rear wheels

OPTIONS

Designer Mobility Bags
Travel Bag

Colors: Blue, Purple, Red

WEIGHT CAPACITY: 300 lb

WEIGHT: 20.5 lb

SEAT HEIGHT: 21.5"

SEAT DIMENSIONS: 18"w x 9"d

HANDLE HEIGHT ADJ: 48" - 53"

WIDTH BETWEEN HANDLES: 19"

OVERALL DIMENSIONS (HIGH): 25"w x 33"d x 53.25"h

APPROX. USER HEIGHT: 5'2" - 6'0"

POUCH DIMENSIONS: 15"w x 9"d x 10"h

DIM. WHEN FOLDED: 12"w x 33"d x 49.25" - 53.25"h



Cruiser Classic

4200C

The **Cruiser Classic** allows you to cruise along with a **weight activated** brake system. This walker has 5" front and rear wheels and **DOES NOT** have locking hand brakes for those unable to use them. Moderate-to-good stability is required to use the weight-activated brake system, which is non-locking.



COMES STANDARD

- Weight-activated Drag Brake (non-locking)
- Padded Seat
- Under Seat Pouch
- Flip-up Back
- Foldable
- 5" Wheels
- Lock N Load

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag

Color: Red

WEIGHT CAPACITY: 250 lb

SEAT HEIGHT: 21.25"

SEAT DIM: 12"w x 12"d

HANDLE HEIGHT ADJUSTMENT: 31.75" - 36.25"

WIDTH BETWEEN HANDLES: 17.75"

OVERALL DIMENSIONS (HIGH): 24.5"w x 22.5"

APPROX. USER HEIGHT: 5'3" - 6'0"

POUCH DIMENSIONS: 15.5"w x 9.25"d x 6.25"h

DIM. WHEN FOLDED: 24.5"w x 10.75"d x 37.75" - 42.5"h



4201

Cruiser II

The **Cruiser II** is ideal for beginning **gait training** with 5 inch front wheels and rear rubber tips to prevent the walker from rolling away. The rolling walker frame and seat provide the stability, safety and confidence to establish a natural walking pattern.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Padded Seat
- Under Seat Pouch
- Flip-up Back
- Foldable
- 5" Wheels
- Lock N Load

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Back and Seat Covers
- Travel Bag
- Glide Skis (40026GR)

Color: Blue

WEIGHT CAPACITY: 250 lb

SEAT HEIGHT: 20.75"

SEAT DIM: 12"w x 12"d

HANDLE HEIGHT ADJUSTMENT: 31" - 35.25"

WIDTH BETWEEN HANDLES: 17.5"

OVERALL DIMENSIONS (HIGH): 21.5"w x 21"d

APPROX. USER HEIGHT: 5'3" - 6'0"

POUCH DIMENSIONS: 15.5"w x 9.25"d x 6.25"h

DIM. WHEN FOLDED: 21.5"w x 10" - 10.25"d x 36.25" - 41.25"h



The **Cruiser De-Light** is a real **pleasure!** It is the only 4-wheeled rolling walker that folds and has adjustable legs like a standard folding walker with the added benefits of locking hand brakes and a seat.



Scan here to watch our quick video and learn more!

COMES STANDARD

- Locking Hand Brakes
- Plastic seat with built-in rubber seat pad
- Foldable
- 5" Wheels

OPTIONS

- Cup Holder
- Flashlight Attachment
- Cane Holder
- Designer Mobility Bags
- Basket* (Item #4010B)
- Tray
- Travel Bag

Colors: Blue, Red *Basket not included

WEIGHT CAPACITY: 250 lb **WEIGHT:** 16.25 lb
SEAT HEIGHT: 19.5" - 23.5" **SEAT DIM.:** 15.25"w x 10.5"d
HANDLE HEIGHT ADJUSTMENT: 34.25" - 38.25"
WIDTH BETWEEN HANDLES: 18.75"
OVERALL DIMENSIONS (HIGH): 28"w x 23.75"d
APPROX. USER HEIGHT: 5'4" - 6'2"
BASKET DIMENSIONS: 19.5"w x 7"d x 7.5"h
DIMENSIONS WHEN FOLDED: 28"w x 9"d x 35" - 39"h



ROLLATOR ACCESSORIES

For NOVA rollators with **oval-shaped frames**



- CH-1101, CH-3200R and CH-4200R are made exclusively for NOVA rollators with oval-shaped frames
- Fits the following models:
 - STAR series
 - Express
 - Phoenix



Item # CH-1101

ADJUSTABLE CUP HOLDER


- Adjustable arms fit cup widths from 2.25" to 3.75" diameter
- Compact design - folds flat when not in use
- Easy to install (requires Phillips screw driver)



Item # CH-3200R

SWIVEL CUP HOLDER

- Adjustable arms fit cup widths from 2.6" to 3.75" diameter
- Swing-away design for easy transport
- Fits through tight spaces
- Easy to install (requires Phillips screw driver)

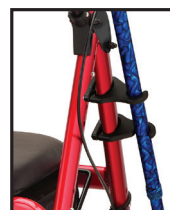

Item # CH-4200R

CANE HOLDER

- Allows user to bring cane along without having to carry it
- Cane snaps easily in and out of holder (fits 7/8" diameter canes)
- Easy to install (requires Phillips screw driver)

CUP HOLDERS

Item # CH-1000

Deluxe Cup Holder

- Attaches easily to walkers, wheelchairs, bed rails and transport chairs
- Adjusts to hold different sized items
- Folds up when not in use
- Easy to install (requires Phillips screw driver)



Item # CH-3000R

Swivel Cup Holder

- Attaches easily to walkers, wheelchairs, bed rails and transport chairs
- Adjusts to hold different sized items
- Swivels inward to fit through tight spaces
- Easy to install (requires Phillips screw driver)



CANE HOLDER

Item # CH-4000R



- Cane (with 7/8" diameter) snaps easily in and out of the holder

FLASHLIGHT

- Adds extra light for safety in dark places
- Fits any mobility device with 7/8"-1" tubing including bikes and strollers
- Rotates 360 degrees



Item # FL-2000

TRAVEL BAG

- Fits all folding and rolling walkers (except 4216)
- Nylon bag, perfect for airline travel or storage
- 34"w x 26.5"h x 9.25"d



Item # 4000TB

WALKER TRAY

- Carries food, drinks, and various personal belongings
- Fits 4202, 4207, 4010, 4214, & 4215



Item # 4000T

HANGING WALKER POUCHES

- Fit all folding and 4200 series walkers
- Velcros to front of crossbar of folding walker or backrest of 4200 series
- Extra storage for lightweight objects



Butterflies

Garden Flowers

Item # 4001WP (Black)

Item # P4200SB (Aloha Blue)

Item # 4002GF (Garden Flowers)

Item # 4002PB (Butterflies)

UNIVERSAL MOBILITY BAGS

- Item # 4009BK (Black)
- Item # 4008BB (Boho Blossoms)
- Item # 4008GA (Enchanted Garden)
- Item # 4008EG (English Garden)



MOBILITY HANDBAG

- Item # 4006BK (Black)



Discover the most beautiful and functional mobility style in the world. Yes, you have style and you deserve to look beautiful.

BACK AND SEAT COVERS

- Add glam, add style, add luxurious feel
- Make it all about you. Stand out from the crowd and make a statement!



Item # 4007PB (Butterflies)



Item # 4007BP (Black Puma)



Item # 4007GD (Grey Dusk)



Item # 4007GF (Garden Flowers)



Item # 4007EG (English Garden)



Item # 4007AB (Aloha Blue)



Item # 4007BB (Boho Blossoms)



Item # 4007AW (Anchors Aweigh)

Rolling Walker Color Guide

Aluminum



Red



Pink



Purple



Green



Black



Blue



Diamond Sky Blue



Black



Blue



Purple



Red

CARE • QUALITY • INNOVATION



THE NOVA PROMISE

CONTACT US!
We're here to help

✉ info@novajoy.com
☎ 1 800 557 6682

I love my NOVA

Here's why:

- #1 Recommended Walker
- Most Diverse Walker Line
- World Class Service

WARNING

- Lock hand brakes before sitting and while seated
- Seat is for stationary sitting only. Do not ambulate or use as a transport chair while seated
- Do not use on stairs, escalators, or moving walkways
- Backrest must be attached to the walker prior to operating
- Consult your user manual before operating




as unique as you are
1-800-557-6682
novajoy.com

Authorized NOVA Dealer