



nova

# 379

Parts Breakdown  
*For models: 379B, 379BK, 379R*



## An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your transport chair corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (**see Figure 1**).

If you need further assistance, please call our customer service team at **(800) 557-6682**.

**Thank you!**



Figure 1

# Handle and Folding Mechanism Components

CH-37711



CH-37712



CH-37713



CHT-37742



CH-37743



CH-37902



CH-37903



CHT-37746



CHT-37747




CH-37726



CH-37727



CHT-37744

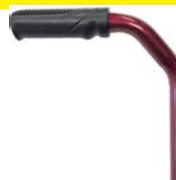

CHT-37745



CH-37729



CH-37724



CH-37715




CHT-31954




CH-37710




# Main Body Components

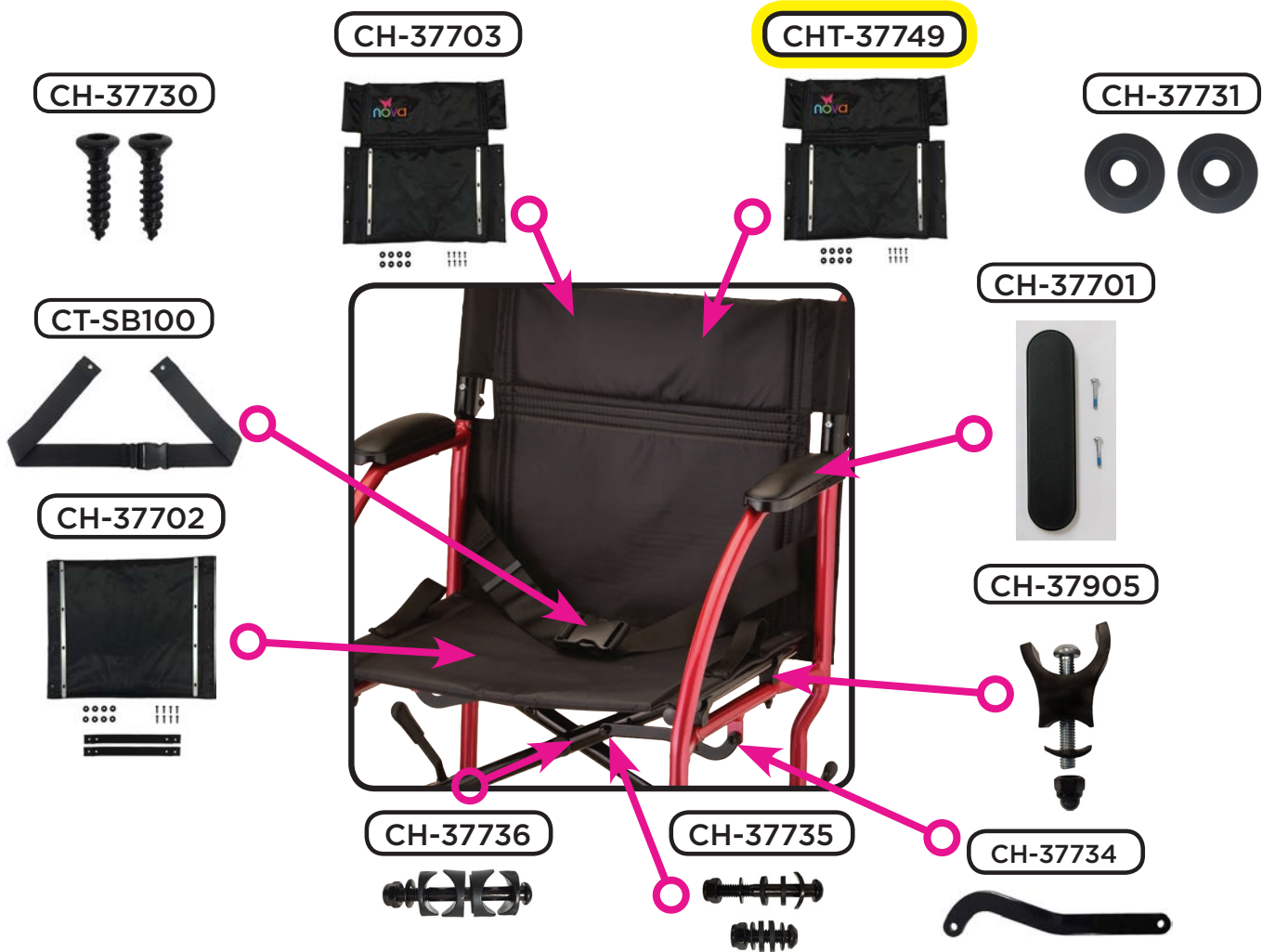


# Footrest Assembly

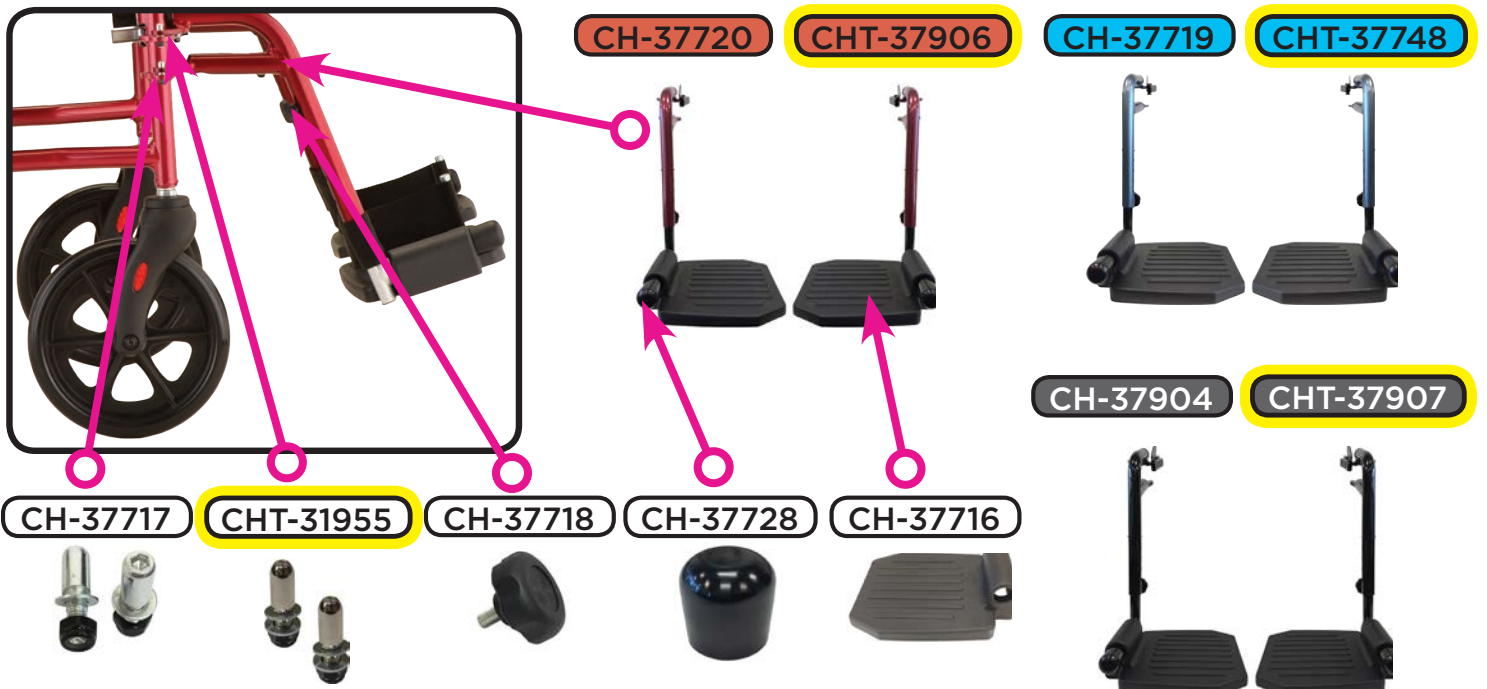

## Front Wheel Assembly

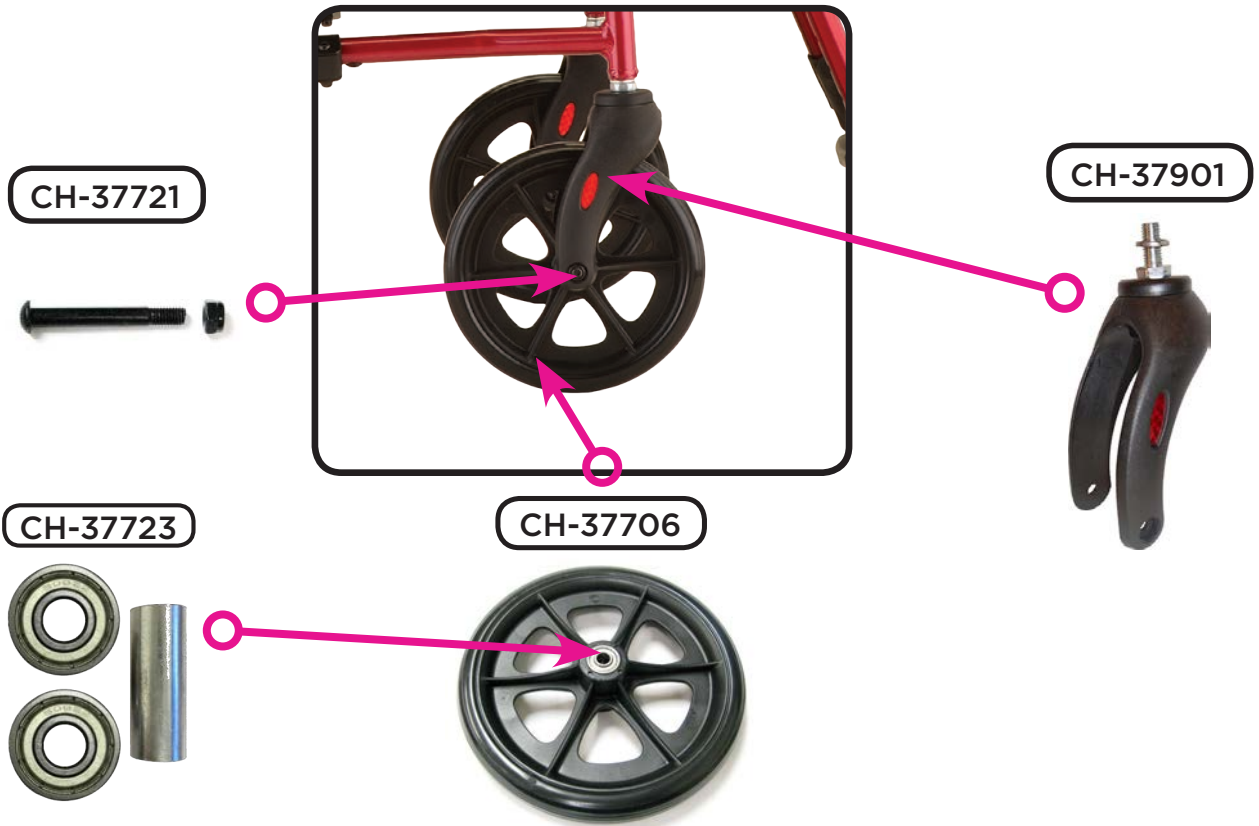

## Rear Wheel Assembly

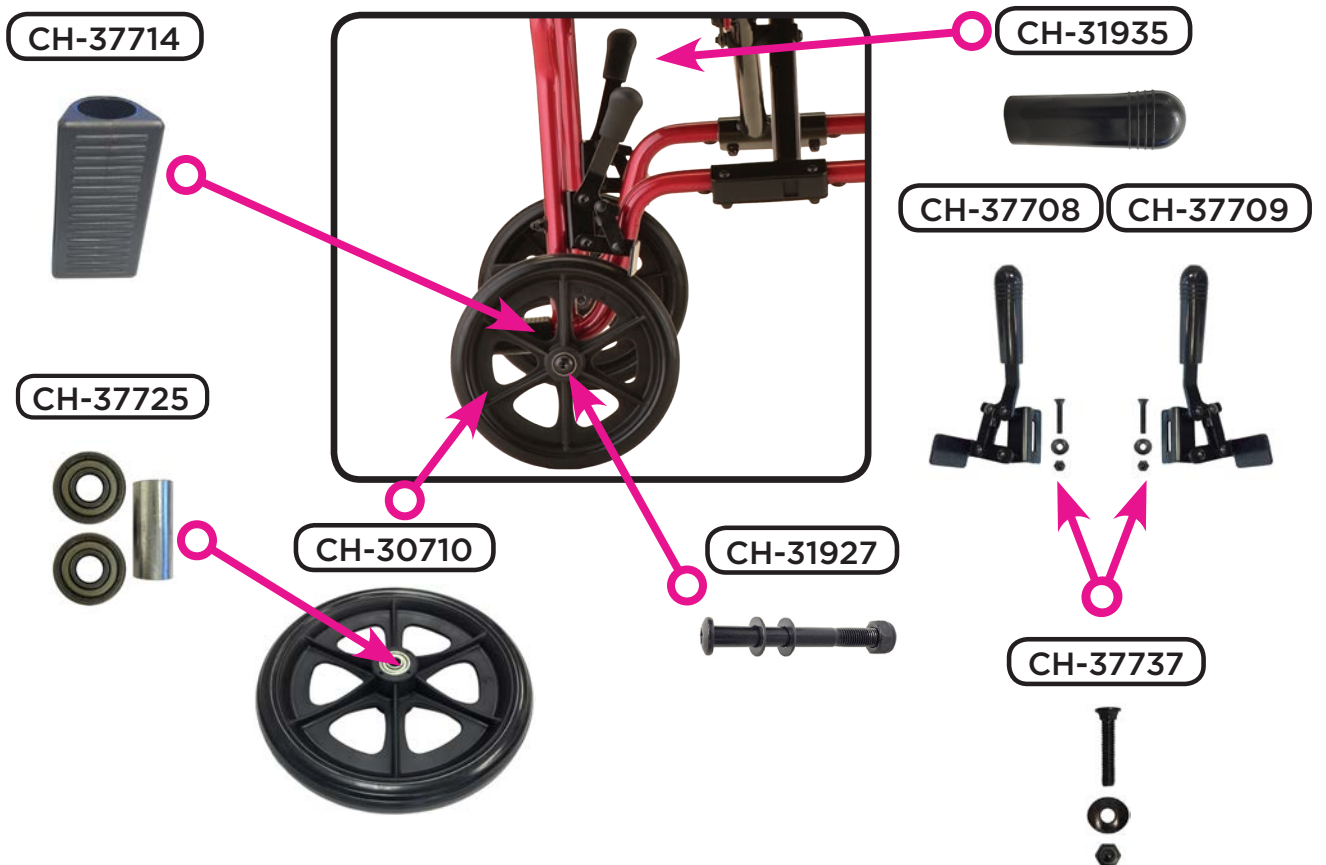




Item Number 379

### 379 Transport Chair

SERIAL # BEGINNING WITH "CHT" & "CH"

ITEM NUMBER	DESCRIPTION
CH-31927	AXLE SET FOR REAR WHEEL
CH-37701	ARM PAD FOR 377/379 SN:CH
CH-37721	AXLE FOR FRONT WHEEL 377/379
CH-37723	BEARING & SPACER FOR FRONT WHEEL
CH-37708	BRAKE LEFT 307/309/319/327/329
CH-37709	BRAKE RIGHT 307/309/319/327/329
CH-37728	CAP FOR FOOT REST
CH-37905	CRADLE FOR SEAT FOR 377/379
CH-37733	CROSSBAR SUPPORT FOR 377/379
CH-37734	CROSSBAR SUPPORT FOR 377/379
CH-37710	FOLDING MECHANISM
* CHT-31954	FOLDING MECHANISM (UPPER)
CH-37716	FOOT PLATE FOR 377/379 SN:CH
CH-37719	FOOT REST FOR 379B
CH-37904	FOOT REST FOR 379BK
CH-37720	FOOT REST FOR 379R SN:CH
* CHT-37748	FOOTRESTS (PAIR) FOR 377/379 BLUE
* CHT-37907	FOOTRESTS (PAIR) FOR 379 BLACK
* CHT-37906	FOOTRESTS (PAIR) FOR 379 RED
CH-37901	FORK FOR FRONT WHEEL 379
CH-37713	GRIP FOR HANDLE BAR
CH-37711	HANDLE BAR LEFT FOR 377B/379B
CH-37902	HANDLE BAR LEFT FOR 379BK
CH-37726	HANDLE BAR LEFT FOR 379R
CH-37712	HANDLE BAR RIGHT FOR 377B/379B
CH-37903	HANDLE BAR RIGHT FOR 379BK
CH-37727	HANDLE BAR RIGHT FOR 379R
* CHT-37742	HANDLE LEFT FOR 377/379 BLUE
* CHT-37746	HANDLE LEFT FOR 379 BLACK
* CHT-37744	HANDLE LEFT FOR 379 RED
* CHT-37743	HANDLE RIGHT FOR 377/379 BLUE
* CHT-37747	HANDLE RIGHT FOR 379 BLACK
* CHT-37745	HANDLE RIGHT FOR 379 RED
CH-37718	KNOB WITH SCREW FOR FOOTREST
CH-37724	LEVER FOR FOLDING MECHANISM
* CHT-31955	PIN (PAIR) FOR FOOTREST

	Item Number 379
	<b>379 Transport Chair</b>
	SERIAL # BEGINNING WITH "CHT" & "CH"
<b>ITEM NUMBER</b>	<b>DESCRIPTION</b>
CH-37717	PIN FOR FOOTREST
CH-37729	RELEASE PIN FOR FOLDING MECH
CH-37730	SCREW - UPHOLSTERY/HAND BRAKE
CH-37737	SCREW FOR BRAKE
CH-37736	SCREW FOR CROSSBAR 377/379
CH-37715	SCREW FOR FOLDING MECHANISM
CH-37735	SCREWS FOR CROSSBAR SUPPORT
CT-SB100	SEAT BELT FOR ALL TRANSPORT CH
CH-31935	TIP FOR BRAKE HANDLE
CH-37714	TOE KICK 307/309/319/327/329/3
* CHT-37749	UPHOLSTERY BACK FOR 377/379
CH-37703	UPHOLSTERY BACK FOR 377/379 S
CH-37702	UPHOLSTERY SEAT FOR 377/379 S
CH-37731	WASHER FOR UPHOLSTERY SCREW
CH-37706	WHEEL FRONT
CH-30710	WHEEL REAR

\*

USE "CHT" PARTS FOR SERIAL NUMBERS STARTING WITH "CHT"	
AND	
USE "CHT" PARTS FOR "CH" SERIAL NUMBERS STARTING WITH:	
<b>ITEM NUMBER</b>	<b>SERIAL NUMBER</b>
379B	CH379BGH0237
379BK	CH379BKGH0001
379R	CH379RGH0167

**CONTACT US!**  
We're here to help



[info@novajoy.com](mailto:info@novajoy.com)

1 800 557 6682

[novajoy.com](http://novajoy.com)

CARE • QUALITY • INNOVATION



THE NOVA PROMISE