







Parts Breakdown 8800 - Shower Chair/Commode

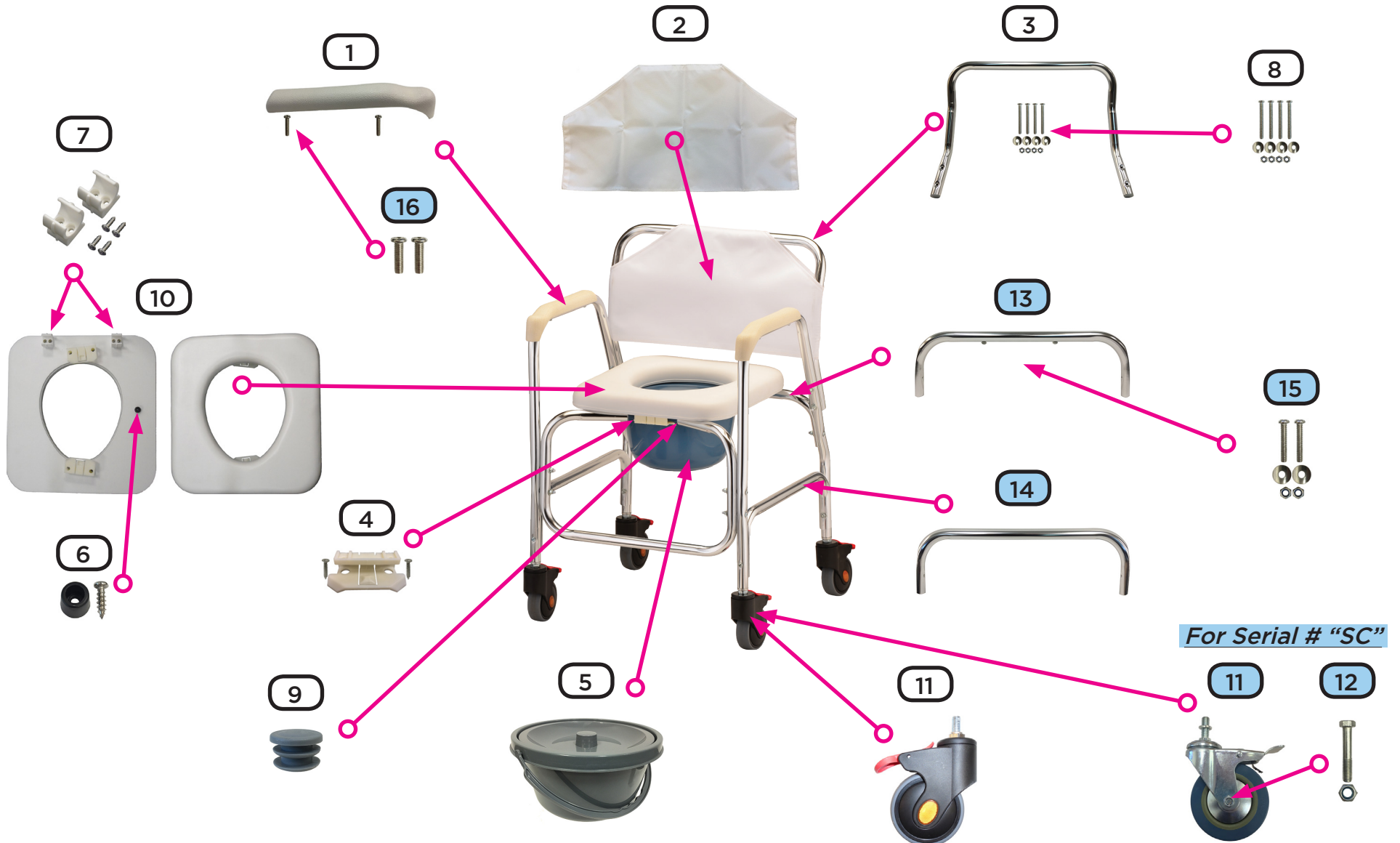



		ITEM # 8800
		COMMODE/ SHOWER CHAIR W/ WHEELS
		SERIAL # BEGINNING WITH "HF"
#	ITEM	DESCRIPTION
1	HF-88004	ARM PAD AND SCREWS FOR 8800 SN: HF
2	P88001	BACK UPHOLSTERY FOR 8800
3	HF-88005	BACKREST AND SCREWS FOR 8800 SN: HF
4	HF-88006	BRACKET FOR 8800 BUCKET SN: HF
5	SC-88001	BUCKET & LID-8800/8582/8900/89
6	SC-88005	BUMPER & SCREW FOR 8800 SEAT
7	HF-88003	CLAMP SET FOR SEAT FOR 8800
8	SC-88010	HARDWARE LONG SCREW FOR 8800
9	SC-88011	PLUG FOR 8800/8900/8901 FRAME
10	HF-88002	SEAT PADDED FOR 8800
11	HF-88001	WHEEL WITH FORK FOR 8800

		ITEM # 8800
		COMMODE/ SHOWER CHAIR W/ WHEELS
		SERIAL # BEGINNING WITH "SC"
#	ITEM	DESCRIPTION
1	SC-88012	ARM PAD & SCREWS FOR 8800/8200
2	P88001	BACK UPHOLSTERY FOR 8800
3	SC-88009	BACKREST & HARDWARE FOR 8800
4	SC-88002	BRACKET FOR 8800 BUCKET
5	SC-88001	BUCKET & LID-8800/8582/8900/89
6	SC-88005	BUMPER & SCREW FOR 8800 SEAT
7	SC-88004	CLAMP & SCREW FOR SEAT
8	SC-88010	HARDWARE LONG SCREW FOR 8800
9	SC-88011	PLUG FOR 8800/8900/8901 FRAME
10	SC-88003	SEAT COMPLETE FOR 8800
11	SC-88006	WHEEL ASSEMBLY FOR 8800
12	SC-88008	AXLE & NUT FOR 8800 WHEEL
13	SC-88014	FRAME REAR FOR 8800 SEAT
14	SC-88015	FRAME SIDE FOR 8800
15	SC-88016	HARDWARE SHORT SCREW FOR 8800
16	SC-88013	SCREW (PAIR) FOR 8800/8582 ARM

		ITEM # 8800
		COMMODE/ SHOWER CHAIR W/ WHEELS
		SERIAL # BEGINNING WITH "NT"
#	ITEM	DESCRIPTION
1	P88004	ARM REST FOR 8800 (INCLUDES 2
2	P88001	BACK UPHOLSTERY FOR 8800
3	P88003	BACK AND HARDWARE FOR 8800
4	P8809	PAIL HOLDER FOR 8800/8801
5	P89001	BUCKET WITH LID FOR 8900/8801/
7	P88002	SEAT CLAMP FOR 8800/8801
8	P8800BH	HARDWARE FOR 8800 BACK
10	P88011	PADDED SEAT FOR 8800/8801
11	P8800WR	WHEEL FOR 8800 & 8801 - SWIVEL

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682
novajoy.com

CARE • QUALITY • INNOVATION



THE NOVA PROMISE