



# nova

# Star 8

Parts Breakdown

*For models: 4258BK, 4258BL, 4258DB, 4258PL, 4258RD*



*\*Revised on March 1, 2018*

## An important note from



Many parts listed and/or pictured on this breakdown may appear to be identical to the part(s) required but may actually be different, due to slight dimensional changes.

Please verify that the serial number on your rolling walker corresponds with the serial number series denoted in the below parts list. The serial number is located on a sticker that is attached to the frame under the seat (**see Figure 1**).


If you need further assistance, please call our customer service team at **(800) 557-6682**.

**Thank you!**




Figure 1

# Hand Brake Mechanism

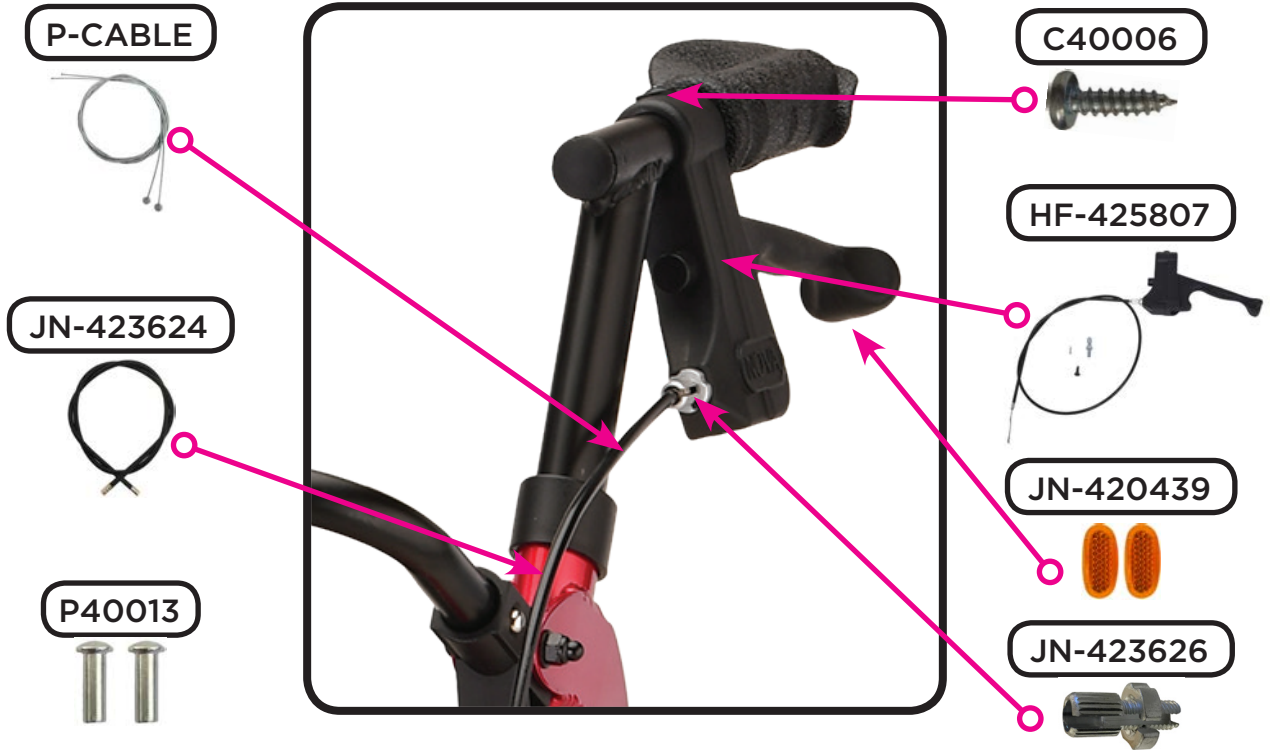


# Upper Assembly Components

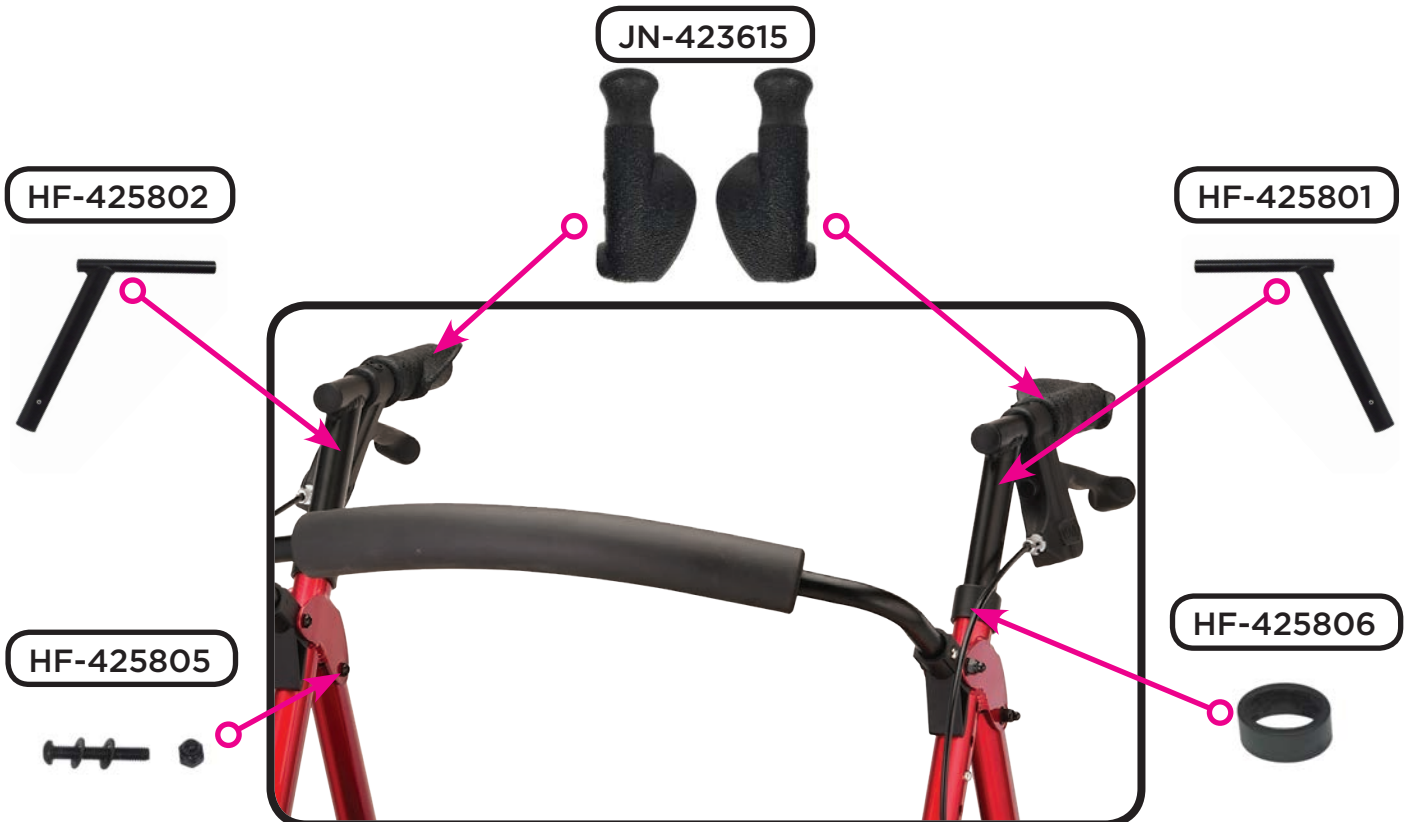


## Seat & Backrest



## Lower Assembly Components

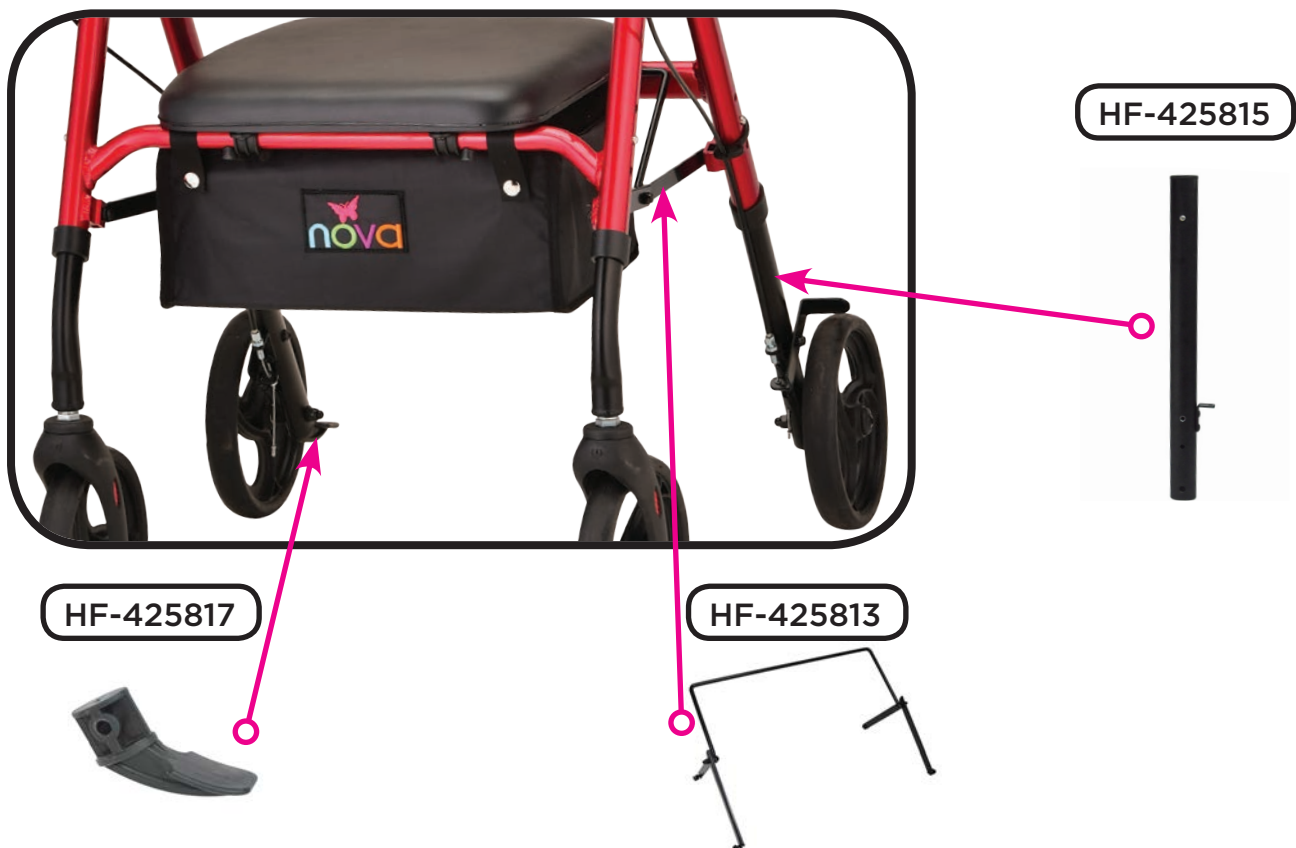

# Fork & Wheel Assembly

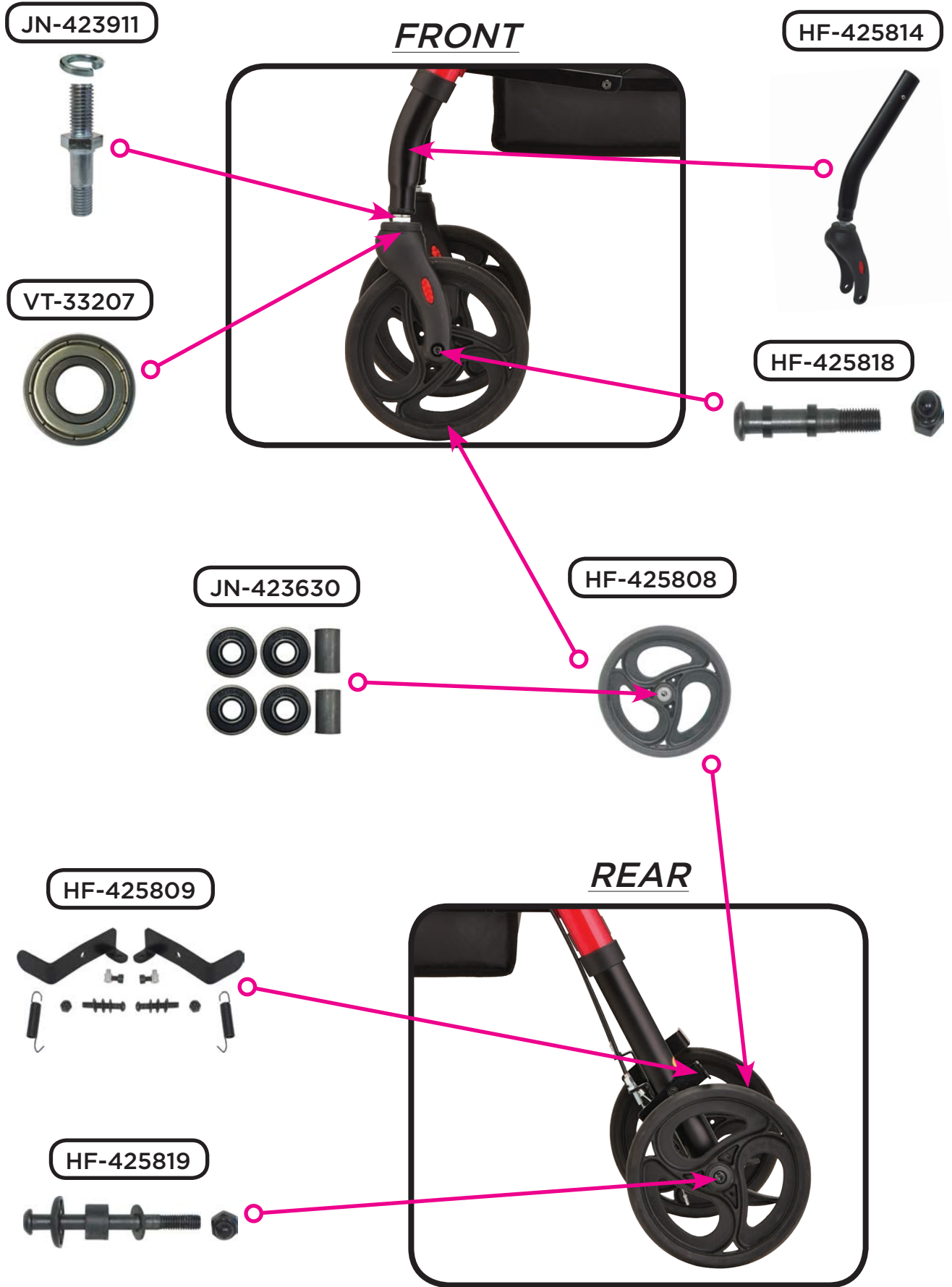



Item Number 4258

## Star 8

SERIAL # BEGINNING WITH "HF"

ITEM NUMBER	DESCRIPTION
HF-425818	AXLE FRONT FOR 4258
HF-425819	AXLE REAR FOR 4258
HF-425803	BACKREST FOR 4258
HF-425810	BAG FOR 4258
P-CABLE	BRAKE CABLE (PAIR)
HF-425821	BRAKE CABLE HOLDER FOR 4258
HF-425809	BRAKE SHOE (SET) FOR 4258
JN-423624	CABLE COVER BLACK
HF-425812	CLAMP & SCREW (SET) FOR SEAT
P40013	CRIMP FOR STEEL CABLE (PAIR)
HF-425813	FOLDING MECHANISM FOR 4258
JN-423615	GRIPS (PAIR) FOR 4204/4205/4206/4236/4237/4238/4258
HF-425807	HAND BRAKE ASSEMBLY FOR 4258
HF-425801	HANDLE LEFT FOR 4258
HF-425802	HANDLE RIGHT FOR 4258
HF-425804	HINGE BACKREST LEFT FOR 4258
HF-425820	HINGE BACKREST RIGHT FOR 4258
HF-425814	LEG FRONT ASSEMBLY FOR 4258
HF-425815	LEG REAR FOR 4258
HF-425805	SCREW A-FRAME AND BACKREST HINGE FOR 4258
HF-425811	SEAT PADDED FOR 4258
HF-425806	SILENCER FOR FRAME FOR 4258
HF-425817	TOE KICK FOR LEG FOR 4258
HF-425808	WHEEL 8" FOR 4258

**CONTACT US!**  
We're here to help



[info@novajoy.com](mailto:info@novajoy.com)



1 800 557 6682

[novajoy.com](http://novajoy.com)

CARE • QUALITY • INNOVATION



THE NOVA PROMISE