






Parts Breakdown 8450 - Bedside Commode

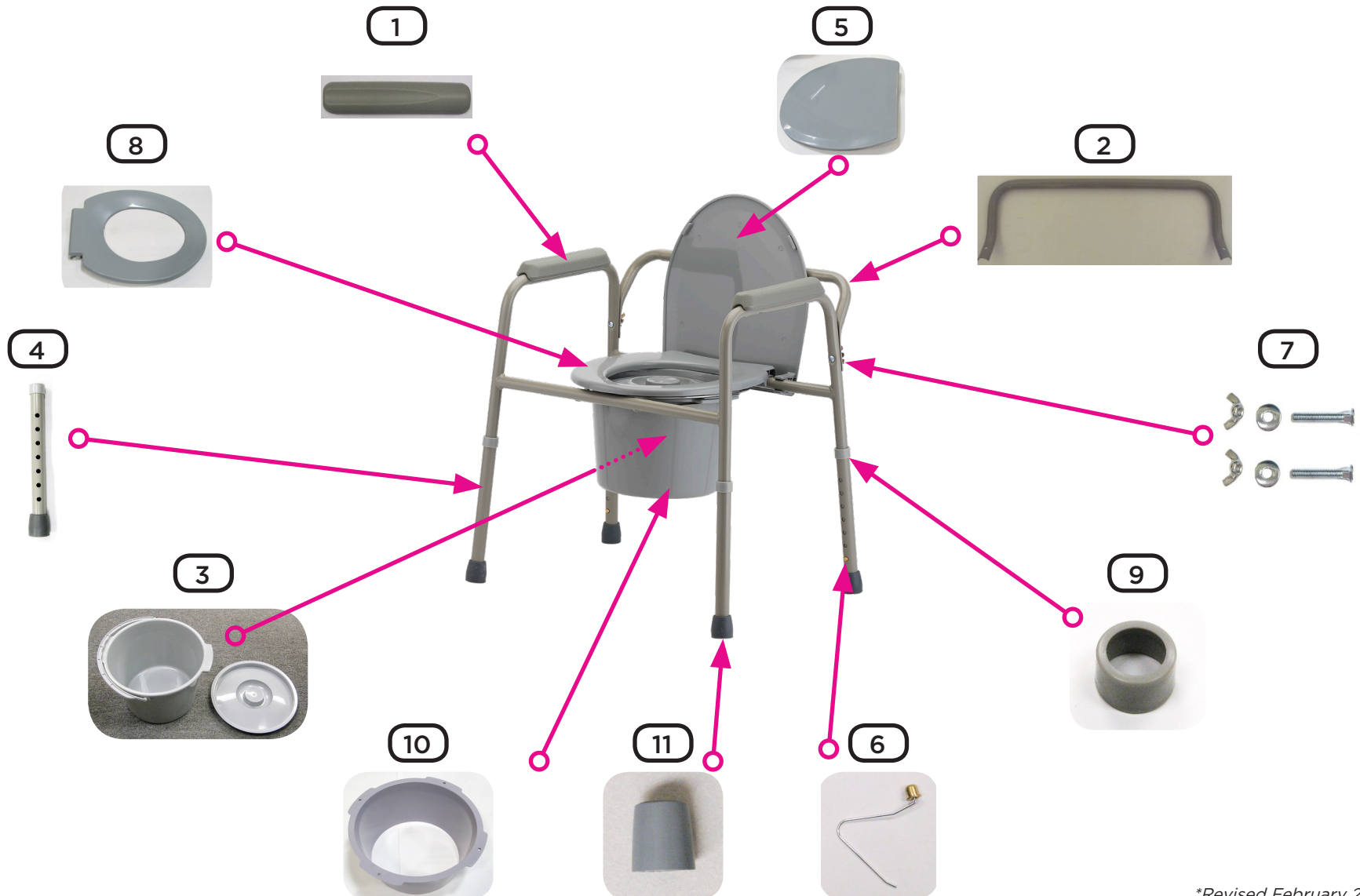

		ITEM # 8450
		BEDSIDE COMMODE
		SERIAL # BEGINNING WITH "XI"
#	ITEM	DESCRIPTION
1	X84504	ARM PAD FOR 8450/8700
2	X84502	BACK FOR 8450
3	8450B	BUCKET W/LID: 8450/8700
4	X84503	LEG FOR 8450/8700
5	X8450L	LID FOR 8450
6	X84501	PUSH BUTTON FOR 8450/8700 LEG
7	X-84505	SCREW & WING NUT W/WASHER
8	X8450A	SEAT FOR 8450
9	XI-87006	SILENCER FOR 8450/8700
10	8450S	SPLASH GUARD FOR 8450/8700
11	X84505	TIP FOR 8450/8700 LEG

		ITEM # 8450
		BEDSIDE COMMODE
		SERIAL # BEGINNING WITH "HF & SC"
#	ITEM	DESCRIPTION
1	SC-87005	ARM PAD & SCREWS FOR 8450/8700
2	SC-845001	BACK REST FOR 8450
3	8450B	BUCKET WITH LID
4	SC-87006	LEG FOR 8450/8700
5	SC-845003	LID FOR 8450
6	SC-858207	PUSH BUTTON FOR 8450/8700/8582/8583/8900/8901
7	X-84505 (PAIR)	SCREW & WING NUT W/WASHER
8	SC-845002	SEAT FOR 8450
9	SC-858209	SILENCER SMALL (SEE NOTES)
10	8450S	SPLASH GUARD FOR 8450/8700
11	X84505	TIP FOR 8450/8700 LEG

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

novajoy.com

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE