

HOW TO ASSEMBLE YOUR



BEDSIDE COMMODE

Item # 8450

Capacity not to exceed 350 lb



NO tools required
Quick assembly

Easy to follow
step-by-step
instructions

PARTS INCLUDED

- a. (1) Frame w/ backrest
- b. (1) Seat

- c. (1) Lid
- d. (1) Bucket w/ lid
- e. (1) Splash guard



a.



b.




c.



d.



e.

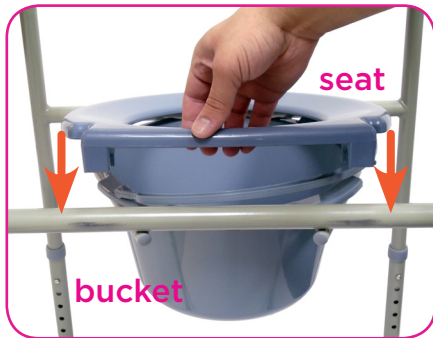

1

Place bucket into the bucket platform. Place the toilet seat onto the frame so that plastic clamps rest on the rear cross frame and on each side of the bucket platform. Push down forcefully until brackets are fully attached to the rear cross frame.




2

Ensure toilet seat is in the down position before attaching the lid. Place the lid onto the frame so that plastic clamps rest on the rear cross frame and on each side of the toilet seat brackets. Push down forcefully until brackets are fully attached to the rear cross frame.



3

Toilet seat and lid should now be completely attached and are able to swing up and down without detaching.

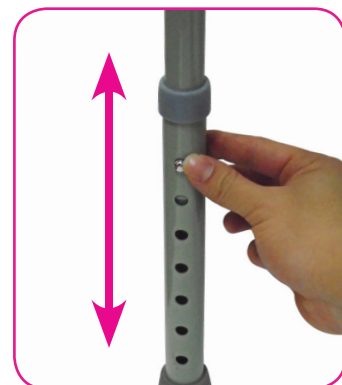

4

Adjust the height of the legs by locating the push button on each leg and sliding the leg up or down to the desired height. **NOTE: Ensure all legs are fully engaged and adjusted to equal heights before using.**

