

HOW TO ASSEMBLE YOUR

nova

SHOWER CHAIR/COMMUNE WITH WHEELS & BACK

Item # 8800


Capacity not to exceed 250 lb



QUICK ASSEMBLY

Follow these SIMPLE STEPS

TOOLS NEEDED


10mm wrench


Phillips screwdriver

WHAT'S IN THE BOX

a. (1) Frame

b. (1) Bucket & Lid

c. (1) Backrest

d. (1) Backrest upholstery

e. (4) Sets of screws, nuts, & washers



a.



b.



c.




d.



e.

1

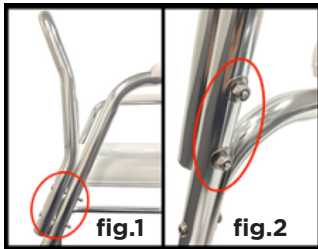

On the back end of the chair where the arms slope at an angle, locate 4 holes, 2 on each side of the frame. Slide the 4 screws through the backrest and frame as shown in figure 1. On the threaded end of the screw, slide on all 4 washers and tighten all 4 nuts as shown in figure 2.

2

Slide backrest upholstery onto backrest starting from the bottom opening of the upholstery.



3



While facing the front of the chair, slide the bucket under the seat through the left hand side until the bucket hits the rubber bumper on the other end of the seat. The lip of the bucket must slide in between the front and rear brackets and the bucket handle should face either the front or rear of seat, as shown.

HOW TO USE YOUR SHOWER CHAIR/COMMUNE

1 Before using the chair, lock all wheels by depressing the hanging end of lever above the wheel. Once the user is seated and stable, unlock the wheels by depressing the raised end of lever, and roll the commode to the desired location.

NOTE: Be especially careful and do not move too quickly over any threshold, mat/carpet, or other uneven surfaces to avoid tipping the user forward.



2 Lock the wheels before the user seated in the chair attempts to stand up.

3 After use, remove the bucket, discard waste and sanitize.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts	One Year Limited Warranty (rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.


IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

- Be sure seat is in place and secure before each use.
- Do not stand on seat.

CONTACT US!
We're here to help


 info@novajoy.com


 1 800 557 6682


novajoy.com


Connect with us!

novajoy.com

 [novamedicalprod](#)

 [novamedicalprod](#)

 [novawalkerstalk](#)

 [NOVA Medical Products](#)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE